Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) Italy
gima@gimaitaly.com
export@gimaitaly.com
www.gimaitaly.com

REF 27735 - 27737
27740 - 27741

Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) Italy
Made in Italy

IT - Manuale d'uso
GB - User instructions
SI - Navodila za uporabo
ES - Instrucciones de usuario
PT - Instruções de montagem
GR - Αλπτροθήκη ηνταίερας
DK - Brugervejledning
PL - Instrukcja Użytkowania
FR - Instructions de fonctionnement
NL - Gebruikershandleiding
DE - Gebrauchsanweisungen

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GB - Installation instructions

Attach the 2 x L-shaped brackets to the Gima raised toilet seat: turn the model upside down on a flat surface and insert the L-shaped brackets into the slots. Push gently on the bracket and turn the knob clockwise until the thread engages.

Position the brackets leaving a greater width than that of the toilet bowl. Lift up existing toilet seat and place the Gima seat directly onto the toilet bowl positioning the brackets towards the rear and the lip against the front of the bowl.

Turn the adjustment knobs clockwise until a resistance is felt and the seat does not move from side to side. Ensure seat is firmly fixed before using.

ATTENTION! The side-locks have been especially designed to allow for easy tightening therefore turning should not be continued once a resistance is felt.

Cleaning Instructions

All parts can be cleaned with warm water and a neutral cleanser. Alternatively, the seat can be totally immersed in water and/or can be sanitized (without brackets) in an autoclave at 85°C for 1 minute.

Cautions

Do not exceed maximum user weight: 225kg /496lb Always ensure the toilet seat is securely fixed before using.

Do not hold onto seat for support while rising/lowering

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