**for.me.sa.**  
forniture medico sanitarie

# Ciambella gonfiabile in gomma

## Inflatable rubber ring

# DELUXE

 **For. Bedsore aids**

### CIAMBELLA GONFIABILE IN GOMMA - DELUXE

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 Carico massimo  
130 Kg  
**MAX**

### INFLATABLE RUBBER RING - DELUXE

Sturdy and comfortable, it is suitable to relieve bedridden patients or people just had a haemorrhoid operation.

#### INSTRUCTION FOR USE

- Before using the device, clean carefully the external surface with tepid water and neutral soap.
- Do not use to clean any abrasive substance or different cleaner from what specifically mentioned.
- Dry and be careful about the inflation valve.
- Inflate the device using the pump (if it is included in the pack) or using a manual compressor (like a bicycle pump).
- Repeat this process as needed. Inflate moderately.
- Excessive air inflations could cause breakage and irreversible damages.
- Put the device on a plane surface before the use.
- At the end of the use, deflate, clean and store the device, away from dust and light sources.

#### WARNINGS

To avoid choking, keep the bag out of reach of babies and children.  
Use it only on intact skin and following the instructions written on this package insert.  
Every different use from what mentioned, has to be considered not correct.  
Store the device at ambient temperature, away from heat sources and dust. Dispose in compliance with the current regulation.  
In case of allergic reactions, discontinue use of the product and contact your doctor.  
You must report any serious accidents due to the use of the medical device, either to the Manufacturer neither to the competent authority of the Member State in which you are based.

**REF** 70-508.01

**LOT** RRS/170

 2027/03/31

**UDI**

(01) 08055060760472  
(17) 270331  
(10) RRS/170

Ciambella Ø 35 cm

A922883804