



GIMA

PROFESSIONAL MEDICAL PRODUCTS

WALLACH® PAPETTE (NON STERILE)
WALLACH® PAPETTE (NON STERILE)
WALLACH® PAPETTE (NON STÉRILE)
WALLACH® PAPETTE (NO ESTÉRIL)
WALLACH® PAPETTE (NÃO ESTÉRIL)
WALLACH® PAPETTE (ΜΗ ΑΠΟΣΤΕΙΡΩΜΕΝΟ)

REF

908004 (29738)



CooperSurgical, Inc.
95 Corporate Drive
Trumbull, CT 06611 USA
Made in USA

EC REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Importato da / Imported by / Importé par /
Importado por / Importado por / Εισαγωγή από:
Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com





Read all safety information and instructions listed below before using this product.

R_xOnly CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Application:

The WALLACH® PAPETTE, a product of CooperSurgical, is the most gentle, yet effective means of simultaneously taking both ectocervical and endocervical cell samples.



Caution: Be sure not to force the shorter bristles into the cervical os to prevent unintended separation of the brush from the handle.

For Cervical Cytology utilizing the Conventional Pap Smear

Instructions:

1. Contact the cervix with the WALLACH® PAPETTE and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. Be sure not to force the shorter bristles into the cervical os to prevent unintended separation of the brush from the handle.
2. While maintaining gentle pressure in the direction of the cervix, rotate the brush two or more times in either direction.
3. Remove the WALLACH® PAPETTE and draw each side once across the slide (like a paint brush), thus transferring both ectocervical and endocervical cell samples in one stroke.
4. Dispose of WALLACH® PAPETTE in accordance with local Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the WALLACH® PAPETTE is being used to obtain samples.

For Cervical Cytology utilizing the ThinPrep® Pap Test

Instructions:

1. Contact the cervix with the WALLACH® PAPETTE and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. Be sure not to force the shorter bristles into the cervical os to prevent unintended separation of the brush from the handle.
2. While maintaining gentle pressure in the direction of the cervix, rotate the brush two or more times in either direction.
3. Rinse the brush into the PreservCyt® solution for use with the ThinPrep® Pap Test by pushing the brush into the bottom of the vial 10 times, forcing the bristles apart.
Swirl the brush vigorously to further release material. Discard the collection device.
4. Tighten the PreservCyt® vial cap so that the torque line on the cap passes the torque line on the vial.
5. Dispose of WALLACH® PAPETTE in accordance with Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the WALLACH® PAPETTE is being used to obtain samples.

PreservCyt® and ThinPrep® are registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and other countries.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

	IT Codice prodotto GB Product code FR Code produit ES Código producto PT Código produto GR Κωδικός προϊόντος
	IT Numero di lotto GB Lot number FR Numéro de lot ES Número de lote PT Número de lote GR Αριθμός παρτίδας
	IT Fabbrikante GB Manufacturer FR Fabricant ES Fabricante PT Fabricante GR Παραγωγός
	IT Dispositivo medico conforme alla Direttiva 93/42/CEE GB Medical Device complies with Directive 93/42/EEC FR Dispositif médical conforme à la directive 93/42 / CEE ES Dispositivo médico segun a la Directiva 93/42 / CEE PT Dispositivo médico em conformidade com a Diretiva 93/42/CEE GR Ιατρική συσκευή σύμφωνα με την οδηγία 93/42 / CEE
	IT Rappresentante autorizzato nella Comunità europea GB Authorized representative in the European community FR Représentant autorisé dans la Communauté européenne ES Representante autorizado en la Comunidad Europea PT Representante autorizado na União Europeia GR Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Ένωση
	IT Dispositivo monouso, non riutilizzare GB Disposable device, do not re-use FR Dispositif pour usage unique, ne pas réutiliser ES Dispositivo monouso, no reutilizable PT Dispositivo descartável, não reutilizar GR Προϊόν μιας χρήσεως. Μην το χρησιμοποιείται εκ νέου.
	IT Non utilizzare se l'imballaggio è danneggiato GB Don't use if package is damaged FR Ne pas utiliser si le colis est endommagé ES No usar si el paquete está dañado PT Não use se o pacote estiver danificado GR Μην το χρησιμοποιείτε αν η συσκευασία είναι κατεστραμμένη
	IT Latex free GB Latex free FR Sans latex ES Sin látex PT Isento de látex GR Χωρίς λάτεξ
	IT Leggere le istruzioni per l'uso GB Consult instructions for use FR Consulter les instructions d'utilisation ES Consultar las instrucciones de uso PT Consulte as instruções de uso GR Διαβάστε προσεκτικά τις οδηγίες χρήσης
	IT Non sterile GB Non-sterile FR Pas stérile ES No estéril PT Não estéril GR όχι αποστειρωμένο