**FORBICI PER BENDE  
BANDAGE SCISSORS  
CISEAUX À BANDAGES  
TIJERAS PARA VENDAS  
VERBANDSCHERE  
TESOURAS PARA LIGADURAS****ITALIANO**

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**ENGLISH**

The blades of bandage scissors are made of 2Cr13 stainless steel, the handles of PP.

The blades of these scissors have been milled in order to cut clothes, gauzes and bandages.

**Bandage scissors are suitable for:**

- Cutting medical gauze
- Cutting bandages away from skin

**Warning**

Remember that scissors may cause harm if not handled with care.

Please pay due attention and care during use.

Choose a safe place for storing the scissors, out of reach of non-authorized personnel.

Clean the scissors after use and eliminate any residue.

Keep the scissors closed when not in

use. Any equipment showing signs of wear should be discarded immediately.

**Instructions for use**

Firmly grip the scissors, cut starting from the front of the body. Do not cut laterally, pay attention in order not to damage the patient's skin.

**Sterilisation:** autoclave at 138°C

**Warranty**

The warranty is valid for 12 months. During this period of time we will repair and/or replace free of charge all defective parts due to ascertained production causes, excluding any labour, travel and transport costs, packaging costs etc. Damages caused by an improper use of the product are not covered by the warranty.

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Gima S.p.A.