# Welch Allyn Wall and Mobile, Sphygmomanometers

## DuraShock™ Handheld, Pocket, Integrated Aneroid Sphygmomanometers, and Aneroid Sphygmomanometer with Integrated Gauge Adapter

Models DS44, DS45, DS44A, DS45A, DS48, DS48A, DS54, DS55, DS56, DS58, DS65, DS66, and 767

### Directions for use

# 723560, DIR 8002076 Ver. B, Revised 2015-05


**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician or licensed healthcare practitioner. This device should be used by trained personnel.

### Standards and symbols



AAMI/ANSI/ISO 80601-2:2007 (R) 2013 Non-invasive sphygmomanometers - Part 1: Requirements and test methods for non-automated measurement type

<b>REF</b>	Product identifier Identifiant du produit Identificador del producto Produkt-ID Identificativo del producto Produktidentifikativ Produkt-ID Produkt ID Tünternehmern Produktidentifikation Avroykupmē mālojums Identifikator del producto Ürün kimliği Kapetřikatop kāpēja Identifikator produktu Identifikator prozvoda Tootenumber 型番 제품 ID
#	Reorder number Numéro de commande Nº de reposición Nachbestellnummer Número per il riordino Bestellnummer Bestellungsnummer Bestellungsnummer Apējus mānojumķīkīs tāpēriņš Número de encendida Yeniden şıyanı numarası Hove norşörçənə zaxşa Numer ponownego zamówienia Broj ponovne narudžbe Kordstellnummber 追加注文番号 재주문 번호

### FlexPort Connection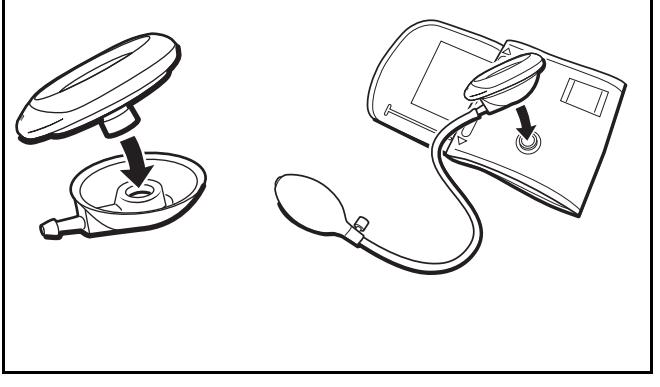

### Pocket Aneroid Connection

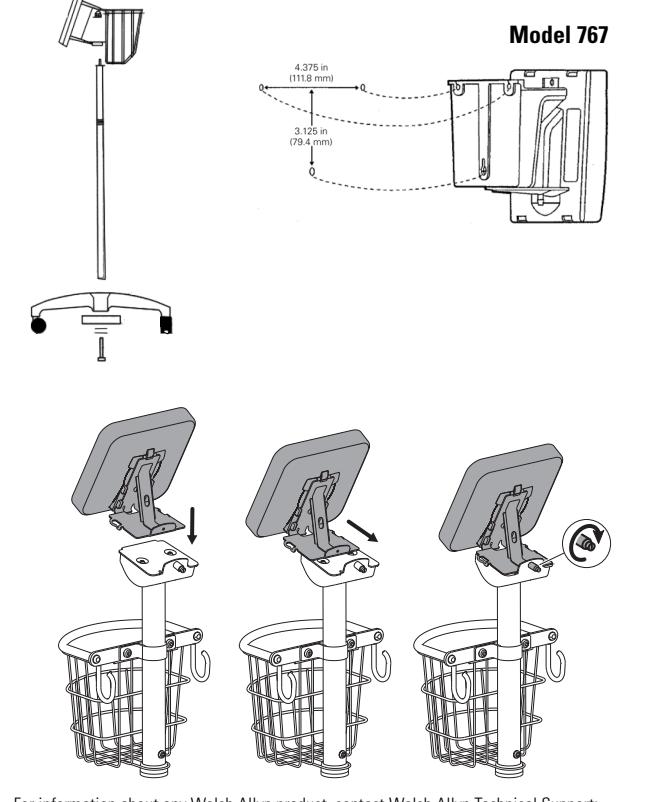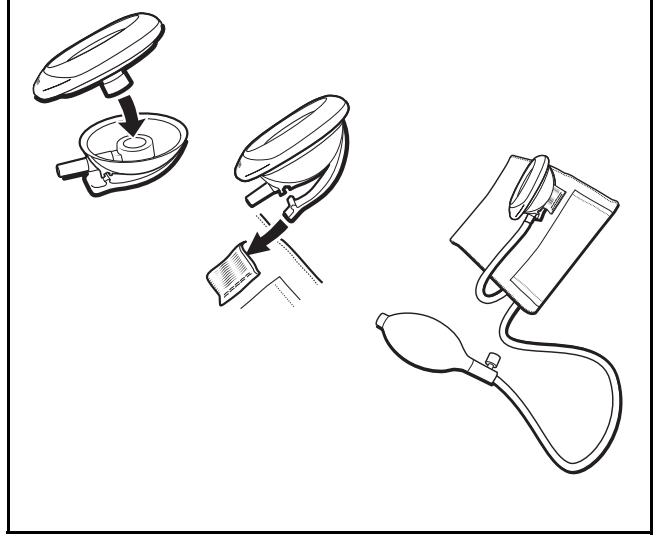For information about any Welch Allyn product, contact Welch Allyn Technical Support: [www.welchallyn.com/ex/about-us/locations.html](http://www.welchallyn.com/ex/about-us/locations.html).

 Welch Allyn Inc.  
4341 State Street Road  
Skaneateles Fall, NY 13153 USA  
Tel: 800-535-6663 (USA only) or 315-685-4560 • Fax: 315-685-3361 • [www.welchallyn.com](http://www.welchallyn.com)

The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

Regulatory Affairs Representative, Welch Allyn Limited, Navan Business Park, Dublin Road, Navan, County Meath, Republic of Ireland

For patent information, visit [www.welchallyn.com/patents](http://www.welchallyn.com/patents).

Gauge models tested: DS44, DS45, DS45T, 2253, DS48, DS48A, DS-54xx-300, DS-5602-300, DS58, DS66, 7670-015

© 2015 Welch Allyn, Inc. All rights reserved. To support the intended use of the product described in this publication, the purchase of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn® and DuraShock™ are trademarks of Welch Allyn, Inc.

## ENGLISH

### Introduction

#### Intended use

Welch Allyn non-automated sphygmomanometers are noninvasive blood pressure measurement devices intended for use in conjunction with an inflatable cuff and stethoscope to determine blood pressure in pediatric through adult patients.

#### Contraindications

Welch Allyn non-automated sphygmomanometers are contraindicated for neonate use.

#### Warnings and cautions

The warning statements in this manual identify conditions or practices which, if not corrected or discontinued immediately, could lead to patient injury, illness, or death.

**WARNING:** Safety and effectiveness with neonate cuffs (sizes from neo 1 to neo 5) is not established.

**WARNING:** Possible measurement error. Use only Welch Allyn blood pressure cuffs and accessories; substitution may result in measurement error.

**WARNING:** Inaccurate measurement risk. Ensure an airtight seal at all connection points before use. Excessive leaking may affect readings.

**WARNING:** No modification of this equipment is allowed.

#### Operation

Blood pressure measurements can be affected by the position of the patient and their physiologic condition. Before beginning a procedure, ensure that the patient rests for at least five minutes; has support of their back and feet; and does not cross their legs. Passively support the patient's lower arm and keep the upper arm at heart level. The procedure needs to take place in a quiet environment with no talking. Failure to follow these recommendations can result in inaccurate blood pressure measurements.

1. Select cuff size appropriate for the patient's arm circumference. The applicable range, in centimeters, is printed on each cuff.

**NOTE:** The "Artery Index Marker" on the cuff should fall within the "Range" indicated on FlexPort cuffs. The "Index Edge" on the cuff should fall within the "Range" indicated on Two-Piece Reusable cuffs. If the artery index marker or index edge falls short of the range, use a larger cuff to ensure accurate results. If the artery index marker or index edge is past the range, use a smaller cuff to ensure accurate results.

2. Wrap the cuff around the arm with the artery index marker located over the brachial artery and with the lower edge of the cuff 2.5 cm above the bend in the elbow.

3. Inflate cuff rapidly to a level 30 mm Hg above estimated (or palpatory) systolic pressure.

4. Partially open the valve to allow deflation at a rate of 2 to 3 mm Hg per second. As the pressure falls, note systolic pressure and diastolic pressure detected with your stethoscope.

5. Rapidly release the remaining pressure and record measurements immediately. After a minimum of 30 seconds, repeat the above steps for a second reading.

#### Operational Guidelines

The operator should have a direct view, and be within a meter, of the gauge during measurement.

For manual blood pressure measurements, K5 is recommended for adult patients and K4 for pediatric patients ages 3 to 12. For pregnant females, clinician should use K5, unless sounds are audible with a deflated cuff, in which case K4 should be used.

**NOTE:** K5 is the point at which the Korotkoff sounds cannot be heard. K4 is the change in the tones heard from a clear sound to a muffled sound.

#### Specifications

##### Technical specifications

These technical specifications are applicable to the device models listed in this Directions for use:

Welch Allyn products listed in these instructions for use rely on bulb inflation/deflation system in combination with 0, 1, or 2 tube cuff attachment systems.

Sphygmomanometers are accurate to  $\pm 3$  mm Hg.

Operational Range: 0 to 300 mmHg

Displayed Range: 0 to 300 mmHg

Indication type: Radial Scale

Scale graduation: Increments of 2 mmHg

Pointer movement: No stop pin

Tube adapter: Integrated, 1, or 2 tube adapter

Pressure generation: Bulb

Pressure reduction: Air release valve

##### Environmental specifications

Storage temperature: -20 °C to 70 °C (-4 °F to 158 °F)

Storage relative humidity: 15% to 95% (noncondensing)

Operating temperature: 10 °C to 40 °C (50 °F to 104 °F)

Operating relative humidity: 15% to 90% (noncondensing)

##### Maintenance

Comply with regional law when non-automated sphygmomanometer or accessories are discarded.

##### Calibration Check of Aneroid Sphygmomanometer

###### Quick Check of Calibration:

At zero pressure, make certain the pointer is within the acceptable tolerance banding of the pressure gage on the dial. If the pointer is fully outside of the tolerant zone, shown in the illustration as the dark shaded area, the device may need calibration. Although an unpressurized reading of zero does not guarantee accuracy at all scale points, failure of the pointer to indicate zero ( $\pm 3$  mm Hg) is an obvious sign of error.

###### Full Check of Calibration:

Welch Allyn recommends a full check of calibration at least every two years according to local law and after maintenance and repair. Use the following procedure:

1. Connect the instrument under test to a high quality, known pressure standard and a 150 to 500 cc test volume using a T-connector.

2. Pressurize gauge to slightly above 300 mm Hg.

3. Bleed pressure down no faster than 10 mm Hg per second, stopping to check the pressure at 300, 250, 200, 150, 100 (50 for US) and 0 mm Hg.

4. Determine the difference between the indicated pressure of the non-automated sphygmomanometer being tested and the corresponding reading of the reference manometer.

5. The difference shall be less than or equal to  $+/-3$  mmHg ( $+/-0.4$  kPa).

**NOTE:** Your ability to measure the accuracy of a gauge depends upon the sensitivity of the pressure standard you use for the calibration procedure.

Welch Allyn recommends using a calibrated reference manometer with a maximum error of 0.8 mmHg (0.1 kPa).

###### Cleaning

Wipe the gauge inflation bulb and valve with a bleach-based germicidal wipe. The caution statement in this manual identifies conditions or practices that could result in damage to the equipment or other property.

**CAUTION:** When cleaning the device, avoid cloths or solutions that include ammonium compounds (ammonium chlorides) or glutaraldehyde-based disinfectants.

###### Warranty

Your Welch Allyn product, is warranted to be free from original defects in material and workmanship and to perform in accordance with manufacturer's specifications under normal use and service. The warranty period begins from the date of purchase from Welch Allyn, Inc. or its authorized distributors. Welch Allyn's obligation is limited to the repair or replacement of components determined by Welch Allyn to be defective within the warranty period. These warranties extend to the original purchaser and cannot be assigned or transferred to any third party. This warranty shall not apply to any damage or product failure determined by Welch Allyn to have been caused by misuse, accident (including shipping damage), neglect, improper maintenance, modification, or repair by someone other than Welch Allyn or one of its authorized service representatives.

###### Gauge warranties:

Should the Welch Allyn DuraShock aneroid sphygmomanometer deviate from the  $\pm 3$  mm Hg accuracy specification during the warranty period, Welch Allyn will recalibrate the sphygmomanometer at no charge. Return the gauge only to Welch Allyn Technical Service or your local Welch Allyn authorized representative.

Welch Allyn DuraShock Models DS44, DS45: Five-year calibration warranty

Welch Allyn DuraShock Models DS46, DS66: Ten-year calibration warranty

Welch Allyn DuraShock Model 767: Lifetime calibration warranty

Accessories warranties:

Welch Allyn DuraShock Bumper: One-year warranty

Welch Allyn Inflation Bulb and Valve for Model DS44: One year

Welch Allyn Inflation Bulb and Valve for Model DS45: Two years

These express warranties are in lieu of any and all other warranties, express or implied, including the warranties of merchantability and fitness for a particular purpose, and no other person has been authorized to assume for Welch Allyn any other liability in connection with the sale of the Welch Allyn product. Welch Allyn shall not be liable for any loss or damages, whether direct, incidental, or consequential, resulting from the breach of any express warranty, except as set forth herein.

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