
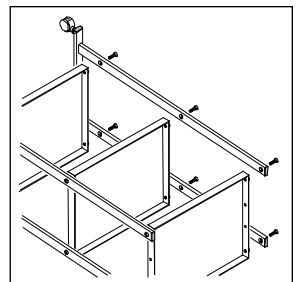
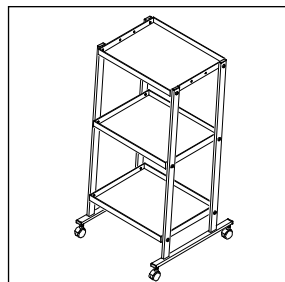
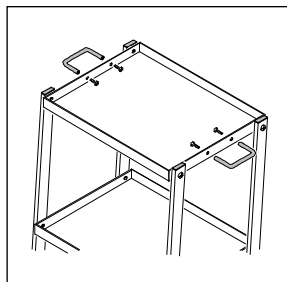
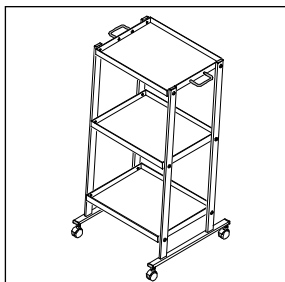
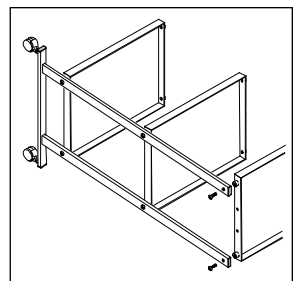
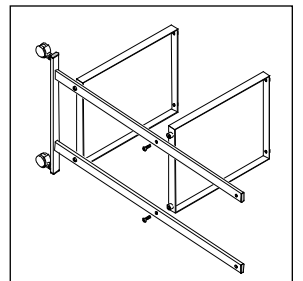
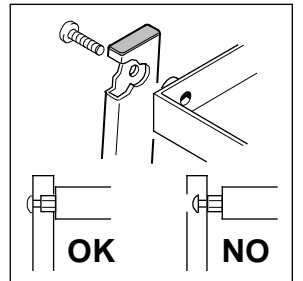
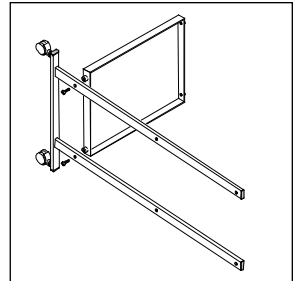
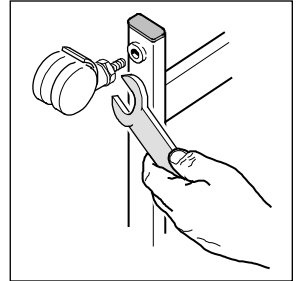
Cod. 27438 **CARRELLO NUOVO TRIS**

Code 27438 **NEW TRIS TROLLEY**

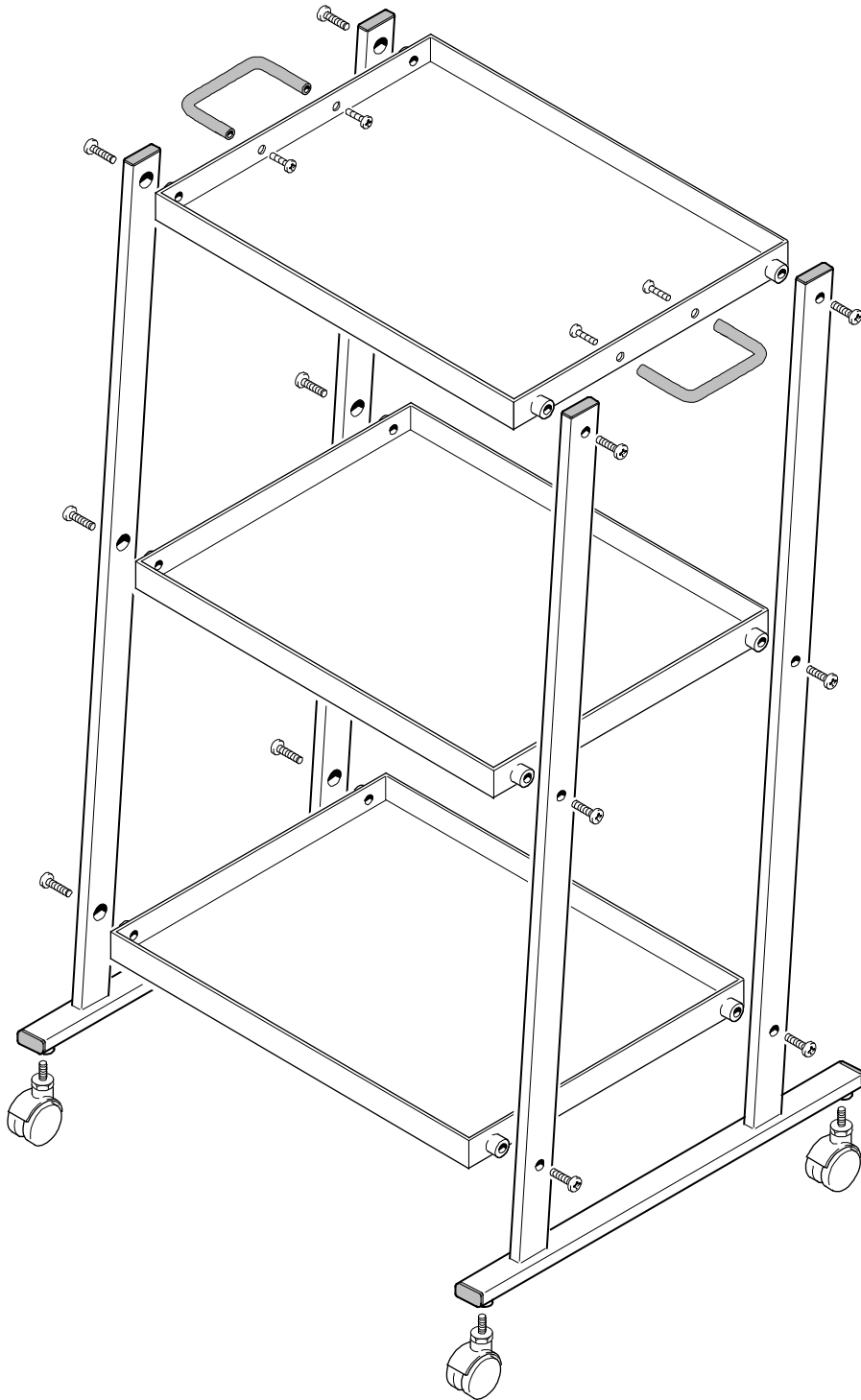## Cod 27438 Carrello Nuovo Tris

### 1. USO PREVISTO

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## Code 27438 New tris trolley

### 1. INTENDED USE

The New Tris is a practical trolley with three shelves made of fire painted sheet metal, equipped with 4 nylon castors and two lateral handles for easy moving, suitable to support and transport small electromedical instruments. In addition to the following standards, the user must follow the prescriptions of the law as far as occupational safety and health concerns. Always abide by safety standards and the instructions here contained. The manufacturer declines any liability for damage due to incorrect use of the equipment. Transport and unloading operations must be carried out by qualified and authorized personnel only. Other personnel must be kept away from the carriage. The trolley must be used, maintained and repaired by qualified operators only; when the trolley is not used, keep it far from people who are not directly involved in work. The trolley must be used on flat surfaces. Pay attention that no obstacle prevents one or more castors from rolling smoothly.

### 2. USE RESTRICTIONS AND SPECIAL WARNINGS

Read the operating instructions

- The couch must **NOT** be used in the following conditions:
  - Fire risk, explosive or corrosive atmosphere.
  - Exposed to weathering.
  - Tampering attempts and/or modifications to the original product.
- Pay attention not to introduce body parts into moving and/or rotating components.
- The trolley must be used under good stability conditions. The support feet must rest on a flat surface.

Clean with mild disinfectant. Corrosive and/or abrasive detergents can damage painting, as well as dragging pointed metal products such as screws and so on.

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