GIMA

PAPETTE

PAPETTE U.S. PATENT #4,700,713

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ

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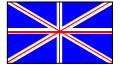
Papette™ (non-sterile)

*Read all safety information and instructions listed below before using this product!
Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician.*

Application: The Papette is the most gentle, yet effective means of simultaneously taking both ectocervical and endocervical cell samples.

For Cervical Cytology utilizing the Conventional Pap Smear

Instructions:

1. Contact the cervix with the Papette and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix.
2. Push gently, and rotate the Papette in either direction.
3. Remove the Papette and draw each side once across the slide (like a paint brush), thus transferring both ectocervical and endocervical cell samples in one stroke.
4. Dispose of Papette in accordance with local Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the Papette is being used to obtain samples.

For Cervical Cytology utilizing the ThinPrep® Pap Test™

Instructions:

1. Contact the cervix with the Papette and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix.
2. While maintaining gentle pressure in the direction of the cervix, rotate the brush 5 times in a clockwise direction.
3. Rinse the brush into the PreservCyt® solution for use with the ThinPrep® Pap Test™ by pushing the brush into the bottom of the vial 10 times, forcing the bristles apart. Swirl the brush vigorously to further release material.

Discard the collection device.

4. Tighten the PreservCyt® vial cap so that the torque line on the cap passes the torque line of the vial.
5. Dispose of Papette in accordance with local Medical Hazardous Waste practices.

Trademarks: ThinPrep® and PreservCyt® are registered trademarks of Cytye Corp.

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Marcas: ThinPrep® e PreservCyt® são marcas registradas da Cytec Corp.
Leisegang Feinmechanik GmbH, Leibnizstrasse 32, D-10625, Berlin, Germany