GONIOMETRO PER LE DITA 6" STD

FINGER GONIOMETER 6" STD

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GONIÓMETRO DE DEDOS 6" STD

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ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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REF 24900

Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) - Italy
Made in Pakistan

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GB - INSTRUCTIONS OF USE

A finger goniometer measures range-of-motion (ROM) of finger joints (metacarpophalangeal and interphalangeal) and other small joints including toes. Standard 6" stainless steel finger goniometer has head with opposing 180° to 30° scales in 5° increments. Linear scale on arm measures in inches and centimeters.

INDICATION OF USE & PRECAUTIONS

Those devices are designed for professional use by qualified operators with necessary skills. The use of the product involves direct contact with patient's skin; always wear protective gloves, do not use on dull skin, handle carefully to avoid accidental abrasion. Clean and sterilize before each use; use neutral non-aggressive disinfectant solution (PH7), do not use solutions containing chloride ions or high alcohol content. This product is intended to provide easy-to-read indicative measurements. Class I Medical Device.

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