
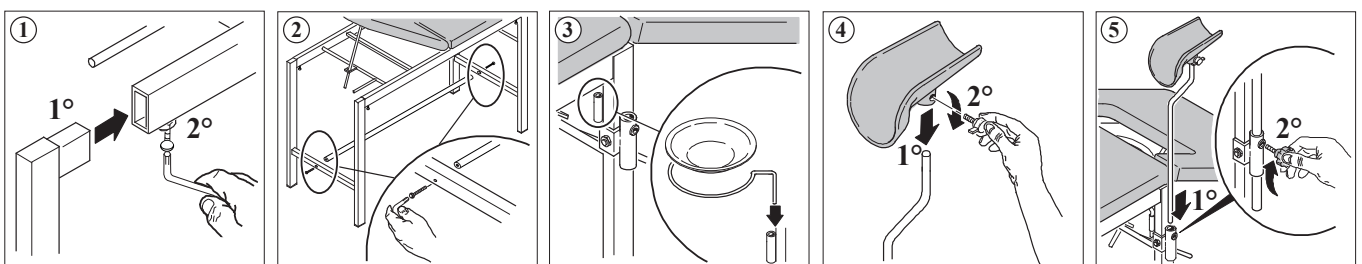
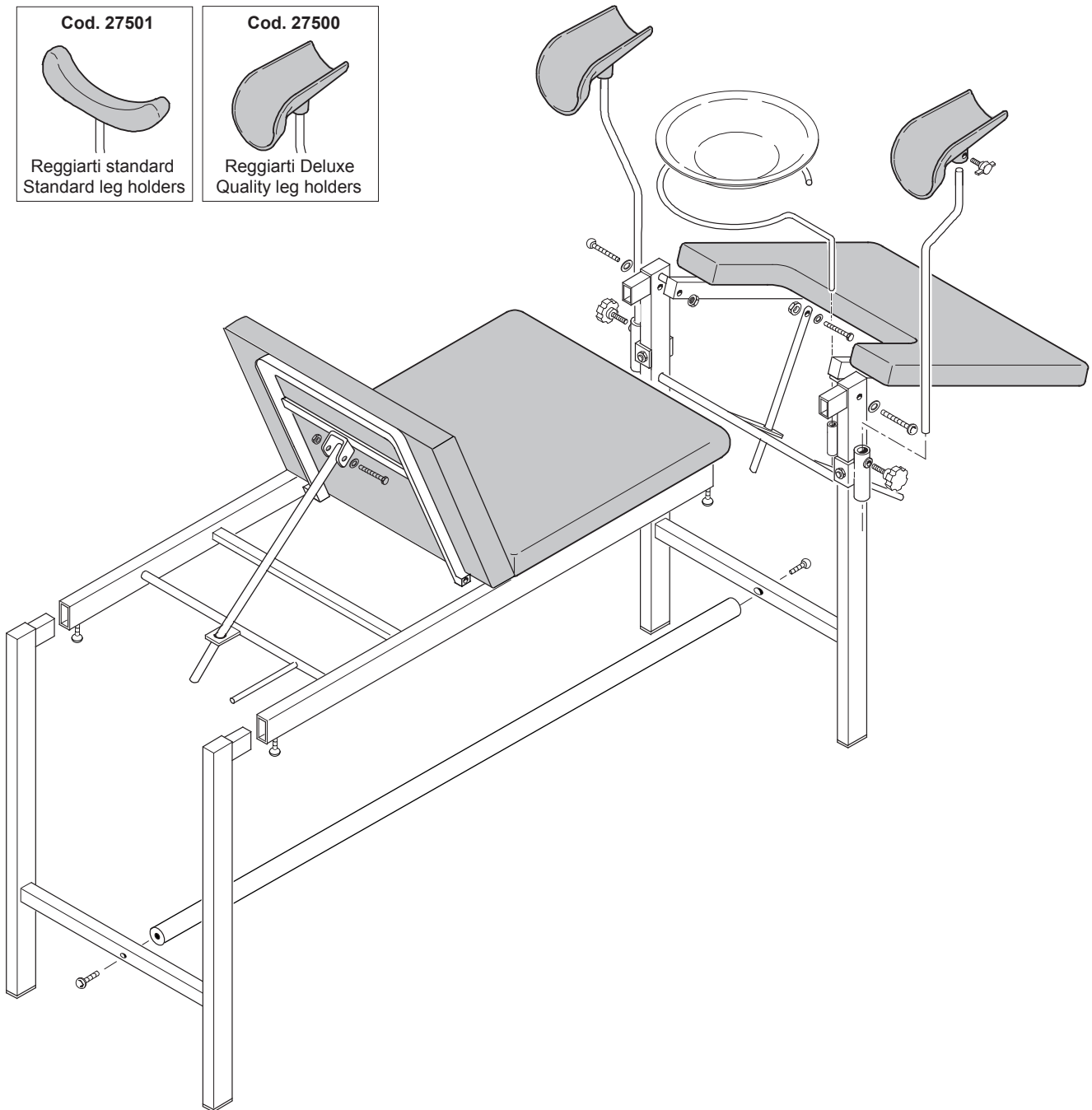
Cod. 27501 **LETTINO GINECOLOGIA STANDARD** - Cod. 27500 **LETTINO GINECOLOGIA DELUXE**
 Cod. 27501 **STANDARD GYNAECOLOGY BED** - Cod. 27500 **DELUXE GYNAECOLOGY BED**

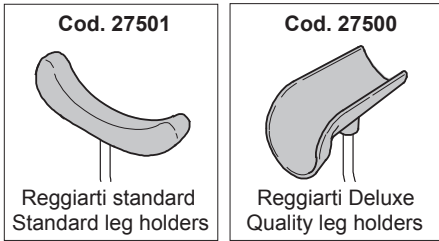Cod 27500/1 - Lettino ginecologia

1. USO PREVISTO

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Code 27500/1 – Gynecology bed

1. INTENDED USE

Examination bed made using a sturdy chromed rectangular steel tube structure. It includes a mattress in washable, self-extinguishing, black material. Back rest and foot rest can be adjusted in height. It is supplied with a basin Ø 30 cm and limb holder.

Always follow the safety regulations and the instructions in this manual.

The manufacturer cannot be held responsible for an incorrect use of the equipment supplied. Transportation and unloading operations must be carried out by specialized and authorized personnel. Do not let unauthorized people near equipment. It is supplied disassembled.

2. LIMITATIONS OF USE AND PARTICULAR WARNINGS

Read operating instructions

- The bed **MUST NOT** be:
 - Used in an environment with risk of fire, explosive or corrosive area.
 - Exposed to severe weather conditions.
 - Tampered with and/or no modifications must be made to original product.
- Do not put parts of the body in areas subject to movement and/or adjustments.
- The bed must be used in conditions of stability and with supporting feet placed on flat surface.
- Keep bed still while moving patient on and off of bed because the bed could fall over.
- There is risk of lateral tilting even with a minor lateral load.
- There is risk of crushing while bending the back piece forwards and backwards, and during the installation of the leg staffs and other accessories.
- Do not allow patient to climb on bed from back side.
- The back piece is fitted with an unstable mechanical block (in all positions), particularly when it is released from its load to change position.

The lamps used for therapeutic irradiation can damage the upholstery imitation leather. Corrosive and/or abrasive detergents can damage the upholstery imitation leather.

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