


**Follow these Instructions step-by-step:**

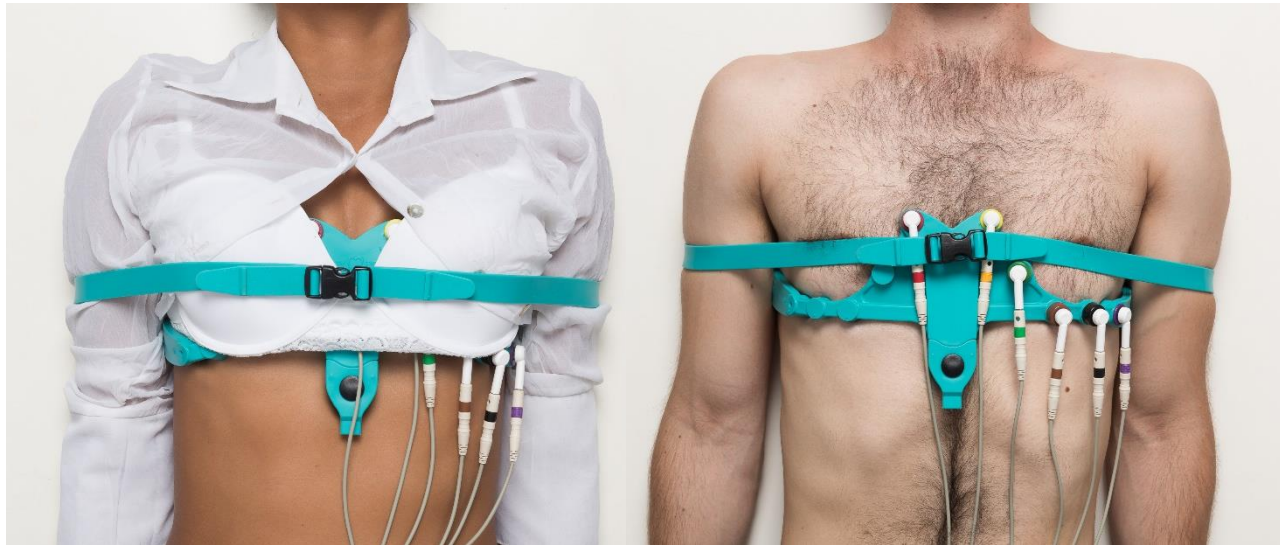

**LevMed AllBrand™ ECG Belt with Straps**

**Ensure the ECG Patient Cable is  
connected to the ECG Device.**

**Ensure the screws on the ECG Belt are tightly closed:**



## Connect the ECG Patient Cable to the screws using Snap/Crocodile connectors:

*Banana to snap adaptor*



*Banana to crocodile adaptor*



\*If your ECG Patient Cable has Banana connectors, use the banana-to-snap or the banana-to-crocodile adaptors.


## Connect the ECG Patient Cable to each of the 4 clamps:

### For Banana Patient Cable:

1. Open the screw.
2. Insert the banana connector
3. Close the screw.




### For Snap/Crocodile Patient Cable:

1. Close the screw.
2. Connect the snap/crocodile.

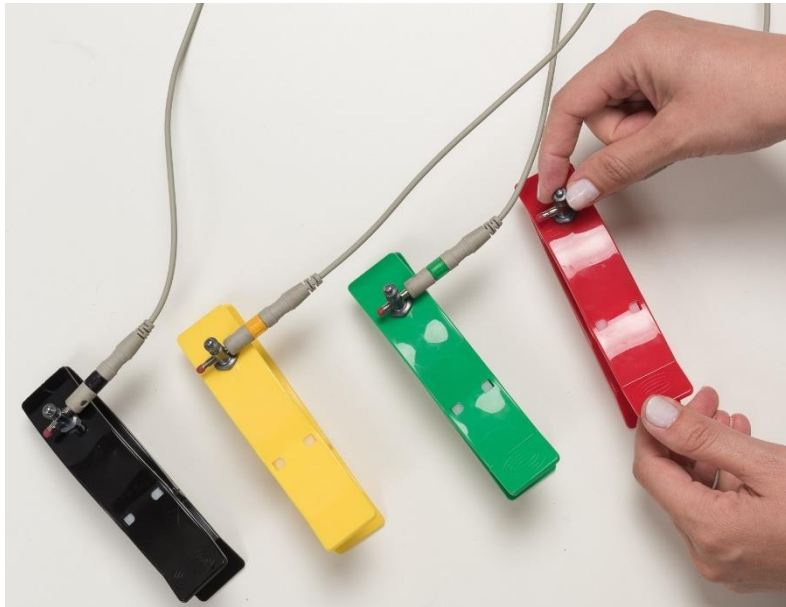


## Open or Remove the patient's shirt:

(Note: ECG Belt buckles will be closed on the FRONT of the chest)

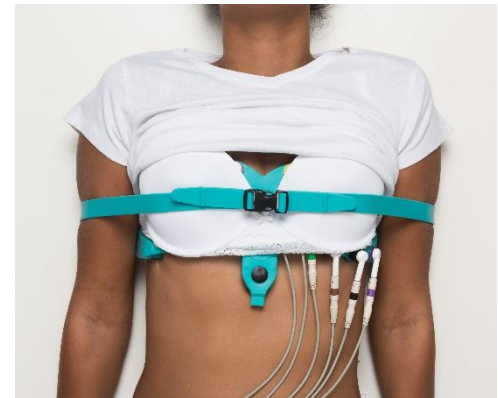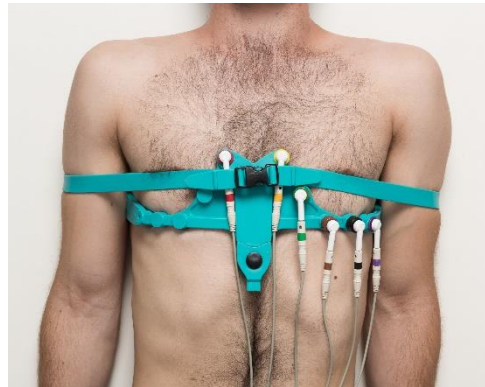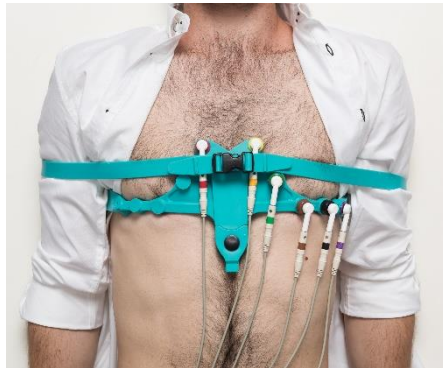



**Wet the 6 Electrodes (V1-V6) with Tap water:**

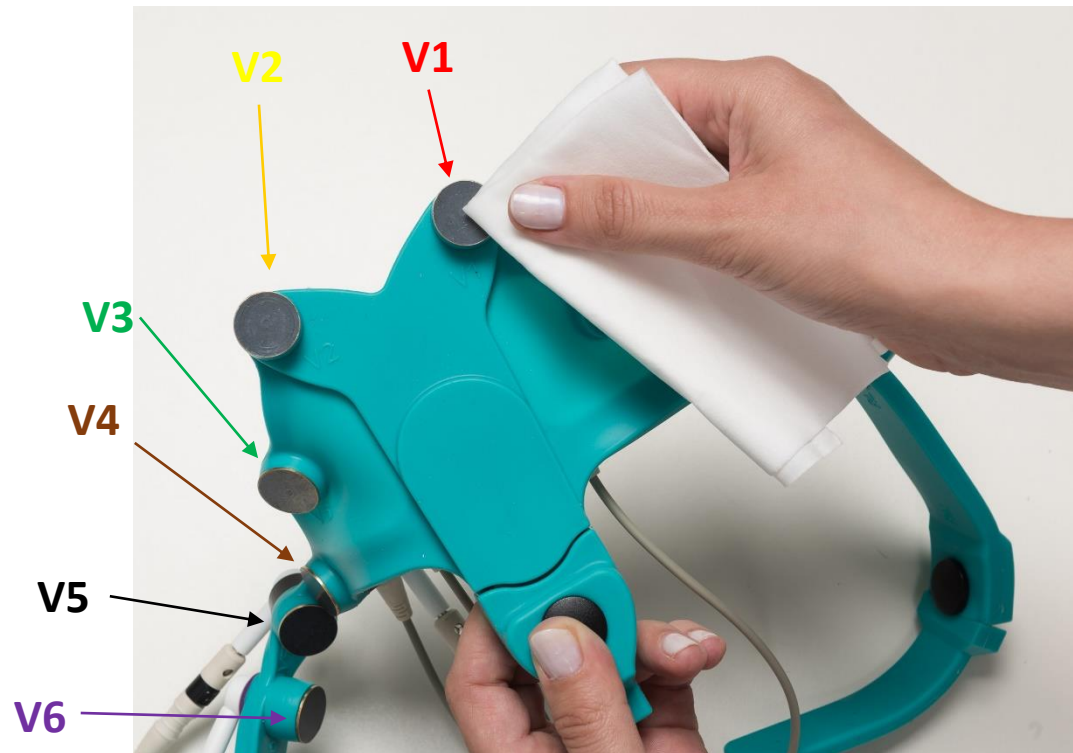


**If necessary, bend V1 & V2 to ensure electrode contact with patient chest:**






**When the patient is lying down comfortably, apply the belt on patient chest with V1 and V2 symmetrically on both sides of the sternum:**

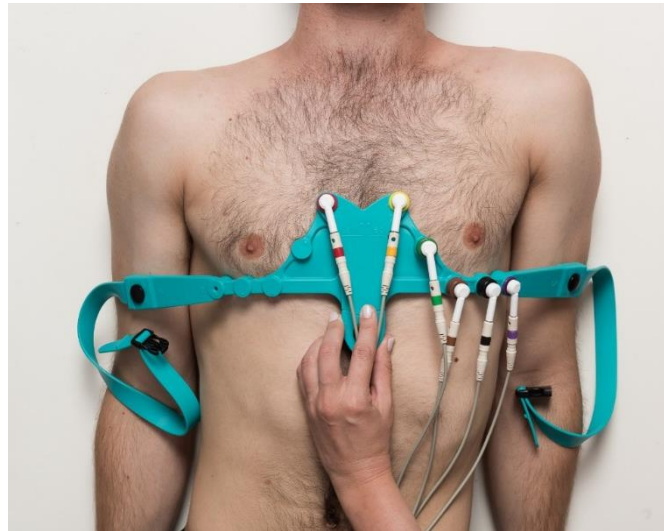


**Note - for women, the belt may be applied under the bra:**




**Apply one strap below the mid-arm,  
between elbow and shoulder,  
and then above the arm:**

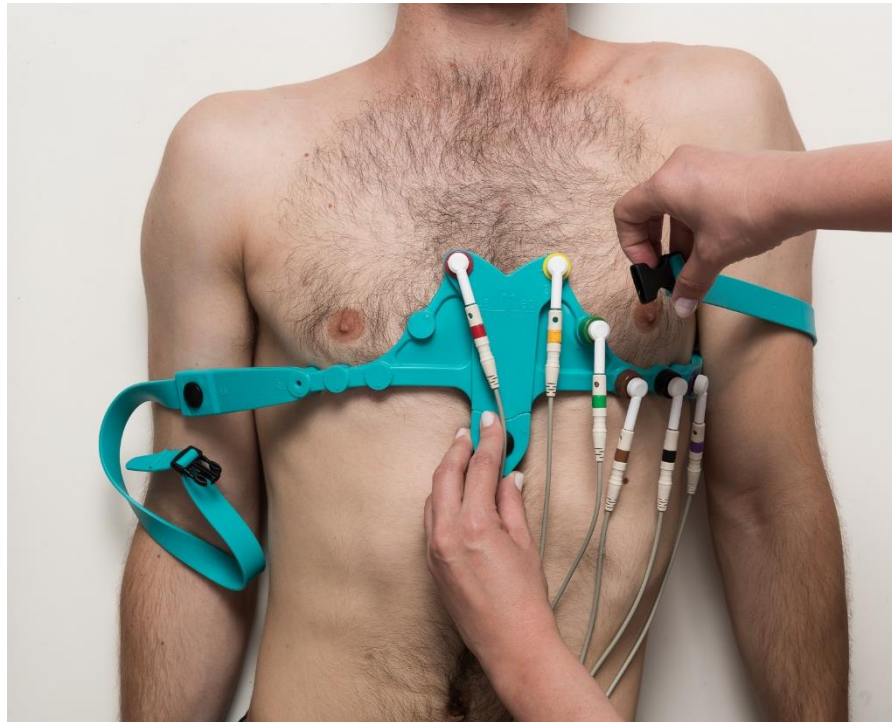


**Apply the second strap below and then  
above the second arm:**

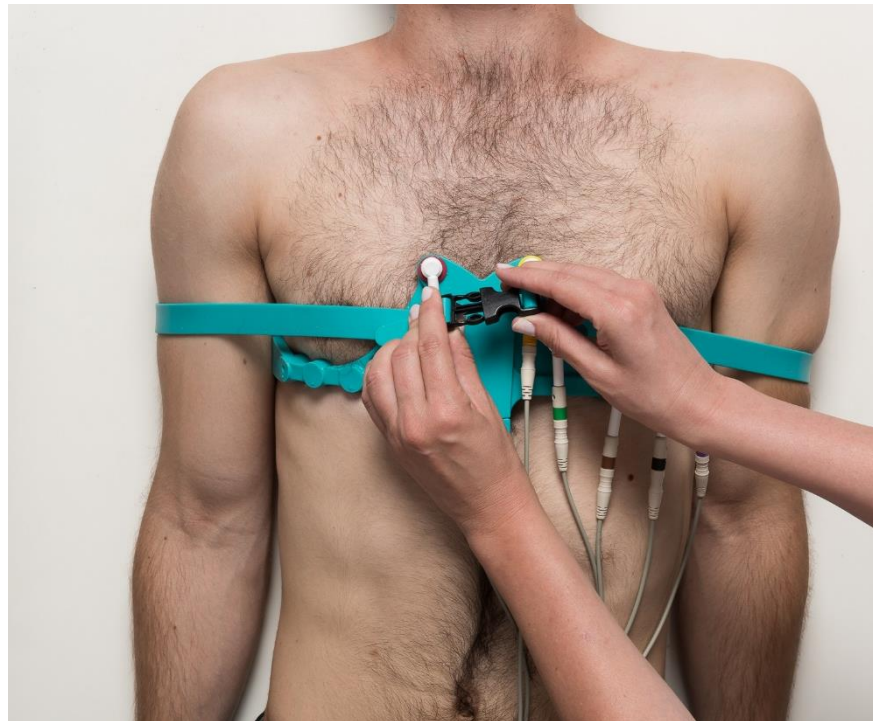

**Close the buckles over V1 and V2:**

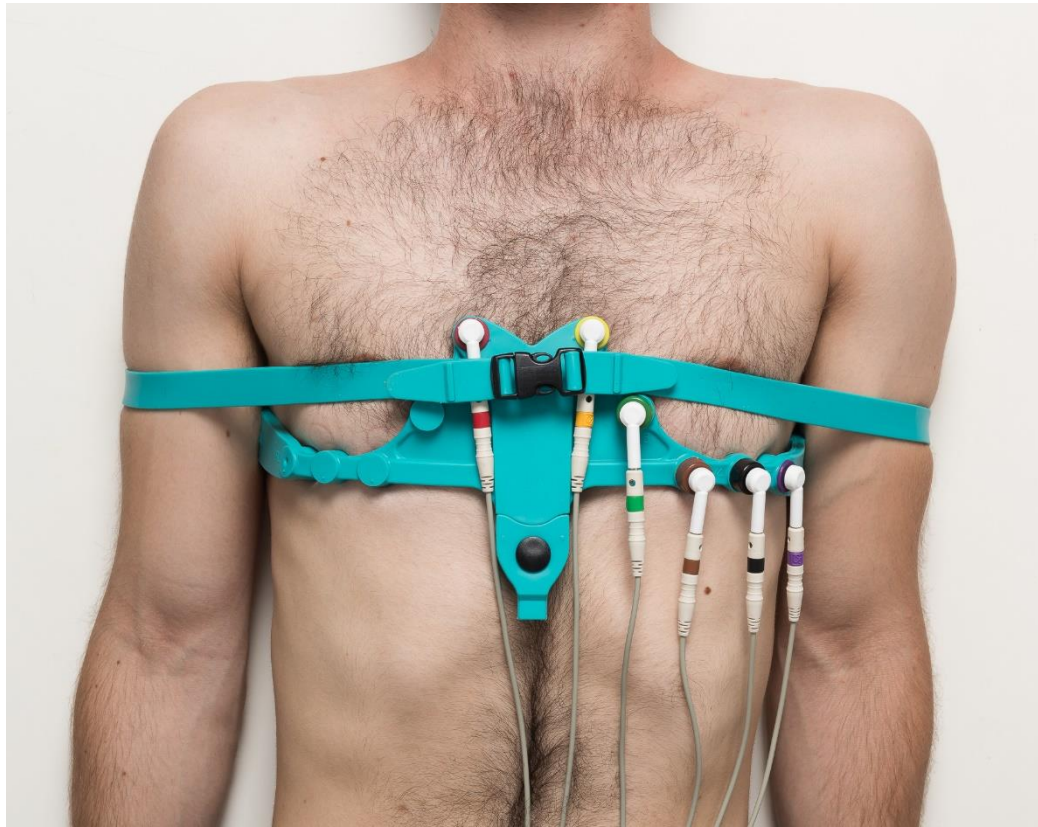


**If needed, adjust the buckles location on the straps to fit patient chest size:**

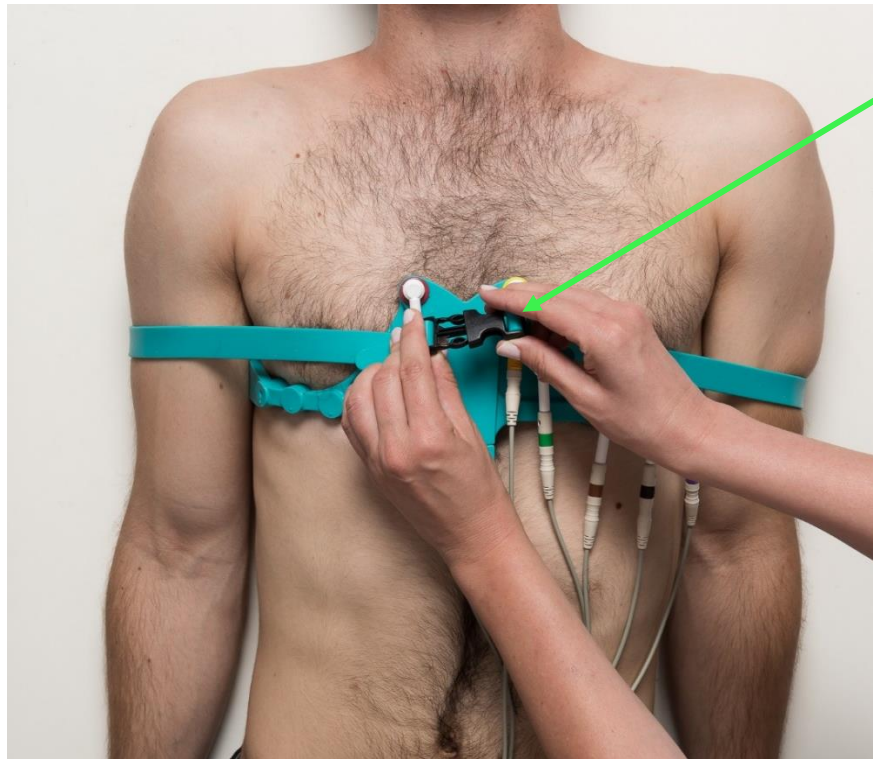




**Confirm electrodes V4, V5, and V6  
are located beneath the breast:**

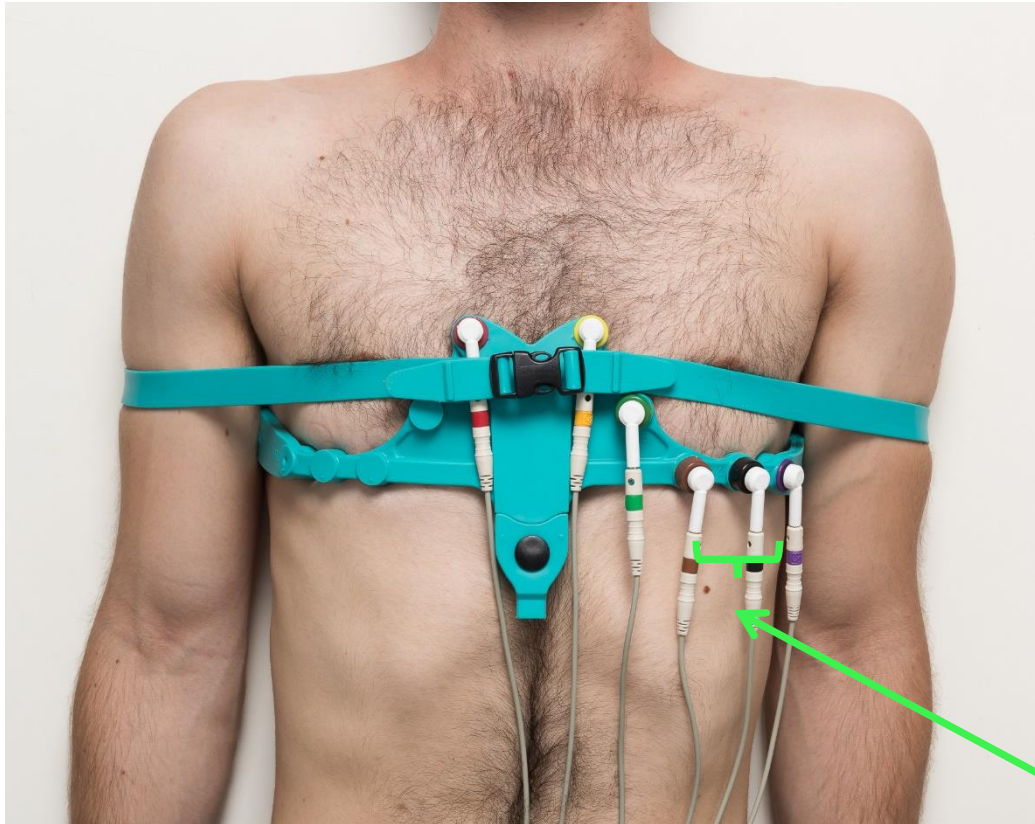


V4, V5, V6



**Ensure all 6 electrodes are in contact  
with patient skin:**

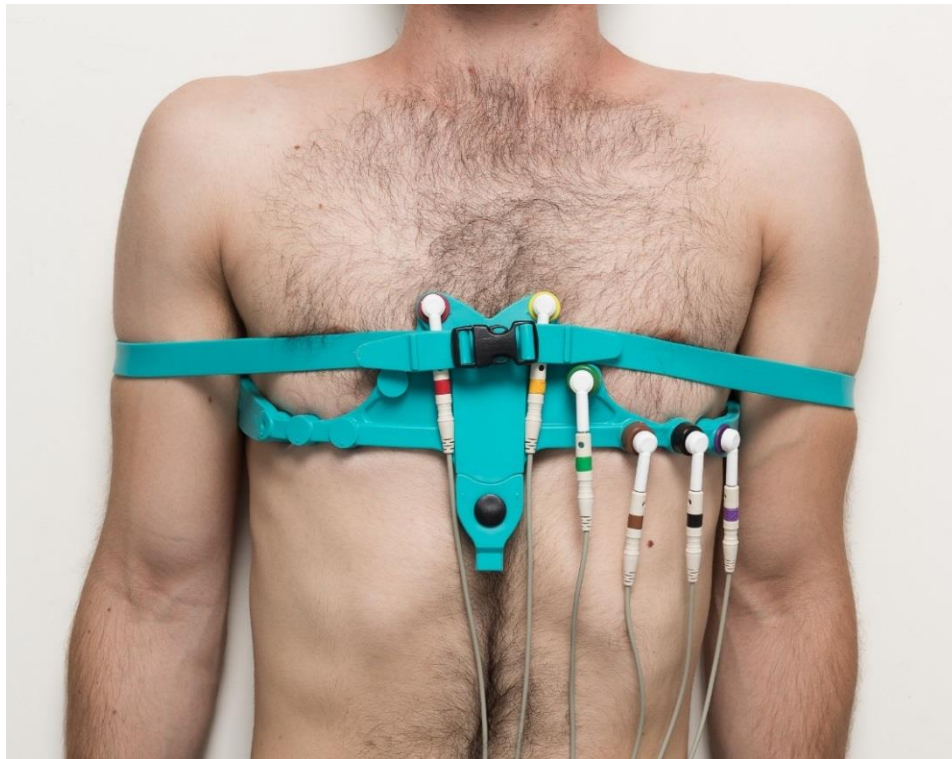

**Wet the Electrode on each of the 4 Clamps  
with Tap water:**

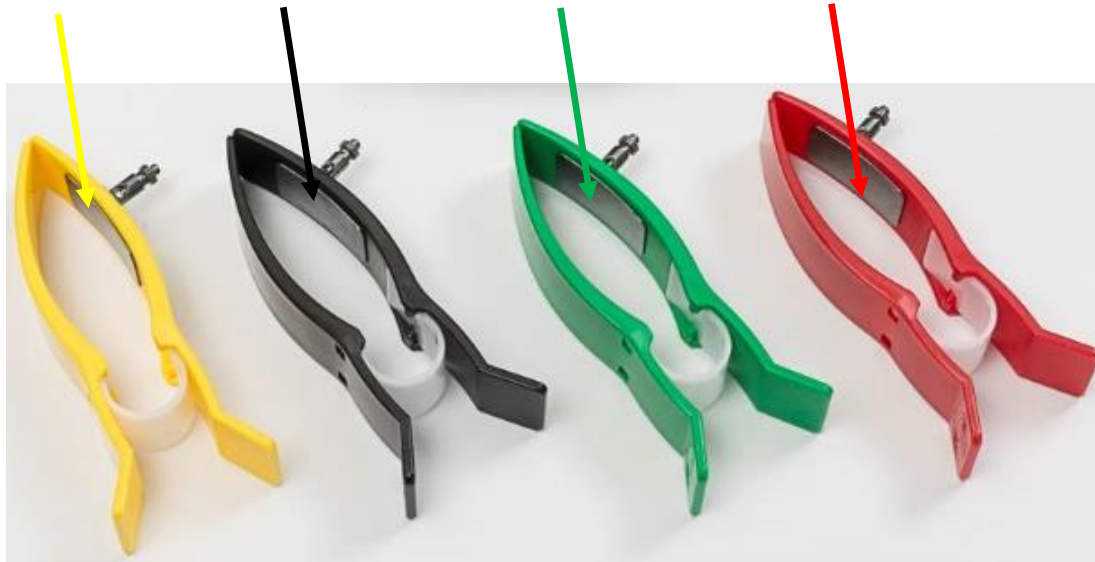

Apply the clamps on the limbs:

**Red = Right Arm**

**Yellow = Left Arm**

**Green = Left Leg**

**Black = Right Leg**




**Ensure the patient is relaxed and does not strain his muscles.**

**Ask the patient not to move or talk during the assessment.**





**Note: in case more comfortable, the assessment may be done while the patient is in a sitting position:**

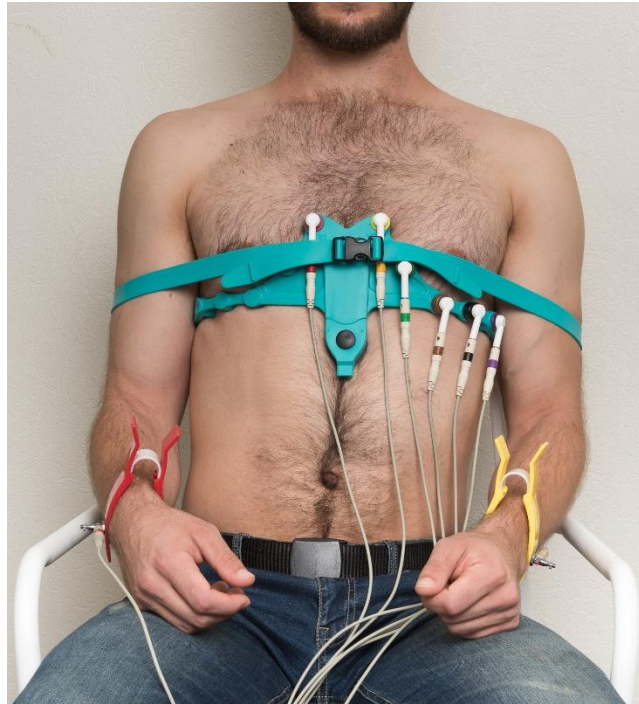



Press **'Start'** on the ECG Device to take ECG Reading.

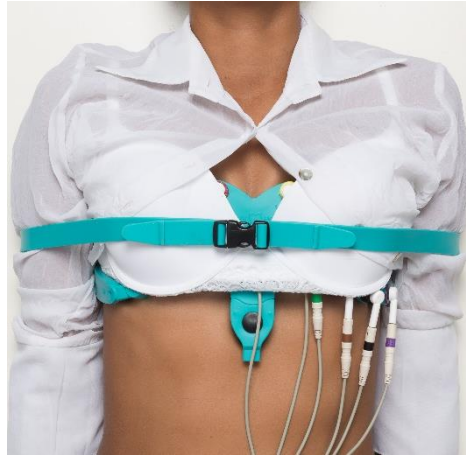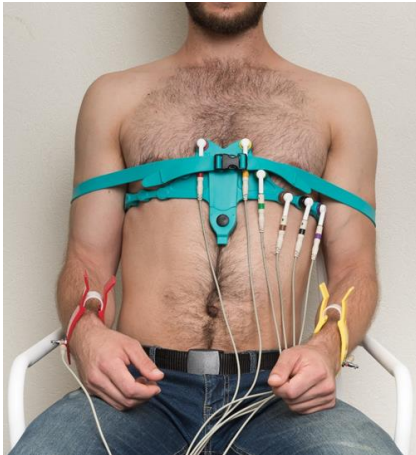



## Troubleshooting

**In case noise is seen on the ECG signal, ensure:**

1. Water is applied on all electrodes.
2. All electrodes are in direct contact with patient skin.
3. Patient is relaxed, muscles not strained.





**Thank you for using LevMed  
AllBrand™ ECG Belt!**

For more information contact us at [info@levmed.net](mailto:info@levmed.net)  
[www.levmed.net](http://www.levmed.net)