ITALIANO

DESTINAZIONE D’USO

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ENGLISH

INTENDED USE

The device is used to visualize a patient’s internal area (Throat, Larynx).

THE DEVICE MUST BE USED ONLY BY QUALIFIED MEDICAL PERSONNEL

CLEANING

Always clean it with proper care. The Product can be cleaned by using the following method:

- 1. Slightly moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wipe the surface of the product.
- 2. Wipe from side to side rather than in a circular motion.
- 3. It should be rinsed in boiling distilled water, gently scrub to remove any residue.

Note: If chemical cleaners are used, see manufacturer’s instructions regarding use.

WARNING

Don’t use the equipment in case it is damaged, apply to your retailer.

DISPOSAL

The product contains no hazardous materials. The product should be discarded accordance with the national or local regulations.

RECOMMENDED OPERATING ENVIRONMENT

Use

Temperature 10°C to 40°C
Humidity 30% to 75%
Atmospheric pressure 700hPa to 1060hPa

Storage and Transport

Temperature from -20°C to 60°C
Humidity 10% to 90% (without condensation)
Air Pressure 500hPa to 1060hPa

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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Gima S.p.A.