GIMA

PROFESSIONAL MEDICAL PRODUCTS

WALLACH® PAPETTE (NON STERILE)
WALLACH® PAPETTE (NON STERILE)
WALLACH® PAPETTE (NON STÉRILE)
WALLACH® PAPETTE (NO ESTÉRIL)
WALLACH® PAPETTE (NÃO ESTÉRIL)
WALLACH® PAPETTE (ΜΗ ΑΠΟΣΤΕΙΡΩΜΕΝΟ)

REF

908004 (29738)

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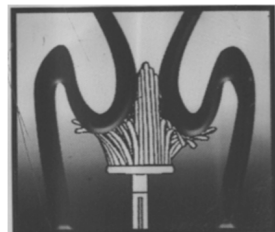
Read all safety information and instructions listed below before using this product.

R_xOnly CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Application:

The WALLACH® PAPETTE, a product of CooperSurgical, is the most gentle, yet effective means of simultaneously taking both ectocervical and endocervical cell samples.

Caution: Be sure not to force the shorter bristles into the cervical os to prevent unintended separation of the brush from the handle.

For Cervical Cytology utilizing the Conventional Pap Smear

Instructions:

1. Contact the cervix with the WALLACH® PAPETTE and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. Be sure not to force the shorter bristles into the cervical os to prevent unintended separation of the brush from the handle.
2. While maintaining gentle pressure in the direction of the cervix, rotate the brush two or more times in either direction.
3. Remove the WALLACH® PAPETTE and draw each side once across the slide (like a paint brush), thus transferring both ectocervical and endocervical cell samples in one stroke.
4. Dispose of WALLACH® PAPETTE in accordance with local Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the WALLACH® PAPETTE is being used to obtain samples.

For Cervical Cytology utilizing the ThinPrep® Pap Test

Instructions:

1. Contact the cervix with the WALLACH® PAPETTE and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. Be sure not to force the shorter bristles into the cervical os to prevent unintended separation of the brush from the handle.
2. While maintaining gentle pressure in the direction of the cervix, rotate the brush two or more times in either direction.
3. Rinse the brush into the PreservCyt® solution for use with the ThinPrep® Pap Test by pushing the brush into the bottom of the vial 10 times, forcing the bristles apart.
Swirl the brush vigorously to further release material. Discard the collection device.
4. Tighten the PreservCyt® vial cap so that the torque line on the cap passes the torque line on the vial.
5. Dispose of WALLACH® PAPETTE in accordance with Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the WALLACH® PAPETTE is being used to obtain samples.

PreservCyt® and ThinPrep® are registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and other countries.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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