MASCHERINA CHIRURGICA LAVABILE DI TIPO II DOPPIO STRATO AD USO MEDICALE
TYPE II WASHABLE DOUBLE LAYERED SURGICAL MASK FOR MEDICAL USE
MASQUE CHIRURGICAL LAVABLE À DOUBLE COUCHE DE TYPE II À USAGE MÉDICAL

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ENGLISH

1 - Product description and intended use
Washable mask, in 99% polyester and 1% carbon fibre fabric, designed to protect the nose and mouth.

2 - Classification
The BP2020V3TN series masks are classified as Type II, according to the UNI EN 14683:2019 standard.

3 - Features
The device consists of a 99% polyester and 1% carbon fibre fabric layer and of a water-repellent outer layer; it is machine washable up to 60°C (the fabric withstands up to 150°C) It is not sterile.

PFE values: Particle Filtration efficiency (%) 0.3 µm 0.5 µm at 95%.
The fabric is **NON-CYTOTOXIC** according to tests carried out on fibroblasts in compliance with the UNI EN ISO 10993-5 standard.

BFE Filtration
BFE = 99.8% (from laboratory tests carried out in compliance with the UNI EN 14683:2019 standard).

Biological load
UFC <30 UFC (from laboratory tests carried out in compliance with the UNI EN 14683:2019 standard).

Air permeability (cm2/cm3/sec) ASTM E96 (Also known as ISO - 11092 or Ret or Hohenstein test).
The result obtained on the BP2020 series masks is 7.5, therefore good or very breathable.

4 - Instructions for use and warnings
Handle with clean and sanitised hands, making sure the mask adheres well to the face, in the right position, i.e. with the smallest part towards the nose, and the larger one over the mouth and chin.
DO NOT MODIFY THE MASK; DO NOT CUT THE ELASTIC BANDS; WEAR IT RIGHT WAY OUT (as shown in the figure).
Otherwise the mask loses its protective function.

5 - Washing
By hand or in the washing machine, with a detergent, up to 60°C. **DO NOT BLEACH.**
By complying with the instructions for use and maintenance, the product maintains the same physical and technical features up to approx. 100 washes.

6 - Disposal
Mask to be disposed of as SUW (unless it has been contaminated, in which case it must be disposed of as "special waste"). The outer bag is disposed of as plastic.

REF	Product code	LOT	Lot number
	Manufacturer		Date of manufacture
	Non-sterile		Consult instructions for use
CE	Medical Device complies with Directive 93/42/EEC		

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