**MASCHERINA CHIRURGICA LAVABILE
DI TIPO II AD USO MEDICALE**
**TYPE II WASHABLE SURGICAL MASK
FOR MEDICAL USE**
**MASQUE CHIRURGICAL LAVABLE
DE TYPE II À USAGE MÉDICAL**
**TYP II WASCHBARE CHIRURGISCHE MASKE
FÜR MEDIZINISCHE ANWENDUNGEN**
**MASCARILLA FACIAL QUIRÚRGICA
DE TIPO II LAVABLE PARA USO MÉDICO**
**MÁSCARA CIRÚRGICA LAVÁVEL TIPO II
PARA USO MÉDICO**
**TYP II TVÄTTBART KIRURGMUNSKYDD MED
FÖR MEDICINSK ANVÄNDNING**
**ΠΛΕΝΟΜΕΝΗ ΧΕΙΡΟΥΡΓΙΚΗ ΜΑΣΚΑ
ΤΥΠΟΥ II ΓΙΑ ΙΑΤΡΙΚΗ ΧΡΗΣΗ**
**MASKA CHIRURGICZNA MEDYCZNA
MOŻLIWA DO PRANIA TYPU II**
**MÁSCA CHIRURGICALĂ DE UZ MEDICAL,
DE TIP II, LAVABILĂ**

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REF 20776 / BP2020V3 -KIDS

MYCROCLEAN ITALIA SRL
Via Carlo Porta, 14
20064 Gorgonzola - Italy
Made in Italy

LOT Vedere etichetta / See label / Voir l'étiquette
Siehe Etikett / Ver la etiqueta / Ver rótulo
Se etikett / Δείτε ετικέτα / Patrz etykieta
Consultati eticheta / انظر الملصق

Vedere etichetta / See label / Voir l'étiquette
Siehe Etikett / Ver la etiqueta / Ver rótulo
Se etikett / Δείτε ετικέτα / Patrz etykieta
Consultati eticheta / انظر الملصق

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ENGLISH

- Product description and intended use**
Washable mask, in 99% polyester and 1% carbon fibre fabric, designed to protect the nose and mouth
- Classification**
The BP2020V3 series masks are classified as Type II, according to the UNI EN 14683:2019 standard.
- Features**
The device consists of one layer and is machine washable up to 60°C (the fabric withstands up to 150°C) It is not sterile
PFE values: Particle Filtration efficiency (%) 0.3 pm 0.5 pm at 95%
The fabric is **NON-CYTOTOXIC** according to tests carried out on fibroblasts in compliance with the UNI EN ISO 10993-5 standard.
BFE Filtration
BFE = 99.8% (from laboratory tests carried out in compliance with the UNI EN 14683:2019 standard)
Biological load
UFC <30 UFC (from laboratory tests carried out in compliance with the UNI EN 14683:2019 standard) **Air permeability (cm2/cm3/sec) ASTM E96** (Also known as ISO - 11092 or Ret or Hohenstein test)

The result obtained on the BP2020 series masks is 7.5, therefore good or very breathable

- Instructions for use and warnings**
Handle with clean and sanitised hands, making sure the mask adheres well to the face, in the right position, i.e. with the smallest part towards the nose, and the larger one over the mouth and chin.
DO NOT MODIFY THE MASK; DO NOT CUT THE ELASTIC BANDS; WEAR IT RIGHT WAY OUT (as shown in the figure) **Otherwise the mask loses its protective function.**
- Washing**
By hand or in the washing machine, with a detergent, up to 60°C. **DO NOT BLEACH**
By complying with the instructions for use and maintenance, the product maintains the same physical and technical features up to approx. 500 washes.
- Disposal**
Mask to be disposed of as SUW (unless it has been contaminated, in which case it must be disposed of as "special waste"). The outer bag is disposed of as plastic.

REF	Product code	LOT	Lot number
	Manufacturer		Date of manufacture
	Non-sterile		Consult instructions for use
	Medical Device complies with Directive 93/42/EEC		

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