PROFESSIONAL MEDICAL PRODUCTS

TELO PORTAFERITI

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CARRYING SHEET

8 lifting handles version

CIVIÈRE DE TRANSPORT

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CAMILLA DE TRANSPORTE

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MACA DE LONA PARA TRANSPORTAR FERIDOS

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ENGLISH

INDICATIONS: the flexible carrying sheet moves patient securely in restricted areas where standard stretchers have difficulties to be used. **It should not be used if there is a spine, backbone or pelvis suspected injury.**

SPECIFICATIONS: strong polyester fabric, PVC coated, with four secured handles per side to enable a comfortable patient carriage. The materials are waterproof and are easily washable and disinfettable, bacterial and fungus proof, non putrescent. Transparent to X rays.

INSTRUCTIONS: should be used by skilled staff. After laying the patient on the carrying sheet, four rescuers (as per fig. 1) should pull up contemporary on the lifting handles and bring the patient to destination. However if four are not available, two rescuers (as per fig. 2).

CLEANING INSTRUCTIONS: after using the carrying sheet, it should be accurately washed with water and soap. It can also be disinfected by a bacterial liquid solution.

STORAGE: no special instructions are requested. Better if kept in a dry place, out of dust and direct sunlight.

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