
The cuff can be wrapped to any limb of the animal, and the base of the tail can also be wrapped if necessary. Please take the measurement in a room with an appropriate environment (see Chapter 18).

Carry out the operation in a room with comfortable temperature.

Wear the cuff accurately following these steps:


- ① Make sure the air plug is securely inserted in the main unit.

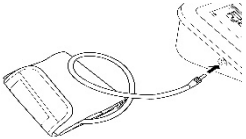- ② Wrap the cuff to the animal's lg, and ensure that the cuff is at the same level as its heart. To ensure the same level and the measurement accuracy, the wrapping method of the cuff can be flexibly adjusted according to the animal's p'sture. The specific methods are as follows:

When the animal is in a supine position, the cuff should be wrapped to the sturdy part of its upper limb, and the position of the cuff can be adjusted to ensure that the cuff is almost at the same level as its heart in the vertical direction.

When the animal is in a squatting position, the cuff should be wrapped to the sturdy part of its forelimb, then lift the forelimb until the height of the cuff is almost the same as its heart. During this process, keep the animal comfortable and prevent it from motion due to nervousness or fear, resulting in measurement failure.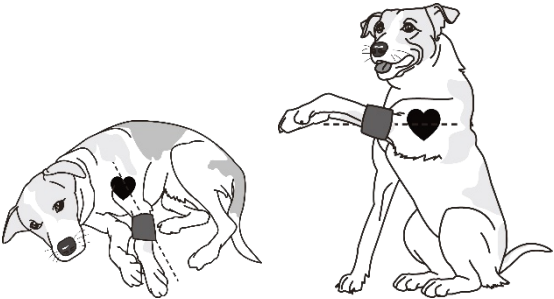

Note

- Confirm that the cuff is not inflated before wrapping it.
- Select an appropriate cuff for the animal based on the circumference of its limb.
- Do not wrap the cuff too tightly around the limb, otherwise it may cause discoloration or even ischemia of the limb.

9.3 BP Measurement

In order to measure accurately, pay attention to applying the cuff properly. Select a proper cuff according to different animals, then select proper cuff type(large, middle and small).

There are two ways to select the cuff mode, in the time interface, press CUFF button to switch cuff mode; or enter the 【SYSTEM SETUP】 submenu in 【SYSTEM MENU】 , and complete switching in 【CUFF TYPE】 item.

Every CUFF TYPE corresponds to different cuff sizes, which are identified on the cuff. As follows:

Cuff Type	Limb perimeter
Small	<13 cm
Middle	8 ~ 26 cm
Big	>25 cm

Note

- ① Press 【START/STOP】 button to start measurement.

During measurement, please keep correct pose and quiet state, do not move.

If you want to abort the measurement,Press 【START/STOP】 button, the device will stop inflating, and release the air from the cuff.

- ② Confirm the Measurement Value

The measurement value can be stored automatically, using [memory function] (refer to Chapter11).

*Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician.

Note

Wait at least 4-5 minutes between measurements

When repeatedly measuring, the accurate blood pressure value may not be measured due to congestion in the limbs. Please measure after the blood flow is smooth.

- When some factors affect the measurement results in measurement process, error messages will appear on the screen, you can obviate the malfunction and restart a measurement.
- The minimum value of the patient's physiological signal is the minimum limit that the device can measure. The device may obtain inaccurate measurement results when it is operated below the minimum amplitude or minimum value of the patient's physiological signal.

- ③In the state of the physiological parameter over-limit prompt is not triggered, press any button to carry on the corresponding button function; in audio prompt state, press any button (except 【ON/OFF】 button) to clear up the audio prompt.
- ④Take off the cuff, press 【ON/OFF】 button to turn the device off.

Chapte r10 Memory Function

The sphygmomanometer is designed to store the blood pressure, pulse rate values and the date and time when measured, which are up to 100 groups. If there have been stored 100 groups, the earliest results will be deleted when saving the 101 group of measurement results.And the "OVERFLOW"will display on the bottom right of the screen,prompting that data record is full. You can delete the data.

10.1 Review Memory Values

1. In the main interface (interface when starting-up), press 【MEMORY】 button to review the latest measured values in big-font with the serial number from 1 to 100.

2. Press 【UP】 / 【DOWN】 button to circularly switch the former measurement values.

*The right figure shows that there is no measurement result can be displayed.

3. Press 【LIST】 button to display the data list interface.

4. Press 【TREND】 button to display trend interface.

End to display the measurement values:

Press 【EXIT】 button to return to the main interface or hold 【ON/OFF】 button to turn the device off.

10.2 Delete Memory Values

Users can delete all memory values of a user instead of separately delete one memory value.

1. Press 【MENU】 button to enter the system menu, select 【DELETE DATA】 option to enter its interface, select the user whose data to be deleted, after confirming again, all measurement results of the selected user will be deleted.

2. Finish Operation

Select 【CONFIRM】 or 【EXIT】 to return to the previous menu, or hold 【ON/OFF】 button to turn the device off.

Chapter 11 SpO₂ Measurement Function(Optional)

Precautions during SpO₂ Measurement:

❶The device may not be suitable for all animals, if you can't g't a satisfactory result, please stop using it.

❷Data averaging and signal processing generate delay on displayed and transmitted SpO₂ value. The data update period is shorter than 30 s, the time for obtaining dynamic average values will increase, which is arisen from signal degradation, low perfusion or other interference, it depends on the PR value.

❸The measurement results may be affected by too long or too much hair. When measuring, insert the part with least hair into the probe sufficiently, or remove the hair of the part to be measured when necessary.

❹The light between the photoelectric receiving tube and the light-emitting tube of the device must pass through the animal's arteriole. Make sure the optical path is free from any optical obstacles like rubberized fabric, to avoid inaccurate results.

❺Excessive ambient light may affect the measured results, such as surgical light (especially xenon light sources), bilirubin lamp, fluorescent lamp, infrared heater and direct sunlight. In order to prevent interference from ambient light, make sure to place the sensor properly and cover it with opaque material.

❻Frequent movement (active or passive) or severe activity of the animal will affect the measurement accuracy.

❼The probe should not be placed on a limb with the blood pressure cuff, arterial ductus or intraluminal tube.

Chapter 12 SpO₂ Measurement Method

(This chapter is only suitable for European Union market)

- 1)Long press the ON/OFF button to turn on the device to enter the main interface.

- 2)Pinch the clip so that the tongue or ear of the animal is fully inserted into the clip of the probe, as shown in the Figure below.

Place SpO₂ probe

- 3)Insert the SpO₂ probe into the USB jack of the device, then it will automatically switch to the SpO₂ display interface from the main interface.


Chapter 13 Error Message

Error message will be displayed in the screen if there is something wrong when measuring. The causes and solutions are shown as follows:

Error Message	Causes	Solutions
Self-test failure System failure	Function abnormal	Please contact us
Loose cuff	Cuff is not connected correctly.	Correctly connect cuff (refer to Chapter 9)
Air leakage	Cuff plug fall off	Make sure the cuff plug is securely inserted in the windpipe (refer to Chapter 9)
Air pressure error	Air pressure error	Refer to the troubleshooting
Weak signal	The pulse signal is too weak or the cuff is loose.	Correctly connect cuff (refer to Chapter 9)
Overpressure	Cuff is blocked or squeezed	Correctly connect cuff (refer to Chapter 9)
Excessive movement Over range Saturated signal	The signal extent is too big owing to the limbs or body moving or other reasons when measuring	Keep limbs, body still, measure again
Time out	It takes too much time	

Chapter 14 Troubleshooting

Abnormal Phenomenons	Causes	Solutions
BP measurement values too high or too low.	Cuff is not connected correctly.	Correctly connect cuff (refer to Chapter 9)
	Move limbs when measuring	Keep quiet and restart a measurement
No pressure	Cuff leakage	Buy a new cuff
	The cuff windpipe is not correctly connected with cuff	Correctly connect
	Cuff is not inflated	Stop using the device and contact us
Cuff deflates in short time	Loose cuff	Correctly apply cuff
It can not carry on measurement when press the measurement button		Switch on the power once again and restart a measurement

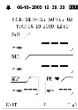Power off suddenly when inflating	No use for a long time, the power of batteries can be exhausted owing to the changed temperature	Replace all four batteries with new ones.
Hold the on/off button but can not start the device	Power of batteries can be exhausted	Replace all four batteries with new ones.
	The battery polarities is reversed	Check the battery installation for proper placement of the battery polarities.
Cuff inflation start before press the measurement button or never stop inflating when measuring		Pull out the cuff to deflate. Stop using the device and contact us.
Cuff never deflation		Pull out the cuff to deflate. Stop using the device and contact us.
Air pressure error	No deflation or deflation error or inflation without stop	Pull out the cuff to deflate. Stop using the device and contact us.
	Others	Keep limbs, body still, measure again.
No press value displayed or the value unchanged or change erratically when cuff inflated		Pull out the cuff to deflate. Stop using the device and contact us.
Other phenomenon		Switch on the power once again and restart an operation. Replace the batteries. If no, please contact us.

Chapter 15 Keys and Symbols

Symbol	Description	Symbol	Description
	Caution: read instructions (warnings) carefully		Open the over-limit prompt sound indication
SYS	Systolic pressure		Close the over-limit prompt sound indication
MAP	Mean pressure		Low-power
DIA	Diastolic pressure		Full-power
PR	Pulse rate (bpm)		1. No NIBP data to review 2. No finger inserted to SpO2 probe(Optional) 3. An indicator of signal inadequacy
LARGE	Large animal		Class II applied
MIDDLE	Middle animal		WEEE disposal
SMALL	Small animal		Type BF Applied Parts
INFO	Information		Serial number
	USB socket connect SpO2 probe (This item only suitable for European Union market)		Interface for connecting cuff
	Socket for power adapter		This item is compliant with Directive 2014/30/EU of the European Parliament and of the 26 February 2014 and Directive 2001/95/EC of the European Parliament and of the Council of 2001.
	Product code		Authorized representative
	This way up		Keep away from sunlight
	Keep in a cool, dry place		Imported by
	Temperature limit		Atmospheric pressure limit
	Follow instructions for use		Humidity limit
	Fragile, handle with care		Lot number
	Manufacturer		Manufacture Date

Chapter 16 Maintenance, Cleaning and Keeping

*Please do obey the precautions and correct operating methods in this user manual. Otherwise, we will not responsible for any fault.

Warning

Remove the batteries before cleaning. The accessories and main unit must be separated for cleaning.

Maintenance is not allowed during device using.

Do not squeeze the rubber tube on the cuff.

Caution

- High pressure disinfection to the device and accessories is not allowed.
- Do not let water or cleaning agent flow into the socket to avoid device damage.
- Do not soak the device and accessories in liquid.
- If any damage or deterioration of the device and accessories is found, please do not use it.

Maintenance:

- Clean the device and accessories regularly. It is recommended to clean them every one month. When the device or accessory gets dirty, use a dry and soft cloth to wipe. If they are very dirty, it is available to dip the soft cloth into water or mild detergent, and wring out, then use the cloth for cleaning.
- The device should be inspected and calibrated periodically (or obey the requirements of the hospital). It is available to inspect in the state specified inspection institution or by professional personal, or you can contact our company.Long press "USER" button in main interface for 5s to enter the calibration interface.

Advice

- Do not use gasoline, volatile oil, diluent, etc. to wipe the device.
- Do not clean or wet the cuff.

Storage:

Advice

- Do not expose the device in direct sunlight for long time, otherwise the display screen maybe damaged.
- The basic performance and safety of the device are not affected by the dust or cotton wool in home environment, while the device shall not be placed where with high temperature, humidity or dusty.
- Aged cuff may result in inaccurate measurement, please replace the cuff periodically according to the user manual.
- To avoid device damage, keep the device out the reach of children and pets.
- Avoid the device close to extreme high temperature such as fireplace, otherwise the device performance may be affected.
- Do not store the device with chemical medicine or corrosive gas.
- Do not place the device where there is water.
- Do not place the device where with slope, vibration or impact
- Take the batteries out if the device is not to be used for three months or longer.

Chapter 17 NIBP Specification

Name	Veterinary Blood Pressure Monitor
Display mode	2.8" color LCD Display
Measurement method	Oscillometric method
Working mode	Automatic
Operation mode	Continuous operation
Measurement Range	Pressure:0~297mmHg (0~39.6kPa)
	Pulse: 40~240bpm
Resolution	Pressure: 1mmHg(0.1 kPa)
	Pulse: 1bpm
Accuracy	Static pressure: ±3 mmHg(±0.4 kPa)
	Pulse: 5 bpm or 5% select lager
Operating	Temperature:+5 ℃~+40 ℃ „Measurement accuracy not Affected altitude
	Humidity:15 %RH~85 %RH(Non-condensing)
	Altitude:0 ~3000m
Transport	Transport by general vehicle or according to the order contract, avoid pounded, shake and splash by rain and snow in transportation.
Storage	Temperature: -20 ℃~+55 ℃; Relative humidity: ≤95 %; No corrosive gas and drafty.
Atmospheric pressure	700 hPa~1060 hPa
Power supply	4 "AA" alkaline batteries, AC Adapter(AC, 100 V-240 V, optional)
Rated current	600 mA
Battery life	When the temperature is 23 ℃, limb circumference is 270 mm, the measured blood pressure is normal, 4 "AA" alkaline batteries cab be used about 300 times.
Dimensions	130(L)*110(W)*80(H) mm
Unit Weight	300 gram(without batteries)
Service life	The service life of the device is five years.
Date of manufacturer	See the label

Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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