PROFESSIONAL MEDICAL PRODUCTS

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ENGLISH

DESCRIPTION/INTENDED USE:

These devices are single-use supplied sterile. Check plug matching on generator base and connect. Press CUT/yellow button for cutting and COAG/blue button for coagulation.

CONTRAINDICATIONS:

These devices should never be used when:

- There is visible evidence of damage to the exterior of the device such as cracked plastic or connector damage.
- These devices fail the inspection described herein.
- In the presence of flammable gases, liquids, and/or oxygen enriched environments.

SAFETY TIPS:

- Use lowest possible power setting on electrosurgical unit capable of achieving desired surgical effect.
- Never allow the cables connected to these devices to be in contact with skin of the patient or operator during electrosurgical activations.
- Do not permit the cables connected to these devices to be parallel and in close proximity to the leads of other electrical devices.
- Always place unused electrosurgical accessories in a safe insulated location such as a holster when not in use.
- Inspect and test each device before each use.
- Discard devices that have reached their life expectancy.

- Activation of an electrosurgical device when not in contact with target tissue or in position to deliver energy to target tissue (fulguration) may cause capacitive coupling.

INSPECTION:

These devices should be inspected before and after each use. Visually examine the devices for obvious physical damage including:

- Cracked, broken or otherwise distorted plastic parts.
- Broken or significantly bent connector contacts.
- Damage including cuts, punctures, nicks, abrasion, unusual lumps, significant discoloration.
- Tips for damage, corrosion or misalignment condition.

STORAGE & HANDLING:

These devices must be stored in a clean, cool and dry area. Protect from mechanical damage and direct sunlight. Handle with extreme care.

	Keep away from sunlight		Keep in a cool, dry place		Temperature limit
	Caution: read instructions (warnings) carefully		Medical Device complies with Directive 93/42/EEC		Product code
	Manufacturer		Consult instructions for use		Lot number
	Date of manufacture		Expiration date		Authorized representative in the European community
	Disposable device, do not re-use		Sterilized using ethylene oxide		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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