## CAMICE STANDARD MONOUSO IN SMS

### ITALIANO

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## DISPOSABLE STANDARD SMMS GOWN

## ENGLISH

**Product Name** Disposable Standard SMMS Gown**Product Code** FGS-SG-SM50-M, FGS-SG-SM50-L, FGS-SG-SM50-XL, FGS-SG-SM50-XXL**Product Size** Unit: cm

| GIMA CODE                | 21046         | 21047         | 21048          | 21049           |
|--------------------------|---------------|---------------|----------------|-----------------|
| FULL GUARD CODE          | FGS-SG-SM50-M | FGS-SG-SM50-L | FGS-SG-SM50-XL | FGS-SG-SM50-XXL |
| Sizes                    | 120x140 (M)   | 120x150 (L)   | 130x150 (XL)   | 140x160 (XXL)   |
| Length (Shoulder to hem) | 120           | 120           | 130            | 140             |
| Width (monolayer)        | 140           | 150           | 150            | 160             |
| Belt length              | 65            | 65            | 65             | 69              |
| Sleeve Length            | 58            | 59            | 59             | 61              |
| Deviation                | ±2            | ±2            | ±2             | ±2              |

**Product Model** Gown Type. M/L/XL/XXL**Structure and Composition** The product uses 50g SMMS non-woven fabric. It is composed of front, back, sleeve, ties and cuff. The technology process is by cutting, ultrasonic welding, ties, velcro, cuff, folding, sealing.**Intended use** These items are sterilized by EO method and they are used as a barrier to liquid and microbial penetration between patient and operation room, other surgical devices and the operation room personnel.**Contraindication** The products can be used only once and

should be used immediately after opening the package. If the package is damaged, the product is wet or moldy. Don't use them.

**How to use the product**

Open the package and take out the product, gently shake the gown, lift the sides of the collar, insert the hands into the sleeves, extend the arms forward and tie the belts.

**Storage** It should be stored in a clean room with relative humidity not exceeding 80%, no corrosive gas, stored in a cool, dry and well ventilated.**Warning**

1. This product is disposal product, don't reuse.
2. This product is single use, never share your product with another.
3. It is forbidden to use when the packaging or the products are damaged.

**Notice:** Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the Member State in which the user or patient is established.**Validity** 3 years.**Date of production** See date on packaging**Date of expiration** See date on packaging**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

|  |                                 |  |                                                        |  |                                                 |
|--|---------------------------------|--|--------------------------------------------------------|--|-------------------------------------------------|
|  | Product code                    |  | Keep away from sunlight                                |  | Keep in a cool, dry place                       |
|  | Lot number                      |  | Medical Device compliant with Regulation (EU) 2017/745 |  | Caution: read instructions (warnings) carefully |
|  | Manufacturer                    |  | Date of manufacture                                    |  | Medical Device                                  |
|  | Sterilized using ethylene oxide |  | Expiration date                                        |  | Don't use if package is damaged                 |
|  | Humidity limit                  |  | Consult instructions for use                           |  | Disposable device, do not re-use                |
|  | Temperature limit               |  | Authorized representative in the European community    |  | Fragile, handle with care                       |
|  | This side up                    |  | Unique device identifier                               |  |                                                 |

GIMA 21046  
GIMA 21047  
GIMA 21048  
GIMA 21049

Xianning Full Guard Medical Products Co., Ltd  
Yongan East Road Wenquan Economic Development Zone, Xianning City, Hubei Province, China  
Made in China

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Gima S.p.A.