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GIMA 21052
GIMA 21053
GIMA 21054
GIMA 21055

Xianning Full Guard Medical Products Co., Ltd
 Yongnan East Road Wenquan Economic Development Zone, Xianning City, Hubei Province, China
 Made in China

REF FGS-RF-SM50-M
 FGS-RF-SM50-L
 FGS-RF-SM50-XL
 FGS-RF-SM50-XXL

ZOUSTECH S.L.
 Pso. Castellana, 141 - Planta 19,
 28046 - Madrid, Spain

Importato da:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

0123

REINFORCED SURGICAL GOWNS - STERILE

ENGLISH

Product Name Disposable medical standard SMMS Gown (Sterile)

Product Code FGS-RF-SM50-M, FGS-RF-SM50-L, FGS-RF-SM50-XL, FGS-RF-SM50-XXL

Product Size Unit: cm

GIMA CODE	21052	21053	21054	21055
FULL GUARD CODE	FGS-RF-SM50-M	FGS-RF-SM50-L	FGS-RF-SM50-XL	FGS-RF-SM50-XXL
Sizes	120x140 (M)	120x150 (L)	130x150 (XL)	140x160 (XXL)
Length (Shoulder to hem)	120	120	130	140
Width (monolayer)	140	150	150	160
Belt length	65	65	65	69
Sleeve Length	58	59	59	61
Deviation	±2	±2	±2	±2

Product Model Gown Type M/L/XL/XXL

Structure and Composition The product uses 50g SMMS non-woven fabric, It is composed of front,back,sleeve,ties and cuff. the technology process is by cutting, ultrasonic welding, ties, velcro, cuff, folding,sealing.

Intended use The products are mainly used for general isolation in outpatient clinics, wards, inspection rooms, cleaning room, laboratory etc. It is intended to be used by healthcare professionals in hospitals and clinics to maintain a good level of hygiene and contribute to infection control.

Contraindication The products can be used only once and should be used immediately after opening the package. If the package is damaged,the product is wet or moldy. Don't use them.

How to use the product

Open the package and take out the product, gently shake the gown, lift the sides of the collar, insert the hands into the sleeves, extend the arms forward and tie the belts.

Storage It should be stored in a clean room with relative humidity not exceeding 80%, no corrosive gas, stored in a cool, dry and well ventilated.

⚠ Warning

1. This product is disposal product, don't reuse.
2. This product is single use, never share your product with another.
3. It is forbidden to use when the packaging or the products are damaged.

Notice: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the Member State in which the user or patient is established.

Validity 3 years.

Date of production See date on packaging

Date of expiration See date on packaging

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF	Product code	Keep away from sunlight	Keep in a cool, dry place	MD	Medical Device
LOT	Lot number	Medical Device compliant with Regulation (EU) 2017/745	Expiration date	Temperature limit	
Manufacturer	Date of manufacture	Sterilized using ethylene oxide	This side up		
Humidity limit	Don't use if package	Authorized representative in the European community	Fragile, handle with care		
Disposable device, do not re-use	Consult instructions for use	Caution: read instructions (warnings) carefully	Unique device identifier		

GIMA 21052
GIMA 21053
GIMA 21054
GIMA 21055

Xianning Full Guard Medical Products Co., Ltd
Yongan East Road Wenquan Economic Development Zone, Xianning City, Hubei Province, China
Made in China

FGS-RF-SM50-M
FGS-RF-SM50-L
FGS-RF-SM50-XL
FGS-RF-SM50-XXL

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