## TRUGLYDE FAST®

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ENGLISH

## DESCRIPTION

Truglyde Fast is an absorbable, sterile, surgical suture, composed of homopolymers of Glycolide (100%). For added lubrication and smoothness, Truglyde Fast is coated with unique combination of Polycaprolactone and Calcium Stearate. Truglyde Fast meets all the requirements, established by the United States Pharmacopeia for Absorbable Surgical suture. Truglyde Fast is available as undyed. Truglyde Fast is available in range of gauge sizes and lengths, non-needled or attached to stainless steel needles of varying types and sizes.

## INTENDED PURPOSE

Truglyde Fast is intended for use in soft tissue approximation where only short-term wound support is required and where the rapid absorption would be beneficial. Due to the absorption profile Truglyde Fast is useful for skin closure, particularly in pediatric surgery, episiotomies, circumcision and closure of oral mucosa. Truglyde Fast is not intended for use in ophthalmic surgery cardiovascular and neurological procedures.

## APPLICATION

Sutures should be selected and implanted depending on patient condition, surgical experience, surgical technique and wound size. Truglyde Fast typically falls off in 7 – 10 days post operatively or can be wiped off subsequently with sterile gauze. Normally the removal of the sutures is not required.

## PERFORMANCE

Truglyde Fast sutures elicit a minimal to moderate initial inflammatory tissue reaction. Progressive loss of tensile strength and eventual absorption of Truglyde Fast sutures occur by means of hydrolysis, where the polymer degrades to Glycolic acid. It gets absorbed quickly and metabolized in

the site of implantation. Absorption begins with loss of tensile strength followed by loss of mass. Truglyde Fast shows that 7 days post implantation approximately 50% of the original strength remains and 14 days post implantation approximately 10% of the original strength remains. The absorption of Truglyde Fast occurs thereafter and is essentially complete Approximately 42 days.

## CONTRA INDICATIONS

Truglyde Fast should not be used where extended approximation of tissues under stress is required or where wound support beyond 7 days is required due to the rapid loss of tensile strength.

## WARNINGS/PRECAUTIONS/INTERACTIONS

Users should be familiar with surgical procedures and techniques involving absorbable sutures before employing Truglyde Fast for wound closure as risk of wound dehiscence may vary with the site of application and the suture material used. Surgeons should consider the invivo performance when selecting a suture. As with any foreign body, prolonged contact of this suture or any other suture with salt solutions such as those found in urinary and biliary tracts, may result in calculus formation. As an absorbable suture, it may act transiently as a foreign body. Acceptable surgical practice should be followed for the management of contaminated or infected wounds. As Truglyde Fast is an absorbable suture materials, the use of supplemental nonabsorbable sutures should be considered by the surgeon in the closure of sites which may undergo expansion, stretching or distention or which may require additional support. Skin sutures which must remain in place longer than 7 days may cause localized irritation and should be snipped off or removed as indicated. Consideration should be taken in the use of absorbable sutures in tissues with poor blood supply as suture extrusion and delayed absorption may occur. Subcuticular sutures should be placed as deeply as possible to minimize the erythema and induration normally associated with the absorption process. The use of Truglyde Fast may be inappropriate in elderly, malnourished or debilitated patients, or patients suffering from conditions, which may delay, wound healing. In handling Truglyde Fast suture or any other suture material, care should be taken to avoid damage from handling. Avoid crushing or crimping damage due to application of surgical instruments such as forceps or needle holders. As with any suture material, adequate knot security requires the accepted surgical techniques of flat and square ties, with additional throws as warranted by surgical circumstances and the experience of the surgeon. Care should be taken to avoid damage when handling surgical needles.

Grasp the needle in an area one third (1/3) to one half (1/2) of the distance from the attachment end to the point. Grasping in the point area could impair the penetration performance & cause fracture of the needle.

Grasping at the but or attachment end could cause bending or breakage of the needle. Reshaping needles may cause them to loose strength and make less resistant to bending and breaking. Users should exercise caution when handling surgical needles to avoid inadvertent needle stick injury. Discard used needles in 'Sharps' containers.

## ADVERSE REACTIONS

Adverse reactions associated with the use of the Truglyde Fast include: allergic response in certain patients, transient local irritation at the wound site, followed by moderate transient inflammatory foreign body response, erythema and induration during the absorption process of subcuticular su-

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**Healthium Medtech Limited**
**MED DEVICES LIFESCIENCES B.V.**