Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

TRUWAX®

CERA STERILE PER OSSA 2,5GM

STERILE BONEWAX 2.5GM

CIRE D'OS STÉRILE 2,5GM

STERILES KNOCHENWACHS 2,5G

CERA ÓSEA ESTÉRIL 2.5GM

BONEWAX ESTÉRIL 2.5GM

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ENGLISH

DESCRIPTION

TRUWAX is sterile bone wax, a sterile mixture of White Bees wax, Hard Paraffin and Isopropyl Palmitate, a wax - softening agent. Truwx is opaque and has a waxy odor.

INDICATIONS

TRUWAX may be used for the control of bleeding from bone surfaces.

ACTION

TRUWAX achieves local haemostasis of bone by acting as a mechanical (tamponade) barrier. It does not act biochemically and is minimally resorbable.

CONTRAINDICATIONS

TRUWAX should not be used where rapid osseous regeneration and fusion are desired.

WARNINGS

TRUWAX should not be re sterilized or subjected to excessive heat. TRUWAX may inhibit osteogenesis and may act as a physical barrier to the reparative process. Do not use if package is opened or damaged. Discard opened, unused bonewax.

PRECAUTIONS

TRUWAX should be used sparingly. Excess Truwx should be re-moved from the operative site. The package should be opened just prior to use to minimize the possibility of contamination and excessive drying.

ADVERSE REACTIONS

Mild inflammatory reactions have been reported in tissues immediately adjacent to the site of implantation. Studies have suggested that bone wax as a foreign body may impair the ability of cancellous bone to clear bacteria. In animal models, the local accumulation of foreign body giant cells has been observed and histologic examination has revealed the appearance of macrophages and occasionally polymorphonuclear leukocytes and lymphocytes.

DOSAGE ADMINISTRATION

TRUWAX should be used immediately after removal from the package. Using aseptic technique, Truwx should be warmed to desired consistency by manipulation with the fingers or by immersion of the unopened foil packet in a warm sterile solution.

HOW SUPPLIED

TRUWAX is available sterile in individual foil envelopes, each containing 2.5 grams and packed in an individually sealed overwrap packet.

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Healthium Medtech Limited
MED DEVICES LIFESCIENCES B.V.