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INTENDED USE

The electrode is applied to the surface of the body to transmit the electrical signal at the body surface to a recording and monitoring device that produces an electrocardiogram or vectorcardiogram.

CONTRAINDICATIONS

Note known.

DIRECTIONS FOR USE

- Check that the package, the electrodes and the gel are intact; in case of any damage do not use the product.
- If needed, clean application areas with a mild soap and water and dry thoroughly.
- For adults only: Clip off hair on excessively hairy areas if necessary. Only gently abrade if skin impedance is too high.
- The correct placement of electrodes is locally determined by the hospital/physician and/or local protocol/procedures.WARNINGS
- Peel the electrode away from the protective liner and place it on the skin. Dispose of the protective liner immediately after removal of the electrode.
- Place the electrode on the skin by gently pressing around the adhesive edge with a circular smooth and firm motion.
- Connect the leadwire from the ECG monitoring device to the electrode. If the electrode is supplied with a pre-attached leadwire it may be connected directly to the ECG monitoring device.
- Electrode Removal: Gently lift the tab or the edge of the electrode, maintaining a low profile, and slowly pull off the electrode from the skin, holding and supporting down the newly exposed skin as the electrode is removed. Keep the electrode close to the skin surface while pulling back and gently removing the electrode at 45 degrees angle, to reduce the risk of mechanical trauma.
- Gel residue can be removed using paper towels or water.
- Close the bag carefully so that the remaining electrodes do not dry out.

As a general precept, replacement of the electrodes is recommended after 48 hours and mandatory after 72 hours.

WARNINGS

- Surface electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas).
- Do not use only for reuse of the product may cause contamination, leading to infections.
- After removal of electrodes from the protective liner, the protective liner should be disposed of in assigned waste bin.
- Do not use surgical procedure electrodes should be placed as far as possible from the electro-surgical area to minimize unwanted RF current flow which could lead to skin burns.
- Electrodes should only be used by or in consultation with a health care provider familiar with the proper placement and use.
- Do not soak, trim, or sterilize the electrodes as these procedures may leave harmful residues or cause malfunction of the electrodes.
- To minimize the risk of skin irritation, when replacing electrodes find a new area of skin on which to place them.

Rx ONLY - CAUTION: U.S. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

The equipment and its accessories are to be used with these products must comply with the regulations in force. Do not use the products on patients with previous episodes of skin irritation or contact allergies.

The electrodes comply with the biocompatibility standard ISO 10993-1.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified in the pouch's label. Avoid contact with external heavy weights on the package, the product could be damaged.

GENERAL NOTES

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority for any malfunction or defect of the device, inform the Manufacturer's Quality Service.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force.

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