SGABELLO DA DOCCIA PU con braccioli
PU SHOWER STOOL with armrest
TABOURET DE DOUCHE PU avec accoudoir
TABURETE DE DUCHA PU con reposabrazos
BANQUETE DE DUCHE PU com apoio de braço
PU-DUSCHHOCKER mit Armlehne PU
DUSCHPALL med armstöd
ΣΚΑΜΠΟΝ ΜΠΑΝΙΟΥ PU με υποβραχιόνιο

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HE10678103

KD ALUMINUM SHOWER CHAIR WITHOUT BACKREST

Parts of the product

1. Foam Armrest
2. PU Seat
3. Height adjustable foot tube
4. Tip

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- When using a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way.
- Never use the footrests as assistance when getting up - risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user.

- This information can be found in the technical data in these instructions for use as well as on the identification plate.

Assembly

Step 1

Attach armrests to the frame, by depressing push buttons and sliding the armrest pipes into the tubes located on each side of the seat.

**Step 2**

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs.

Make sure that push button “snap” correctly through holes and are stable and adjusted the same way

Technical data

Product	KD ALUMINUM SHOWER CHAIR W/O BACKREST
Product number	HE10678103
Weight capacity (lbs/kg)	300 lbs/ 136 kg
Product dimension (mm) L x W x H	448-472)×(500-526)×(622-719)mm
Product weight (lbs/kg)	4.0kg
Seat dimension (inch/mm) L x W	407*337*52mm
Backrest material	PU
Seat material	PU
Arm pad material	EVA Foam
Frame material	Aluminum
Frame color	Sliver
Quantity in box (piece)	1
Product weight including box (lbs/kg)	5.0kg
Carton dimension L x W x H	460*220*415mm

Symbols

	Caution: read instructions (warnings) carefully		Keep in a cool, dry place
	Manufacturer		Date of manufacture
	Lot number		Product code
	Medical Device compliant with Regulation (EU) 2017/745		Authorized representative in the European community
	Medical Device		Consult instructions for use
	Serial number		Imported by

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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