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1. PU Backrest
2. PU Seat
3. Height adjustable foot tube
4. Tip

## Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- When using a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way.
- Never use the footrests as assistance when getting up - risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate

## Assembly

### Step 1

**Attach backrest to the frame, by depressing push buttons and sliding the backrest pipes into the tubes located on each side and on the back of the seat.**

**Step 2**

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs.

Make sure that push button “snap” correctly through holes and are stable and adjusted the same way.

**Technical data**

|                                              |                                 |
|----------------------------------------------|---------------------------------|
| <b>Product</b>                               | KD ALUMINUM SHOWER CHAIR        |
| <b>Product number</b>                        | HE10688023                      |
| <b>Weight capacity (lbs/kg)</b>              | 300 lbs/ 136 kg                 |
| <b>Product dimension (mm) L x W x H</b>      | (505-516)×(500-526)×(735-835)mm |
| <b>Product weight (lbs/kg)</b>               | 4.02kg                          |
| <b>Seat dimension (inch/mm) L x W</b>        | 407*337*52mm                    |
| <b>Backrest material</b>                     | PU                              |
| <b>Seat material</b>                         | PU                              |
| <b>Frame material</b>                        | Aluminum                        |
| <b>Frame color</b>                           | Sliver                          |
| <b>Quantity in box (piece)</b>               | 1                               |
| <b>Product weight including box (lbs/kg)</b> | 5.74kg                          |
| <b>Carton dimension L x W x H</b>            | 460×200×430mm                   |

**Symbols**

|                                                                                     |                                                        |                                                                                     |                                                     |
|-------------------------------------------------------------------------------------|--------------------------------------------------------|-------------------------------------------------------------------------------------|-----------------------------------------------------|
|  | Caution: read instructions (warnings) carefully        |  | Keep in a cool, dry place                           |
|  | Manufacturer                                           |  | Date of manufacture                                 |
| <b>LOT</b>                                                                          | Lot number                                             | <b>REF</b>                                                                          | Product code                                        |
| <b>CE</b>                                                                           | Medical Device compliant with Regulation (EU) 2017/745 | <b>EC REP</b>                                                                       | Authorized representative in the European community |
| <b>MD</b>                                                                           | Medical Device                                         |  | Consult instructions for use                        |
| <b>SN</b>                                                                           | Serial number                                          |  | Imported by                                         |

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

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