SEDIA DA DOCCIA GIREVOLE con schienale e braccioli in PU**ROTATING SHOWER CHAIR with PU backrest and seat****CHAISE DE DOUCHE ROTATIVE avec dossier et siège PU****SILLA DE DUCHA GIRATORIA con respaldo y asiento en PU****CADEIRA DE DUCHE ROTATIVO com encosto e assento em PU****DREHBARER DUSCHSTUHL mit PU-Rückenlehne und -Sitz****ROTTERANDE DUSCHSTOL med ryggstöd och sits i PU****ΠΕΡΙΣΤΡΕΦΟΜΕΝΗ ΚΑΡΕΚΛΑ ΜΠΑΝΙΟΥ με πλάτη και κάθισμα PU**

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HE10688023

KD ALUMINUM SHOWER CHAIR WITH ROATING SEAT

Parts of the product

- 1. PU Backrest**
- 2. Foam Armrest**
- 3. PU Seat**
- 4. Rotating lever**
- 5. Rotating plate system**
- 6. Flip-up button of armrest**
- 7. Height adjustable foot tube**
- 8. Tip**

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- When using a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way.
- Never use the footrests as assistance when getting up - risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate.

Assembly

Step 1

Attach backrest to the frame, by depressing push buttons and sliding the backrest pipes into the tubes located on each side and on the back of the seat.

Step 2

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs. Make sure that push button “snap” correctly through holes and are stable and adjusted the same way.

**Step 3**

Check stability, the seat shall not be

unstable or shaky. Using the rotating shower chair

(1) Lift up the lever and turn the seat to face the user, then lock the seat into position by lowering the lever and keep swiveling the seat until it locks in position.

(2) Unlock the seat by lifting the lever. (3) When users are seated on the chair position lock the seat and ensure that it is locked into position and no longer rotates freely.

(4) Lock the shower chair when the user faces the open side of the bath. The user can now stand up carefully and slowly.

(5) Using the armrests, press on the button and lift backwards the armrests.

**Dati tecnici**

Product	KD ALUMINUM SHOWER CHAIR WITH ROATING SEAT
Product number	HE13741000
Weight capacity (lbs/kg)	300 lbs/ 136 kg
Product dimension (mm) L x W x H	506*(475-500)*(812-914)mm
Product folding dimension L x W x H	510*345*400mm
Product weight (lbs/kg)	5.37kg
Seat dimension (inch/mm) L x W	407*337*52mm
Backrest material	PU
Seat material	PU
Arm pad material	EVA Foam
Frame material	Aluminum
Frame color	Sliver
Quantity in box (piece)	1
Product weight including box (lbs/kg)	6.74kg
Carton dimension L x W x H	525*360*425mm

Symbol

	Caution: read instructions (warnings) carefully		Keep in a cool, dry place
	Manufacturer		Date of manufacture
	Lot number		Product code
	Medical Device compliant with Regulation (EU) 2017/745		Authorized representative in the European community
	Medical Device		Consult instructions for use
	Serial number		Imported by

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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