**GIMA BRUSH - SPAZZOLINI
CITOLOGIA NON STERILI
GIMA BRUSH - NOT STERILE
BROSSETTE GIMA - NON
STÉRILE
GIMA BRUSH - NICHT STERIL
GIMA BRUSH - NO ESTÉRIL
ESCOVA GIMA - NÃO ESTÉRIL
GIMA BRUSH - NENÍ STERILNÍ
GIMA BRUSH - OXI
ΑΠΟΣΤΕΙΩΜΕΝΟ
GIMA BRUSH - EI STERIILI
GIMA ČETKICA - NIJE STERILNA**

ITALIANO

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ENGLISH

Brush for taking cytologic endocervical samples

The brush consists of nylon bristles held by a metal winding. It is fixed on a plastic stem.

It is indicated for taking cytologic endo-

cervical samples.

The device must exclusively be used by qualified, trained personnel authorised to perform the PAP test.

Disposable device:

Any reuse could cause the risk of cross-contaminations and/or failed sterility or cleaning.

- Do not expose to sources of heat;
- Extract the device from the package immediately before use;
- Before use, visually examine the device to check its integrity.

Intended use:

ENDOBRUSH must be used for taking uterine cervix endocervical cells

Indications:

It is indicated also when there is cervical canal stenosis

Contra-indications:

it is not suitable during pregnancy

Directions for use:

insert the speculum into the vagina to put in evidence the uterine cervix, open the package, insert the endobrush slowly into the endocervical canal, make a complete rotation, withdraw slowly, rub the removed material on the slide. After use, dispose of as normal hospital waste.

- All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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GIMA S.p.A.