PROFESSIONAL MEDICAL PRODUCTS

SCARPA POST-OPERATORIA
POST OP SHOE
CHAUSSURE POST-OPÉRATOIRE
ZAPATO POSTOPERATORIO
SAPATO POST OP
POSTOPERATIVER SCHUH
POST OP SHOE
BUT POOPERACYJNY
POOPERAČNÍ BOTA
POST OP SHOE
LEIKKAUKSEN JÄLKEINEN JALKINE
ČEVELJ ZA PO OPERACIJI
POSTOPERAČNÁ OBUV
ÎNCĂLȚĂMINTE
POSTOPERATIEVE SCHOEN
POST OP CIPELA
MŰTÉT UTÁNI CIPŐ
POST OP SKO
ПОСТОПЕРАТИВНА ОБУВКА
POOPERACINIS BATAS
PĒCOPERĀCIJAS APAVI
OPERATSIOONIJÄRGNE JALANÕ

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Product name:

Post Op Shoe

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended use should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings, and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

Provide support and protection to the foot following surgery or trauma to the foot.

PERFORMANCE CHARACTERISTICS:

Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS:

None.

WARNINGS AND PRECAUTIONS:

This product is to be used under the supervision of a medical professional namely a physician and not intended for unsupervised public use.

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or packaging has been opened. If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and the competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS:

1. Unfasten ankle strap and forefoot closure. Slip foot into the shoe.

2. Secure both strap and forefoot closure for a snug but comfortable fit.

CLEANING INSTRUCTIONS:

To clean the shoe, wipe with a damp cloth and air dry.

SHELF LIFE

2 years

FOR SINGLE PATIENT USE ONLY

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY, AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

Notice to the user and/or patient:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

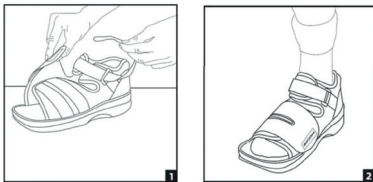
GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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