PROFESSIONAL MEDICAL PRODUCTS

TUTORE PER BRACCIO E SPALLA
SHOULDER & ARM BRACE
ATTELLE POUR ÉPAULE ET BRAS
ORTESIS DE BRAZO Y HOMBRO
ÓRTESE DE OMBRO E BRAÇO
SCHULTER- UND ARMORTHESE
ΝΑΡΘΗΚΑΣ ΩΜΟΥ & ΒΡΑΧΙΟΝΑ
ORTEZA STAWU BARKOWEGO I RAMIENNEGO
RAMENNÍ A PAŽNÍ ORTÉZA
AXEL-OCH-ARMBANDAGE
OLKAPÄÄ- JA KÄSIVARSITUKI
OPORNICA ZA RAMO IN ROKO
RAMENNÁ ORTÉZA
ORTEZĂ PENTRU UMĂR ȘI BRAȚ
SCHOUDER- EN ARMSPALK
STEZNIK ZA RAME I RUKU
VÁLL- ÉS KARMEREVÍTŐ
SKULDER- OG ARMBØJLE
OPTEZA ZA RAMO И РЪКА
PETIES IR RANKOS ĮTVARAS
PLECU UN ROKU BALSTI
ÕLA- JA KÄETUGI

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GIMA 42000 – 42001 – 42002 - 42003

XIAMEN ORTOCARE SPORTS HEALTH CO., LTD.
Unit 218-2, New Times Building, No.1 Huli Avenue, Xiamen, China
(Fujian) Pilot Free Trade Zone
MADE IN CHINA

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Shoulder & Arm Brace

Intended use:

To be used by who suffering shoulder surgery, shoulder dislocation or subluxation.

Usage:

1. Detach all the belts & straps.
2. First, wrap the thoracic belt.
3. Second, wrap the arm belt and attach it to the thoracic belt.
4. Finally, wrap the wrist belt and attach to the thoracic belt.
5. Attach the strap over contralateral shoulder to wrist belt and thoracic belt.

Cleaning Methods / Maintenance:

- Washing in cold water, gentle cycle and a mild soap.
- Do not dry clean or tumble dry.
- Do not use bleach.
- Dry in an air current, and do not expose this brace to any heat source.

Precaution and Warning:

Please consult with your physician if any allergic symptom occurs. Do not use more than 24 hours of continuous.

Storage:

Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Shelf Life:

5 years

Disposal:

Please dispose the product after use to comply with local regulation.

Notice to the user and/or patient:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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