TUTORE PER GOMITO**ELBOW BRACE****ATTELLE POUR LE COUDE****ORTESIS DE CODO****IMOBILIZADOR DE COTOVELO****ELLENBOGENORTHESIE****ΝΑΡΘΗΚΑΣ ΑΓΚΩΝΑ****ORTEZA STAWU ŁOKCIOWEGO****LOKETNÍ ORTÉZA****ARMBÅGSBANDAGE****KYYNÄRPÄÄTUKI****OPORNICA ZA KOMOLEC****LAKŤOVÁ ORTÉZA****ORTEZĂ PENTRU COT****ELLEBOOGSPALK****STEZNIK ZA LAKAT****KÖNYÖKMEREVÍTŐ****ALBUEBØJLE****ОРТЕЗА ЗА ЛАКЪТ****ALKŪNĒS ĪTVARAS****ELKOŅA BALSTS****KÜÜNARNUKIORTOOS**

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GIMA 42010 – 42011 - 42040

XIAMEN ORTOCARE SPORTS HEALTH CO., LTD.

Unit 218-2, New Times Building, No.1 Huli Avenue, Xiamen, China (Fujian) Pilot

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Elbow Brace

Intended use:

Correct postural and medical immobilization Helps immobilize and provide support on elbow joint.

Usage:

1. Measure circumference of elbow.
2. Refer to sizing chart to select proper size..
3. Loosen the strap of Brace and pass elbow through it.
4. Put pad over painful site.
5. Fasten the strap.

Cleaning Methods / Maintenance:

- Washing in cold water, gentle cycle and a mild soap.
- Do not dry clean or tumble dry.
- Do not use bleach.
- Dry in an air current, and do not expose this brace to any heat source.
- Remove metal splint or hinge prior to washing.

Precaution and Warning:

Please consult with your physician if any allergic symptom occurs.
Do not use more than 24 hours of continuous.

Storage:

Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Shelf Life:

5 years

Disposal:

Please dispose the product after use to comply with local regulation.

Notice to the user and/or patient:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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