TUTORE PER POLSO**INSTRUCTIONS FOR USE****WRIST BRACE****ORTESIS DE MUÑECA****IMOBILIZADOR DE PULSO****HANDGELENKORTHESE****ΝΑΡΘΗΚΑΣ ΚΑΡΠΟΥ****ORTEZA NADGARSTKA****ZÁPĚSTNÍ ORTÉZA****HANDLEDSBANDAGE (SV)****RANNETUKI****OPORNICA ZA ZAPESTJE (SI)****ZÁPÄSTNÁ ORTÉZA****ORTEZĂ PENTRU ÎNCHEIETURA MÂINII****POLSSPALK****STEZNIK ZA RUČNI ZGLOB****CSUKLÓMEREVÍTŐ****HÅNDLEDSBØJLE (DK)****ΟΡΤΕΖΑ ΖΑ ΚΙΤΚΑ****RIEŠO JTVARAS****PLAUKSTAS BALSTS****RANDMEORTOOS**

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XIAMEN ORTOCARE SPORTS HEALTH CO., LTD.
Unit 218-2, New Times Building, No.1 Huli Avenue, Xiamen, China
(Fujian) Pilot Free Trade Zone
MADE IN CHINA

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Wrist Brace

Intended use:

Correct postural and medical immobilization Helps immobilize and provide support on wrist area.

Usage:

1. Choose an appropriate size from size information..
2. Loosen linking tapes. Put on the brace to affected hand.
3. Pass your thumb through thumb tunnel and make sure the metal stays on the palm side.
4. Fasten the strap through the hook to the dorsal side..
5. Secure circumferential wrist strap to ensure that palmar stay is properly seated.

Cleaning Methods / Maintenance:

- Remove metal splints and close all hook and loop closures before washing.
- Hand wash braces with mild detergent in cold water, and hang dry only (Not near a heating source or in a tumble drier).

- Do not dry clean, bleach and iron.

Precaution and Warning:

Please consult with your physician if any allergic symptom occurs. Do not use more than 24 hours of continuous.

Storage:

Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Shelf Life:

5 years

Disposal:

Please dispose the product after use to comply with local regulation.

Notice to the user and/or patient:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

FRANÇAIS

Name:

Wrist Brace

Intended use:

Correct postural and medical immobilization Helps immobilize and provide support on wrist area.

Usage:

1. Choose an appropriate size from size information..
2. Loosen linking tapes. Put on the brace to affected hand.
3. Pass your thumb through thumb tunnel and make sure the metal stays on the palm side.
4. Fasten the strap through the hook to the dorsal side..
5. Secure circumferential wrist strap to ensure that palmar stay is properly seated.

Cleaning Methods / Maintenance:

- Remove metal splints and close all hook and loop closures before washing.
- Hand wash braces with mild detergent in cold water, and hang dry only (Not near a heating source or in a tumble drier).
- Do not dry clean, bleach and iron..

Precaution and Warning:

Please consult with your physician if any allergic symptom occurs. Do not use more than 24 hours of continuous.

Storage:

Store the product in a cool dry place away from sunlight and heat and preferably in the original packaging.

Shelf Life:

5 years

Disposal:

Please dispose the product after use to comply with local regulation.

Notice to the user and/or patient:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

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