



**Multi-Drug Rapid Test 1-Step Cup
With/Without Adulteration (Urine)
Package Insert**

REF

DOA-R127-A1	DOA-R137-A1	DOA-R147-A1	DOA-R157-A1
DOA-R167-A1	DOA-R177-A1	DOA-R187-A1	DOA-R197-A1
DOA-R1107-A1	DOA-R1117-A1	DOA-R1127-A1	DOA-R1137-A1
DOA-R1147-A1	DOA-R1157-A1	DOA-R1167-A1	DOA-R1177-A1
DOA-R1187-A1			
DUA-R127-A1	DUA-R137-A1	DUA-R147-A1	DUA-R157-A1
DUA-R167-A1	DUA-R177-A1	DUA-R187-A1	DUA-R197-A1
DUA-R1107-A1	DUA-R1117-A1	DUA-R1127-A1	DUA-R1137-A1
DUA-R1147-A1	DUA-R1157-A1	DUA-R1167-A1	DUA-R1177-A1

Instruction Sheet for testing of any combination of the following drugs:

ACE/AMP/IBAR/BZO/BUPI/COC/THC/MTD/MET/MDMA/MOP/MQL/OPI/PCP/PPX/TCAT/ML/ KET/OXY/COT/EDDP/FYL/K2/6-MAM/MDA/ETG/CLO/LSD/MPD/ZOL/DIA/ZOP/MCAT/7-ACL /CFYL/CAF/CAT/TRO/MDPV/MEP/ALP/ABP(K3)/α-PVP/CNB/MPRD/PGB/TZD/UR-144(K4) /ZAL/MES/GAB/TLDO/QTP/PAPI/KRA/ICAR/IFLX/CIT/FKET/OZP/IRPD/ITAP/INND/ISCOPI/ MTZ/ALC

Including Specimen Validity Tests (S.V.T.) for:

Oxidants/PCC, Specific Gravity, pH, Nitrite, Glutaraldehyde, Creatinine and Bleach

A rapid test for the simultaneous, qualitative detection of multiple drugs and drug metabolites in human urine. For healthcare professionals including professionals at point of care sites. Immunoassay for in vitro diagnostic use only.

【INTENDED USE AND SUMMARY】

The Multi-Drug Rapid Test is a rapid chromatographic immunoassay for the qualitative detection of multiple drugs and drug metabolites in urine at the following cut-off concentrations that can be performed with the use of the Cup Reader.

Test	Calibrator	Cut-off (ng/mL)
Acetaminophen (ACE)	Acetaminophen	5,000
Amphetamine (AMP)	d-Amphetamine	1,000/500/300
Barbiturates (BAR)	Secobarbital	300/200
Benzodiazepines (BZO)	Oxazepam	500/300/200/100
Buprenorphine (BUP)	Buprenorphine	10/5
Cocaine (COC)	Benzoylcegonine	300/200/150/100
Marijuana (THC)	11-nor-Δ ⁹ -THC-9 COOH	300/200/150/50/30/25/ 20
Methadone (MTD)	Methadone	300/200
Methamphetamine (MET)	d-Methamphetamine	1,000/500/300
Methylenedioxymethamphetamine (MDMA)	d,l-Methylenedioxy- methamphetamine	1,000/500/300
Morphine (MOP/OPI)	Morphine	300/200/100
Methaqualone(MQL)	Methaqualone	300
Opiate (OPI)	Morphine	2,000/1000
Phencyclidine (PCP)	Phencyclidine	50/25
Propoxyphene (PPX)	Propoxyphene	300
Tricyclic Antidepressants (TCA)	Nortriptyline	1,000/500/300
Tramadol (TML)	Cis-Tramadol	500/300/200/100
Ketamine (KET)	Ketamine	1,000/500/300/100
Oxycodone (OXY)	Oxycodone	300/100
Cotinine(COT)	Cotinine	500/300/200/100/50/10
2-ethylidene-1,5-dimethyl- 3,3-diphenylpyrrolidine (EDDP)	2-ethylidene-1,5-dimethyl- 3,3-diphenylpyrrolidine	300/100
Fentanyl (FYL)	Fentanyl	20/10/100/200/300
Synthetic Marijuana (K2)	JWH-018、 JWH-073	50/30/25
6-mono-aceto-morphine (6-MAM)	6-mono-aceto-morphine	10
(±) 3,4-Methylenedioxy- Amphetamine (MDA)	(±) 3,4-Methylenedioxy- Amphetamine	500
Ethyl- β-D-Glucuronide (ETG)	Ethyl- β -D-Glucuronide	1,000/500/300
Clonazepam (CLO)	Clonazepam	400/150
Lysergic Acid Diethylamide (LSD)	Lysergic Acid Diethylamide	50/20/10
Methylphenidate (MPD)	Methylphenidate	300/150
Methylphenidate (MPD)	Ritalin acid	1,000
Zolpidem (ZOL)	Zolpidem	50
Diazepam (DIA)	Diazepam	300/200
Zopiclone (ZOP)	Zopiclone	50
Methcathinone (MCAT)	S(-)-Methcathinone	500
7-Aminoclonazepam (7-ACL)	7-Aminoclonazepam	300/200/100
Carfentanyl (CFYL)	Carfentanyl	500/250
Caffeine (CAF)	Caffeine	1,000

Cathine (CAT)	(+)-Norpseudoephedrine	150
Tropicamide (TRO)	Tropicamide	350
3, 4-methylenedioxypropylvalerone (MDPV)	3, 4-methylenedioxy pyrovalerone	1,000/500/300
Mephedrone (MEP)	Mephedrone	100/500
Alprazolam (ALP)	Alprazolam	100
AB-PINACA (ABP/K3)	AB-PINACA	10
α-Pyrrolidinovalephorphenone (α-PVP)	α-Pyrrolidinovalephorphenone	2,000/1,000/500/300
Cannabinol (CNB)	Cannabinol	500
Meperidine (MPRD)	Meperidine	100
Pregabalin (PGB)	Pregabalin	50,000/500
Trazodone (TZD)	Trazodone	200
UR-144/K4	UR-144 5-Pentanoic acid	25
Zaleplon (ZAL)	Zaleplon	100
Mescaline (MES)	Mescaline	100/300
Gabapentin (GAB)	Gabapentin	2,000
Tilidine (TLD)	Nortilidine	50
Quetiapine (QTP)	Quetiapine	1,000
Papaverine (PAP)	Papaverine	500
Kratom (KRA)	Mitragynine	300
Carisoprodol (CAR)	Carisoprodol	2,000/1,000
Fluoxetine (FLX)	Fluoxetine	500
Citalopram (CIT)	Citalopram	500
Fluoxetine (FKET)	Fluoxetine	1,000
Olanzapine (OZP)	Olanzapine	1,000
Risperidone (RPD)	Risperidone	150
Tapentadol (TAP)	Tapentadol	1,000
N,N-Dimethyltryptamine (NND)	N,N-Dimethyltryptamine	1,000
Scopolamine (SCOP)	Scopolamine	500
Mirtazapine (MTZ)	Desmethylmirtazapine	500
Test	Calibrator	Cut-off
Alcohol (ALC)	Alcohol	0.02%

Configurations of the Multi-Drug Rapid Test come with any combination of the above listed drug analytes with or without S.V.T. This assay provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.

【SUMMARY OF ADULTERATION】

Adulteration is the tampering of a urine specimen with the intention of altering the test results. The use of adulterants can cause false negative results in drug tests by either interfering with the screening test and/or destroying the drugs present in the urine. Dilution may also be employed in an attempt to produce false negative drug test results.

One of the best ways to test for adulteration or dilution is to determine certain urinary characteristics such as pH, specific gravity and creatinine and to detect the presence of oxidants/PCC, nitrites or glutaraldehyde in urine.

【PRINCIPLE (FOR DOA TESTS EXCLUDING ALCOHOL)】

During testing, a urine specimen migrates upward by capillary action. A drug, if present in the urine specimen below its cut-off concentration, will not saturate the binding sites of its specific antibody. The antibody will then react with the drug-protein conjugate and a visible colored line will show up in the test region of the specific drug dipstick. The presence of drug above the cut-off concentration will saturate all the binding sites of the antibody. Therefore, the colored line will not form in the test region.

A drug-positive urine specimen will not generate a colored line in the specific test region of the dipstick because of drug competition, while a drug-negative urine specimen will generate a line in the test region because of the absence of drug competition.

To serve as a procedural control, a colored line will always appear at the control region, indicating that proper volume of specimen has been added and membrane wicking has occurred.

【PRINCIPLE OF ADULTERATION】

Oxidants/PCC (Pyridiniumchlorochromate) tests for the presence of oxidizing agents such as bleach and hydrogen peroxide. Pyridiniumchlorochromate (sold under the brand name Urine Luck) is commonly used adulterant.¹ Normal human urine should not contain oxidants of PCC.

Specific gravity tests for sample dilution. The normal range is from 1.003 to 1.030. Values outside this range may be the result of specimen dilution or adulteration.

pH tests for the presence of acidic or alkaline adulterants in urine. Normal pH levels should be in the range of 4.0 to 9.0. Values outside of this range may indicate the sample has been altered.

Nitrite tests for commonly used commercial adulterants such as Klear and Whizzies. They work by oxidizing the major cannabinoid metabolite THC-COOH.² Normal urine should contain no trace of nitrite. Positive results generally indicate the presence of an adulterant.

Glutaraldehyde tests for the presence of an aldehyde. Adulterants such as Urin Aid and Clear Choice contain glutaraldehyde which may cause false negative results by disrupting the enzyme used in some immunoassay tests.² Glutaraldehyde is not normally found in urine; therefore, detection of glutaraldehyde in a urine specimen is generally an indicator of adulteration.

Creatinine is a waste product of creatine; an amino-acid contained in muscle tissue and found in urine.³ A person may attempt to foil a test by drinking excessive amounts of water or diuretics such as herbal teas to “flush” the system. Creatinine and specific gravity are two ways to check for dilution and flushing, which are the most common mechanisms used in an attempt to circumvent drug testing. Low Creatinine and specific gravity levels may indicate dilute urine. The absence of Creatinine (<5 mg/dL) is indicative of a specimen not consistent with human urine. **Bleach** tests for the presence of bleach bleach refers to a number of chemicals which remove color, whiten or disinfect, often by oxidation, Bleaches are used as household chemicals to whiten clothes and remove stains and as disinfectants. Normal human urine should not contain bleach.

【PRINCIPLE (FOR ALCOHOL)】

The urine Alcohol Rapid Test consists of a plastic strip with a reaction pad attached at the tip. On contact with alcohol, the reaction pad will change colors depending on the concentration of alcohol present. This is based on the high specificity of alcohol oxidase for ethyl alcohol in the presence of peroxidase and enzyme substrate such as TMB.

【REAGENTS (FOR DOA TESTS EXCLUDING ALCOHOL)】

Each test line contains anti-drug mouse monoclonal antibody and corresponding drug-protein conjugates. The control line contains goat anti-rabbit IgG polyclonal antibodies and rabbit IgG.

【REAGENTS (FOR ALCOHOL)】

Tetramethylbenzidine /Alcohol Oxidase/Peroxidase

【S.V.T REAGENTS】

Adulteration Pad	Reactive indicator	Buffers and non-reactive ingredients
Creatinine	0.04%	99.96%
Nitrite	0.07%	99.93%
Bleach	0.39%	99.61%
Glutaraldehyde	0.02%	99.98%
pH	0.06%	99.94%
Specific Gravity	0.25%	99.75%
Oxidants / PCC	0.36%	99.64%

【PRECAUTIONS】

- For healthcare professionals including professionals at point of care sites.
- Immunoassay for *in vitro* diagnostic use only. The test should remain in the sealed pouch until use.
- All specimens should be considered potentially hazardous and handled in the same manner as an infectious agent.
- The used test should be discarded according to local regulations.
- For use exclusively with the Cup Reader. **Do not interpret test results visually.**

【STORAGE AND STABILITY】

Store as packaged in the sealed pouch at 2-30 °C. The test is stable through the expiration date printed on the sealed pouch. The Test Cup must remain in the sealed pouch until use. **DO NOT FREEZE.** Do not use beyond the expiration date.

【SPECIMEN COLLECTION AND PREPARATION】

Urine Assay

The urine specimen should be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible precipitates should be centrifuged, filtered, or allowed to settle to obtain a clear specimen for testing.

Specimen Storage

Urine specimens may be stored at 2-8 °C for up to 48 hours prior to testing. For prolonged storage, specimens may be frozen and stored below -20 °C. Frozen specimens should be thawed and mixed well before testing. When testing cards with S.V.T. or Alcohol storage of urine specimens should not exceed 2 hours at room temperature or 4 hours refrigerated prior to testing.

【MATERIALS】

Materials Provided

- Test Cups
 - Package Insert
 - Procedure Card
- Materials Required But Not Provided**

- Timer
- Cup Reader


【DIRECTIONS FOR USE】

Allow the test, urine specimen, and/or controls to reach room temperature (15-30 °C) prior to testing.

- Bring the pouch to room temperature before opening it. Remove the cup from the sealed pouch and use it within 1 hour.
- Donor provides specimen.
- Technician replaces and secures cap while the cup is on a flat surface.
- Check the temperature label (Temp Label) up to 4 minutes after specimen collection. A green color will appear to indicate the temperature of the urine specimen. The proper range for an unadulterated specimen is 32-38 °C (90-100 °F).
- Technician dates and initials the security seal and attaches the security seal over the cup cap.
- Technician peels off the label on the multi-drug test cup to read results.
- Put the cup into cup reader detection chamber at 5 min** and close the chamber cap to read the results by the cup reader. **Do not read the test results visually.** Refer to your Drug Free Policy for guidelines on adulterated specimens. We recommend not interpreting the drug test results and either retest the urine or collect another specimen in case of any positive result for any adulteration test.

Note: For the Installation, startup, system calibration and complete test operations of the cup reader, please refer to the Cup Reader User Manual carefully. Operator must consult the Cup Reader User Manual prior to use and become familiar with the operations and quality control procedures.

TEST PROCEDURE Cup

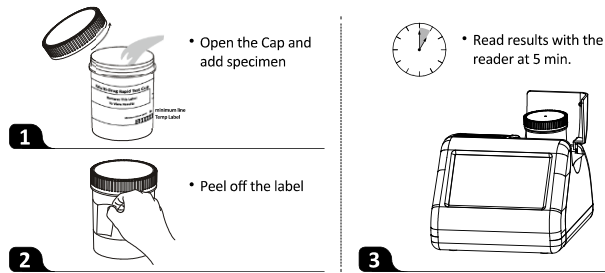

【INTERPRETATION OF DOA RESULTS】

● Results read by Cup Reader

The result of positive or negative for each analyte is determined by the Cup Reader.

Results Example:

Analyte	Display	Result
MOP	POS	Positive
OXY	NEG	Negative
AMP	INV	Invalid

NEGATIVE: This negative result means that the concentrations in the urine sample are below the designated cut-off levels for a particular drug tested.

POSITIVE: The positive result means that the drug concentration in the urine sample is greater than the designated cut-off for a specific drug.

INVALID: Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for Control line failure. Read the directions again and repeat the test with a new test. If the result is still invalid, contact your manufacturer.

【INTERPRETATION OF RESULTS (S.V.T/ ADULTERATION AND ALCOHOL)】

Results Example:

Analyte	Display	Result
ALC	NOR	Normal
OXI/SG	ABN/NOR	Abnormal/Normal
NIT/GLU	NOR/NOR	Normal/Normal
CRE/pH	ABN/ABN	Abnormal/Abnormal

【QUALITY CONTROL】

A procedural control is included in the test. A line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume, adequate membrane wicking and correct procedural technique.

Control standards are not supplied with this kit. However, it is recommended that positive and negative controls be tested as good laboratory practice to confirm the test procedure and to verify proper test performance.

【LIMITATIONS】

- The Multi-Drug Rapid Test provides only a qualitative, preliminary analytical result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method.^{4,5}
- There is a possibility that technical or procedural errors, as well as interfering substances in the urine specimen may cause erroneous results.
- Adulterants, such as bleach and/or alum, in urine specimens may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test should be repeated with another urine specimen.
- A positive result does not indicate level or intoxication, administration route or concentration in urine.
- A negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
- This test does not distinguish between drugs of abuse and certain medications.
- A positive test result may be obtained from certain foods or food supplements.

【S.V.T/ADULTERATION LIMITATIONS】

- The adulteration tests included with the product are meant to aid in the determination of abnormal specimens. While comprehensive, these tests are not meant to be an “all-inclusive” representation of possible adulterants.
- Oxidants/PCC: Normal human urine should not contain oxidants or PCC. The presence of high levels of antioxidants in the specimen, such as ascorbic acid, may result in false negative results for the oxidants/PCC pad.
- Specific Gravity: Elevated levels of protein in urine may cause abnormally high specific gravity values.
- Nitrite: Nitrite is not a normal component of human urine. However, nitrite found in urine may indicate urinary tract infections or bacterial infections. Nitrite levels of > 20 mg/dL may produce false positive glutaraldehyde results.
- Glutaraldehyde: is not normally found in urine. However certain metabolic abnormalities such as ketoacidosis (fasting, uncontrolled diabetes or high protein diets) may interfere with the test results.
- Creatinine: Normal Creatinine levels are between 20 and 350 mg/dL. Under rare conditions,

certain kidney diseases may show dilute urine.

- Bleach: Normal human urine should not contain bleach. The presence of high levels of bleach in the specimen may result in false negative results for the bleach pad.

【PERFORMANCE CHARACTERISTICS】

Accuracy % Agreement with GC/MS

	ACE 5,000	AMP 1,000	AMP 500	AMP 300	BAR 300	BAR 200	BZO 500	BZO 300	BZO 200	BZO 100	BUP 10
Positive Agreement	95.6%	97.1%	97.0%	96.7%	96.7%	95.0%	95.5%	96.0%	96.1%	95.8%	97.1%
Negative Agreement	98.7%	98.7%	98.6%	98.9%	99.0%	99.0%	99.0%	98.4%	98.4%	98.5%	99.0%
Total Results	97.6%	98.2%	98.1%	98.4%	98.5%	97.9%	98.0%	97.7%	97.8%	97.8%	98.5%

	BUP 5	COC 300	COC 200	COC 150	COC 100	THC 300	THC 150	THC 50	THC 25	THC 20	THC 200
Positive Agreement	96.4%	95.1%	96.3	95.7%	97.2%	95.8%	96.0%	96.3%	96.5%	96.1%	95.5%
Negative Agreement	98.9%	98.9%	98.9%	98.9%	98.9%	98.6%	99.0%	98.6%	99.0%	98.5%	99.2%
Total Results	98.3%	97.7%	98.3%	97.8%	98.5%	97.9%	97.9%	97.6%	98.1%	97.8%	97.9%

	THC 30	MET 1,000	MET 500	MET 300	MDMA 1,000	MDMA 500	MDMA 300	MOP/ OPI 300	MOP/ OPI 100	MOP/ OPI 200	PPX 300
Positive Agreement	96.2%	95.3%	96.0%	95.8%	95.2%	95.7%	96.2%	95.0%	95.2%	95.1%	99.2%
Negative Agreement	98.6%	99.2%	99.2%	98.6%	99.1%	99.3%	98.6%	99.0%	99.1%	98.9%	98.5%
Total Results	98.0%	97.8%	97.9%	97.6%	97.7%	98.1%	98.0%	97.6%	98.0%	97.7%	98.8%

	TCA 1,000	TCA 500	TCA 300	TML 100	TML 200	TML 300	TML 500	KET 1,000	KET 500	KET 300	KET 100
Positive Agreement	98.9%	97.6%	96.4%	96.9%	96.7%	96.9%	96.8%	99.1%	96.8%	98.9%	99.1%
Negative Agreement	98.7%	98.8%	99.4%	99.3%	99.4%	99.4%	99.4%	99.9%	98.7%	98.1%	99.3%
Total Results	98.8%	98.4%	98.4%	98.4%	98.4%	98.0%	98.4%	99.6%	98.0%	98.4%	99.2%

	OXY 100	OXY 300	COT 500	COT 200	COT 100	COT 50	COT 10	COT 300	ETG 500	ETG 1,000	ETG 300
Positive Agreement	97.5%	96.5%	96.5%	99.1%	98.2%	96.0%	95.0%	95.0%	95.3%	98.3%	97.4%
Negative Agreement	99.4%	99.4%	99.4%	98.5%	98.5%	99.3%	99.3%	98.8%	99.3%	98.5%	99.4%
Total Results	98.8%	98.4%	98.4%	99.0%	98.4%	98.0%	98.1%	97.6%	97.6%	98.4%	98.8%

	K2 50	K2 30	6-MAM 10	MDA 500	EDDP 300	EDDP 100	CLO 400	CLO 150	LSD 10	LSD 20	LSD 50
Positive Agreement	98.4%	95.5%	98.2%	96.2%	99.2%	98.1%	96.1%	99.2%	95.2%	97.9%	98.9%
Negative Agreement	99.2%	99.2%	99.3%	99.4%	99.2%	99.3%	98.6%	98.4%	98.8%	98.1%	99.4%
Total Results	98.8%	97.6%	98.8%	98.4%	99.2%	98.8%	97.6%	98.8%	97.6%	98.0%	99.2%

	MEP 100	MEP 500	ZOL 50	DIA 300	DIA 200	ZOP 50	MCAT 500	7-ACL 300	7-ACL 200	7-ACL 100	CFYL 500
Positive Agreement	97.7%	97.5%	98.1%	96.7%	97.0%	98.6%	95.5%	98.0%	95.2%	96.3%	97.8%
Negative Agreement	98.8%	98.8%	99.3%	99.9%	99.3%	98.9%	99.5%	99.3%	98.8%	98.8%	99.4%
Total Results	98.4%	98.4%	98.8%	98.8%	98.4%	98.8%	98.0%	98.8%	97.6%	98.0%	98.8%

	CAF 1,000	CAT 150	TRO 350	MDPV 1000	MDPV 500	α-PVP 300	α-PVP 2,000	α-PVP 500	α-PVP 1,000	CNB 500	MPRD 100
Positive Agreement	95.2%	98.8%	96.8%	97.7%	96.9%	98.5%	95.7%	98.6%	96.0%	96.9%	95.9%
Negative Agreement	99.5%	99.4%	99.4%	99.4%	99.5%	98.9%	99.4%	98.9%	98.7%	99.3%	99.3%
Total Results	98.4%	99.2%	98.4%	98.8%	98.8%	98.8%	98.0%	98.8%	97.6%	98.4%	98.0%

	PGB 50,000	TZD 200	UR-144 25	ZAL 100	MES 100	GAB 2,000	MQL 300	ALP 100	ABP 10	TLD 50	QTP 1,000
Positive Agreement	95.7%	97.2%	96.9%	97.0%	97.8%	97.8%	95.7%	97.5%	96.1%	97.3%	96.2%
Negative Agreement	98.7%	98.9%	99.5%	99.5%	99.5%	99.8%	99.1%	98.8%	99.4%	98.9%	98.8%
Total Results	97.6%	98.4%	98.8%	98.8%	98.8%	98.4%	98.1%	98.4%	98.4%	98.4%	98.0%

	PAP 500	KRA 300	TAP 1,000	FLX 500	K2 25	CIT 500	FKET 1,000	RPD 150	OPI 2,000	OPI 1,000	CFYL 250
Positive Agreement	96.4%	98.6%	95.2%	96.5%	99.1%	96.8%	97.2%	96.4%	95.1%	97.6%	96.3%
Negative Agreement	99.2%	99.3%	99.1%	99.3%	98.6%	99.3%	99.2%	99.1%	99.0%	99.4%	99.4%
Total Results	98.1%	99.0%	97.5%	98.5%	98.8%	98.5%	98.5%	98.0%	97.9%	98.8%	98.4%

	PGB 500	MES 300	OZP 1,000	MDPV 300	CAR 2,000	CAR 1,000	NND 1,000	SCOP 500	MTZ 500	MTD 300	MTD 200
Positive Agreement	97.4%	97.8%	96.2%	97.7%	96.8%	97.8%	97.0%	96.4%	96.8%	95.7%	96.4%
Negative Agreement	98.8%	99.4%	99.2%	99.4%	99.3%	98.8%	99.3%	98.8%	99.3%	98.9%	99.2%
Total Results	97.6%	98.8%	98.0%	98.8%	98.5%	98.4%	98.5%	98.0%	98.5%	98.3%	98.8%

	PCP 25	PCP 50	FYL 20	FYL 10	FYL 100	FYL 200	FYL 300	MPD 300	MPD 150	MPD 1,000
Positive Agreement	97.9%	97.4%	98.0%	98.1%	97.4%	97.9%	96.9%	98.0%	98.6%	96.8%
Negative Agreement	98.1%	99.4%	98.8%	98.6%	99.4%	98.1%	98.7%	98.0%	98.9%	99.3%
Total Results	98.0%	98.8%	98.6%	98.4%	98.8%	98.0%	98.0%	98.0%	98.8%	98.5%

Precision

A study was conducted at three hospitals by laypersons using three different lots of product to demonstrate the within run, between run and between operator precision. An identical card of coded specimens, containing drugs at concentrations of negative, - 50% and 300% cut-off level, was labeled, blinded and tested at each site. **The results gained 100% accuracy in negative, -50% and 300% cut-off level specimen.**

Analytical Sensitivity

A drug-free urine pool was spiked with drugs at the listed concentrations. The results are summarized below.

Drug Concentration Cut-off Range	ACE 5,000		AMP 1,000		AMP 500		AMP 300		BAR 300		BAR 200		BZO 500		BZO 300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	BZO 200		BZO 100		BUP 10		BUP 5		COC 300		COC 200		COC 150		COC 100	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	THC 300		THC 200		THC 150		THC 50		THC 30		THC 25		THC 20		LSD 10	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	FYL 100		FYL 200		FYL 300		FYL 20		FYL 10		OPI 2,000		OPI 1000		PPX 300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	MOP/ OPI 300		MOP/ OPI 100		RPD 150		KET 1,000		KET 500		KET 300		KET 100		MQL 300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	OXY 100		MDMA 1,000		MDMA 500		EDDP 300		EDDP 100		MPD 300		MPD 150		K2 50	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	K2 30		6-MAM 10		MDA 500		ETG 500		ETG 1,000		CLO 400		CLO 150	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	LSD 20		LSD 50		MDPV 300		ZOL 50		MDMA 300		OXY 300		DIA 300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	29	1	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	DIA 200		ZOP 50		MCAT 500		7-ACL 300		7-ACL 200		7-ACL 100		CFYL 500	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	29	1	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	CAF 1,000		CAT 150		TRO 350		MDPV 1,000		MEP 100		MEP 500		ALP 100	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	COT 500		COT 300		COT 200		COT 100		COT 50		COT 10		MPD 1,000	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	MDPV 500		ABP 10		TAP 1,000		CNB 500		MPRD 100		PGB 50,000		TZD 200	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	UR-144 25		ZAL 100		MES 100		GAB 2,000		MOP/OPI 200		ETG 300		K2 25	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	TLD 50		QTP 1,000		PAP 500		KRA 300		FLX 500		CAR 2000		CAR 1,000	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	PGB 500		MES 300		OZP 1,000		CIT 500		FKET 1,000		MTD 300		MTD 200	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0

-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	CFYL 250	α-PVP 2,000	α-PVP 500	α-PVP 300	α-PVP 1,000	NND 1,000	SCOP 500	MTZ 500		
-	+	-	+	-	+	-	+	-	+	
0% Cut-off	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	MET 1,000		MET 500		MET 300		PCP 25		PCP 50		TCA 1,000		TCA 500		TCA 300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Drug Concentration Cut-off Range	TML 100		TML 200		TML 300		TML 500	
	-	+	-	+	-	+	-	+
0% Cut-off	30	0	30	0	30	0	30	0
-50% Cut-off	30	0	30	0	30	0	30	0
+300% Cut-off	0	30	0	30	0	30	0	30

Analytical Specificity

The following table lists the concentrations of compounds (ng/mL) that are detected as positive in urine by the Multi-Drug Rapid Test at 5 minutes.

Analytes	Concentration (ng/mL)	Analytes	Concentration (ng/mL)
ACETAMINOPHEN (ACE 5,000)			
Acetaminophen	5,000		
AMPHETAMINE (AMP 1,000)			
D,L-Amphetamine sulfate	300	Phentermine	1,000
L-Amphetamine	25,000	Maprotiline	50,000
(±) 3,4-Methylenedioxy amphetamine	500	Methoxyphenamine	6,000
		D-Amphetamine	1,000
AMPHETAMINE (AMP 500)			
D,L-Amphetamine sulfate	150	Phentermine	500
L-Amphetamine	12,500	Maprotiline	25,000
(±) 3,4-Methylenedioxy amphetamine	250	Methoxyphenamine	3,000
		D-Amphetamine	500
AMPHETAMINE (AMP 300)			
D,L-Amphetamine sulfate	75	Phentermine	300
L-Amphetamine	10,000	Maprotiline	15,000
(±) 3,4-Methylenedioxy amphetamine	150	Methoxyphenamine	2,000
		D-Amphetamine	300
BARBITURATES (BAR 300)			
Amobarbital	5,000	Alphenol	600
5,5-Diphenylhydantoin	8,000	Aprobarbital	500
Allobarbital	600	Butabarbital	200
Barbital	8,000	Butalbital	8,000
Talbutal	200	Butethal	500
Cyclopentobarbital	30,000	Phenobarbital	300
Pentobarbital	8,000	Secobarbital	300
BARBITURATES (BAR 200)			
Amobarbital	3,000	Alphenol	400
5,5-Diphenylhydantoin	5,000	Aprobarbital	300
Allobarbital	400	Butabarbital	150
Barbital	5,000	Butalbital	5,000
Talbutal	150	Butethal	300
Cyclopentobarbital	20,000	Phenobarbital	200
Pentobarbital	5,000	Secobarbital	200
BENZODIAZEPINES (BZO 500)			
Alprazolam	200	Bromazepam	1,500
a-hydroxyalprazolam	2,500	Chlordiazepoxide	1,500
Clobazam	300	Nitrazepam	300
Clonazepam	800	Norchlordiazepoxide	200
Clorazepatedipotassium	800	Nordiazepam	1,500
Delorazepam	1,500	Oxazepam	500
Desalkylflurazepam	300	Temazepam	300
Flunitrazepam	300	Diazepam	500
(±) Lorazepam	5,000	Estazolam	10,000
RS-Lorazepamglucuronide	300	Triazolam	5,000
Midazolam	10,000		

BENZODIAZEPINES (BZO 300)			
Alprazolam	100	Bromazepam	900
a-hydroxyalprazolam	1,500	Chlordiazepoxide	900
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norchlordiazepoxide	100
Clorazepatedipotassium	500	Nordiazepam	900
Delorazepam	900	Oxazepam	300
Desalkylflurazepam	200	Temazepam	100
Flunitrazepam	200	Diazepam	300
(±) Lorazepam	3,000	Estazolam	6,000
RS-Lorazepamglucuronide	200	Triazolam	3,000
Midazolam	6,000		

BENZODIAZEPINES (BZO 200)			
Alprazolam	70	Bromazepam	600
a-hydroxyalprazolam	1,000	Chlordiazepoxide	600
Clobazam	120	Nitrazepam	120
Clonazepam	300	Norchlordiazepoxide	70
Clorazepatedipotassium	300	Nordiazepam	600
Delorazepam	600	Oxazepam	200
Desalkylflurazepam	120	Temazepam	70
Flunitrazepam	120	Diazepam	200
(±) Lorazepam	2,000	Estazolam	4,000
RS-Lorazepamglucuronide	120	Triazolam	2,000
Midazolam	4,000		

BENZODIAZEPINES (BZO 100)			
Alprazolam	40	Bromazepam	300
a-hydroxyalprazolam	500	Chlordiazepoxide	300
Clobazam	60	Nitrazepam	60
Clonazepam	150	Norchlordiazepoxide	40
Clorazepatedipotassium	150	Nordiazepam	300
Delorazepam	300	Oxazepam	100
Desalkylflurazepam	60	Temazepam	40
Flunitrazepam	60	Diazepam	100
(±) Lorazepam	1,000	Estazolam	2,000
RS-Lorazepamglucuronide	60	Triazolam	1,000
Midazolam	2,000		

BUPRENORPHINE (BUP 10)			
Buprenorphine	10	Norbuprenorphine	50
Buprenorphine 3-D-Glucuronide	50	Norbuprenorphine 3-D-Glucuronide	100

BUPRENORPHINE (BUP 5)			
Buprenorphine	5	Norbuprenorphine	25
Buprenorphine 3-D-Glucuronide	25	Norbuprenorphine 3-D-Glucuronide	50

COCAINE (COC 300)			
Benzoylcegonine	300	Cocaethylene	20,000
Cocaine HCl	200	Ecgonine	30,000
COCAINE (COC 200)			
Benzoylcegonine	200	Cocaethylene	13,500
Cocaine HCl	135	Ecgonine	20,000

COCAINE (COC 150)			
Benzoylcegonine	150	Cocaethylene	10,000
Cocaine HCl	120	Ecgonine	15,000

COCAINE (COC 100)			
Benzoylcegonine	100	Cocaethylene	7,000
Cocaine HCl	80	Ecgonine	10,000

MARIJUANA (THC 300)			
Cannabinol	200,000	Δ ⁸ -THC	100,000
11-nor-Δ ⁸ -THC-9 COOH	200	Δ ⁹ -THC	100,000
11-nor-Δ ⁹ -THC-9 COOH	300		

MARIJUANA (THC 200)			
Cannabinol	140,000	Δ ⁸ -THC	68,000
11-nor-Δ ⁸ -THC-9 COOH	120	Δ ⁹ -THC	68,000
11-nor-Δ ⁹ -THC-9 COOH	200		

MARIJUANA (THC 150)			
Cannabinol	100,000	Δ ⁸ -THC	50,000
11-nor-Δ ⁸ -THC-9 COOH	100	Δ ⁹ -THC	50,000
11-nor-Δ ⁹ -THC-9 COOH	150		

MARIJUANA (THC 50)			
Cannabinol	35,000	Δ ⁸ -THC	17,000
11-nor-Δ ⁸ -THC-9 COOH	30	Δ ⁹ -THC	17,000
11-nor-Δ ⁹ -THC-9 COOH	50		

MARIJUANA (THC 30)			
Cannabinol	20,000	Δ ⁸ -THC	10,000

11-nor-Δ ⁸ -THC-9 COOH	20	Δ ⁹ -THC	10,000
11-nor-Δ ⁹ -THC-9 COOH	30		
MARIJUANA (THC 25)			
Cannabinol	17,500	Δ ⁸ -THC	8,500
11-nor-Δ ⁸ -THC-9 COOH	15	Δ ⁹ -THC	8,500
11-nor-Δ ⁹ -THC-9 COOH	25		
MARIJUANA (THC 20)			
Cannabinol	14,000	Δ ⁸ -THC	6,800
11-nor-Δ ⁸ -THC-9 COOH	12	Δ ⁹ -THC	6,800
11-nor-Δ ⁹ -THC-9 COOH	20		
METHADONE (MTD 300)			
Methadone	300	Doxylamine	100,000
METHADONE (MTD 200)			
Methadone	200	Doxylamine	65,000
METHAMPHETAMINE (MET 1, 000)			
ρ-Hydroxymethamphetamine	25,000	(±)-3,4-Methylenedioxy-methamphetamine	1,600
D-Methamphetamine	1,000		
L-Methamphetamine	20,000	Mephentermine	50,000
METHAMPHETAMINE (MET 500)			
ρ-Hydroxymethamphetamine	12,500	(±)-3,4-Methylenedioxy-methamphetamine	800
D-Methamphetamine	500		
L-Methamphetamine	10,000	Mephentermine	25,000
METHAMPHETAMINE (MET 300)			
ρ-Hydroxymethamphetamine	7,500	(±)-3,4-Methylenedioxy-methamphetamine	500
D-Methamphetamine	300		
L-Methamphetamine	6,000	Mephentermine	15,000
METHYLENEDIOXYMETHAMPHETAMINE (MDMA 1,000) Ecstasy			
(±) 3,4-Methylenedioxy methamphetamine HCl	1,000	3,4-Methylenedioxyethyl-amp hetamine	600
(±) 3,4-Methylenedioxy amphetamine HCl	6,000		
METHYLENEDIOXYMETHAMPHETAMINE (MDMA 500) Ecstasy			
(±) 3,4-Methylenedioxy methamphetamine HCl	500	3,4-Methylenedioxyethyl-amp hetamine	300
(±) 3,4-Methylenedioxy amphetamine HCl	3,000		
METHYLENEDIOXYMETHAMPHETAMINE (MDMA 300) Ecstasy			
(±) 3,4-Methylenedioxy methamphetamine HCl	300	3,4-Methylenedioxyethyl-amp hetamine	180
(±) 3,4-Methylenedioxy amphetamine HCl	1,800		
MORPHINE (MOP/OPI 300)			
Codeine	200	Norcodeine	6,000
Levorphanol	1,500	Normorphone	50,000
Morphine-3-β-D-Glucuronide	800	Oxycodone	30,000
Ethylmorphine	6,000	Oxymorphone	50,000
Hydrocodone	50,000	Procaine	15,000
Hydromorphone	3,000	Thebaine	6,000
6-Monoacetylmorphine	300	Morphine	300
MORPHINE/OPIATE (MOP/OPI 200)			
Codeine	160	Norcodeine	4,000
Levorphanol	1,000	Normorphone	40,000
Morphine-3-β-D-Glucuronide	600	Oxycodone	20,000
Ethylmorphine	4,000	Oxymorphone	40,000
Hydrocodone	40,000	Procaine	10,000
Hydromorphone	2,000	Thebaine	4,000
6-Monoacetylmorphine	200	Morphine	200
MORPHINE (MOP/OPI 100)			
Codeine	80	Norcodeine	2,000
Levorphanol	500	Normorphone	20,000
Morphine-3-β-D-Glucuronide	300	Oxycodone	10,000
Ethylmorphine	2,000	Oxymorphone	20,000
Hydrocodone	20,000	Procaine	5,000
Hydromorphone	1,000	Thebaine	2,000
6-Monoacetylmorphine	100	Morphine	100
METHAQUALONE (MQL 300)			
Methaqualone	300		
MORPHINE/OPIATE (OPI 2,000)			
Codeine	2,000	Morphine	2,000
Ethylmorphine	3,000	Norcodeine	25,000
Hydrocodone	50,000	Normorphone	50,000
Hydromorphone	15,000	Oxycodone	25,000
Levorphanol	25,000	Oxymorphone	25,000
6-Monoacetylmorphine	3,000	Procaine	50,000
Morphine 3-β-D-glucuronide	2,000	Thebaine	25,000

MORPHINE/OPIATE (OPI 1,000)			
Codeine	1,000	Morphine	1,000
Ethylmorphine	1,500	Norcodeine	12,500
Hydrocodone	25,000	Normorphone	25,000
Hydromorphone	7,500	Oxycodone	12,500
Levorphanol	12,500	Oxymorphone	12,500
6-Monoacetylmorphine	1,500	Procaine	25,000
Morphine 3-β-D-glucuronide	1,000	Thebaine	12,500
PHENCYCLIDINE (PCP 25)			
Phencyclidine	25	4-Hydroxyphencyclidine	12,500
PHENCYCLIDINE (PCP 50)			
Phencyclidine	50	4-Hydroxyphencyclidine	25,000
PROPOXYPHENE (PPX 300)			
D-Propoxyphene	300	D-Norpropoxyphene	300
TRICYCLIC ANTIDEPRESSANTS (TCA 1,000)			
Nortriptyline	1,000	Imipramine	400
Nordoxepine	500	Clomipramine	50,000
Trimipramine	3,000	Doxepine	2,000
Amitriptyline	1,500	Maprotiline	2,000
Promazine	3,000	Promethazine	50,000
Desipramine	200	Perphenazine	50,000
Cyclobenzaprine	2,000	Dithiaden	10,000
TRICYCLIC ANTIDEPRESSANTS (TCA 500)			
Nortriptyline	500	Imipramine	200
Nordoxepine	250	Clomipramine	25,000
Trimipramine	1,500	Doxepine	1,000
Amitriptyline	750	Maprotiline	1,000
Promazine	1,500	Promethazine	25,000
Desipramine	100	Perphenazine	25,000
Cyclobenzaprine	1,000	Dithiaden	5,000
TRICYCLIC ANTIDEPRESSANTS (TCA 300)			
Nortriptyline	300	Imipramine	120
Nordoxepine	150	Clomipramine	15,000
Trimipramine	900	Doxepine	600
Amitriptyline	450	Maprotiline	600
Promazine	900	Promethazine	15,000
Desipramine	60	Perphenazine	15,000
Cyclobenzaprine	600	Dithiaden	3,000
TRAMADOL (TML 100)			
n-Desmethyl-cis-tramadol	200	o-Desmethyl-cis-tramadol	10,000
Cis-tramadol	100	Phencyclidine	100,000
Procyclidine	100,000	d,l-O-Desmethyl venlafaxine	50,000
TRAMADOL (TML 200)			
n-Desmethyl-cis-tramadol	400	o-Desmethyl-cis-tramadol	20,000
Cis-tramadol	200	Phencyclidine	200,000
Procyclidine	200,000	d,l-O-Desmethyl venlafaxine	100,000
TRAMADOL (TML 300)			
n-Desmethyl-cis-tramadol	600	o-Desmethyl-cis-tramadol	30,000
Cis-tramadol	300	Phencyclidine	300,000
Procyclidine	300,000	d,l-O-Desmethyl venlafaxine	150,000
TRAMADOL (TML 500)			
n-Desmethyl-cis-tramadol	1,000	o-Desmethyl-cis-tramadol	50,000
Cis-tramadol	500	Phencyclidine	500,000
Procyclidine	500,000	d,l-O-Desmethyl venlafaxine	250,000
KETAMINE (KET 1,000)			
Ketamine	1,000	Benzphetamine	25,000
Dextromethorphan	2,000	(+) Chlorpheniramine	25,000
Methoxyphenamine	25,000	Clonidine	100,000
d-Norpropoxyphene	25,000	EDDP	50,000
Promazine	25,000	4-Hydroxyphencyclidine	50,000
Promethazine	25,000	Levorphanol	50,000
Pentazocine	25,000	MDE	50,000
Phencyclidine	25,000	Meperidine	25,000
Tetrahydrozoline	500	d-Methamphetamine	50,000
Mephentermine	25,000	l-Methamphetamine	50,000
(1R, 2S) - (-)-Ephedrine	100,000	3,4-Methylenedioxy-methamphetamine (MDMA)	100,000
Disopyramide	25,000	Thioridazine	50,000
KETAMINE (KET 500)			
Ketamine	500	Benzphetamine	12,500
Dextromethorphan	1,000	(+) Chlorpheniramine	12,500
Methoxyphenamine	12,500	Clonidine	50,000
d-Norpropoxyphene	12,500	EDDP	25,000
Promazine	12,500	4-Hydroxyphencyclidine	25,000
Promethazine	12,500	Levorphanol	25,000
Pentazocine	12,500	MDE	25,000

Phencyclidine	12,500	Meperidine	12,500
Tetrahydrozoline	250	d-Methamphetamine	25,000
Mephentermine	12,500	l-Methamphetamine	25,000
(1R, 2S) - (-)-Ephedrine	50,000	3,4-Methylenedioxy-methamphetamine (MDMA)	50,000
Disopyramide	12,500	Thioridazine	25,000
KETAMINE (KET 300)			
Ketamine	300	Benzphetamine	6,250
Dextromethorphan	600	(+) Chlorpheniramine	6,250
Methoxyphenamine	6,250	Clonidine	30,000
d-Norpropoxyphene	6,250	EDDP	15,000
Promazine	6,250	4-Hydroxyphencyclidine	15,000
Promethazine	6,250	Levorphanol	15,000
Pentazocine	6,250	MDE	15,000
Phencyclidine	6,250	Meperidine	6,250
Tetrahydrozoline	150	d-Methamphetamine	15,000
Mephentermine	6,250	l-Methamphetamine	15,000
(1R, 2S) - (-)-Ephedrine	30,000	3,4-Methylenedioxy-methamphetamine (MDMA)	30,000
Disopyramide	6,250	Thioridazine	15,000
KETAMINE (KET 100)			
Ketamine	100	Benzphetamine	2,000
Dextromethorphan	200	(+) Chlorpheniramine	2,000
Methoxyphenamine	2,000	Clonidine	10,000
d-Norpropoxyphene	2,000	EDDP	5,000
Promazine	2,000	4-Hydroxyphencyclidine	5,000
Promethazine	2,000	Levorphanol	5,000
Pentazocine	2,000	MDE	5,000
Phencyclidine	2,000	Meperidine	2,000
Tetrahydrozoline	50	d-Methamphetamine	5,000
Mephentermine	2,000	l-Methamphetamine	5,000
(1R, 2S) - (-)-Ephedrine	10,000	Thioridazine	5,000
Disopyramide	2,000	3,4-Methylenedioxy-methamphetamine (MDMA)	10,000
OXYCODONE (OXY 100)			
Oxycodone	100	Hydromorphone	50,000
Oxymorphone	300	Naloxone	25,000
Levorphanol	50,000	Naltrexone	25,000
Hydrocodone	25,000		
OXYCODONE (OXY 300)			
Oxycodone	300	Hydromorphone	150,000
Oxymorphone	900	Naloxone	75,000
Levorphanol	150,000	Naltrexone	75,000
Hydrocodone	75,000		
COTININE (COT 500)			
(-)-Cotinine	500	(-)-Nicotine	12,500
COTININE (COT 300)			
(-)-Cotinine	300	(-)-Nicotine	7,500
COTININE (COT 200)			
(-)-Cotinine	200	(-)-Nicotine	5,000
COTININE (COT 100)			
(-)-Cotinine	100	(-)-Nicotine	2,500
COTININE (COT 50)			
(-)-Cotinine	50	(-)-Nicotine	1,250
COTININE (COT 10)			
(-)-Cotinine	10	(-)-Nicotine	250
2-ETHYLIDENE-1,5-DIMETHYL-3,3-DIPHENYLPYRROLIDINE (EDDP 300)			
2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)			300
2-ETHYLIDENE-1,5-DIMETHYL-3,3-DIPHENYLPYRROLIDINE (EDDP 100)			
2-Ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)			100
FENTANYL (FYL 20)			
Fentanyl	20	Cyclopro Fentanyl	500
Norfentanyl	>100,000	(±)cis-3-Methylfentanyl	500
Butyl fentanyl	300	Valeryl Fentanyl	200
Methoxyacetyl-Fentanyl	40	Acetyl Fentanyl	40
Ocfentanil	200	para-Fluorobutyl fentanil	200
4-Fluoro-isobutyl fentanyl	200	para-Fluorofentanil	100
FENTANYL (FYL 10)			
Fentanyl	10	Cyclopro Fentanyl	250
Norfentanyl	>100,000	(±)cis-3-Methylfentanyl	250
Butyl fentanyl	150	Valeryl Fentanyl	100
Methoxyacetyl-Fentanyl	20	Acetyl Fentanyl	20
Ocfentanil	100	para-Fluorobutyl fentanil	100
4-Fluoro-isobutyl fentanyl	100	para-Fluorofentanil	50
FENTANYL (FYL 100)			
Fentanyl	100	Cyclopro Fentanyl	2,500

Norfentanyl	>100,000	(±)cis-3-Methylfentanyl	2,500
Butyl fentanyl	1,500	Valeryl Fentanyl	1,000
Methoxyacetyl-Fentanyl	200	Acetyl Fentanyl	200
Ocfentanil	1,000	para-Fluorobutylfentanyl	1,000
4-Fluoro-isobutylfentanyl	1,000	para-Fluorofentanil	500
FENTANYL (FYL 200)			
Fentanyl	200	Cyclopro Fentanyl	5,000
Norfentanyl	>100,000	(±)cis-3-Methylfentanyl	5,000
Butyl fentanyl	3,000	Valeryl Fentanyl	2,000
Methoxyacetyl-Fentanyl	400	Acetyl Fentanyl	400
Ocfentanil	2,000	para-Fluorobutylfentanyl	2,000
4-Fluoro-isobutylfentanyl	2,000	para-Fluorofentanil	1,000
FENTANYL (FYL 300)			
Fentanyl	300	Cyclopro Fentanyl	7,500
Norfentanyl	>100,000	(±)cis-3-Methylfentanyl	7,500
Butyl fentanyl	4,500	Valeryl Fentanyl	3,000
Methoxyacetyl-Fentanyl	600	Acetyl Fentanyl	600
Ocfentanil	3,000	para-Fluorobutylfentanyl	3,000
4-Fluoro-isobutylfentanyl	3,000	para-Fluorofentanil	1,500
SYNTHETIC MARIJUANA (K2-50)			
JWH-018 5-Pentanoic acid	50	JWH-073 4-butanoic acid	50
JWH-018 4-Hydroxypentyl	400	JWH-018 5-Hydroxypentyl	500
JWH-073 4-Hydroxybuty	500		
SYNTHETIC MARIJUANA (K2-30)			
JWH-018 5-Pentanoic acid	30	JWH-073 4-butanoic acid	30
JWH-018 4-Hydroxypentyl	250	JWH-018 5-Hydroxypentyl	300
JWH-073 4-Hydroxybuty	300		
SYNTHETIC MARIJUANA (K2-25)			
JWH-018 5-Pentanoic acid	25	JWH-073 4-butanoic acid	25
JWH-018 4-Hydroxypentyl	200	JWH-018 5-Hydroxypentyl	250
JWH-073 4-Hydroxybuty	250		
6-MONO-ACETO-MORPHINE (6-MAM 10)			
6-Monoacetylmorphine	10	Morphine	100,000
(±) 3, 4-METHYLENEDIOXYAMPHETAMINE (MDA 500)			
(±) 3, 4-Methylenedioxy amphetamine	500	Methoxyphenamine D-Amphetamine	5,000 2,000
D,L-Amphetamine sulfate	400	Phentermine	2,000
L-Amphetamine	30,000	Maprotiline	100,000
ETHYL- B-D-GLUCURONIDE(ETG 500)			
Ethyl- β -D-Glucuronide	500	Propyl β-D-glucuronide	50,000
Glucuronic Acid	100,000	Ethanol	>100,000
Methanol	>100,000		
ETHYL- B-D-GLUCURONIDE(ETG 1,000)			
Ethyl- β -D-Glucuronide	1,000	Propyl β-D-glucuronide	100,000
Glucuronic Acid	>100,000	Ethanol	>100,000
Methanol	>100,000		
ETHYL- B-D-GLUCURONIDE(ETG 300)			
Ethyl- β -D-Glucuronide	300	Propyl β-D-glucuronide	30,000
Glucuronic Acid	60,000	Ethanol	>100,000
Methanol	>100,000		
CLONAZEPAM(CLO 400)			
Clonazepam	400	Flunitrazepam	300
Alprazolam	200	(±) Lorazepam	1,250
a-hydroxyalprazolam	2,000	RS-Lorazepamglucuronide	250
Bromazepam	1,000	Midazolam	5,000
Chlordiazepoxide	1,000	Nitrazepam	200
Clobazam	250	Norchlordiazepoxide	200
Clorazepatedipotassium	600	Nordiazepam	1,000
Delorazepam	1,000	Oxazepam	350
Desalkylflurazepam	250	Temazepam	150
Diazepam	300	Triazolam	5,000
Estazolam	1,250		
CLONAZEPAM (CLO 150)			
Clonazepam	150	Flunitrazepam	120
Alprazolam	75	(±) Lorazepam	500
a-hydroxyalprazolam	750	RS-Lorazepamglucuronide	100
Bromazepam	400	Midazolam	2,000
Chlordiazepoxide	400	Nitrazepam	75
Clobazam	100	Norchlordiazepoxide	75
Clorazepatedipotassium	250	Nordiazepam	400
Delorazepam	400	Oxazepam	130
Desalkylflurazepam	100	Temazepam	60
Diazepam	120	Triazolam	2,000
Estazolam	500		
LYSERGIC ACID DIETHYLAMIDE (LSD 10)			
Lysergic Acid Diethylamide	10		

LYSERGIC ACID DIETHYLAMIDE (LSD 20)			
Lysergic Acid Diethylamide	20		
LYSERGIC ACID DIETHYLAMIDE (LSD 50)			
Lysergic Acid Diethylamide	50		
METHYLPHENIDATE (MPD 1,000)			
Methylphenidate (Ritalin)	350	Ritalinic Acid	1,000
METHYLPHENIDATE (MPD 300)			
Methylphenidate (Ritalin)	300	Ritalinic Acid	1,000
METHYLPHENIDATE (MPD 150)			
Methylphenidate (Ritalin)	150	Ritalinic Acid	500
ZOLPIDEM(ZOL 50)			
Zolpidem	50		
DIAZEPAM (DIA 300)			
Diazepam	300	Midazolam	6,000
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norchlordiazepoxide	100
Clorazepate dipotassium	500	Nordiazepam	900
Alprazolam	100	Flunitrazepam	200
a-hydroxyalprazolam	1,500	(±) Lorazepam	3,000
Bromazepam	900	RS-Lorazepam glucuronide	200
Chlordiazepoxide	900	Triazolam	3,000
Estazolam	6,000	Temazepam	100
Delorazepam	900	Oxazepam	300
Desalkylflurazepam	200		
DIAZEPAM (DIA 200)			
Diazepam	200	Midazolam	4,000
Clobazam	120	Nitrazepam	120
Clonazepam	300	Norchlordiazepoxide	70
Clorazepate dipotassium	300	Nordiazepam	600
Alprazolam	70	Flunitrazepam	120
a-hydroxyalprazolam	1,000	(±) Lorazepam	2,000
Bromazepam	600	RS-Lorazepam glucuronide	120
Chlordiazepoxide	600	Triazolam	2,000
Estazolam	4,000	Temazepam	70
Delorazepam	600	Oxazepam	200
Desalkylflurazepam	120		
ZOPICLONE (ZOP 50)			
Zopiclone-x-oxide	50	Zopiclone	50
METHCATHINONE (MCAT 500)			
S(-)-Methcathinone HCl	500	R(+)-Methcathinone HCl	1,500
Methoxyphenamine	100,000	3-Fluoromethcathinone HCl	1,500
7-AMINOCLONAZEPAM(7-ACL 300)			
a-hydroxyalprazolam	6,000	Flunitrazepam	3,000
Bromazepam	6,000	RS-Lorazepam glucuronide	2,700
Chlordiazepoxide	6,000	Norchlordiazepoxide	4,500
Clobazam	9,000	Nordiazepam	15,000
Clonazepam	2,400	Temazepam	9,000
Delorazepam	6,000	7-Aminoclonazepam	300
Desalkylflurazepam	6,000		
7-AMINOCLONAZEPAM(7-ACL 200)			
a-hydroxyalprazolam	4,000	Flunitrazepam	2,000
Bromazepam	4,000	RS-Lorazepam glucuronide	1,800
Chlordiazepoxide	4,000	Norchlordiazepoxide	3,000
Clobazam	6,000	Nordiazepam	10,000
Clonazepam	1,600	Temazepam	6,000
Delorazepam	4,000	7-Aminoclonazepam	200
Desalkylflurazepam	4,000		
7-AMINOCLONAZEPAM(7-ACL 100)			
a-hydroxyalprazolam	2,000	Flunitrazepam	1,000
Bromazepam	2,000	RS-Lorazepam glucuronide	900
Chlordiazepoxide	2,000	Norchlordiazepoxide	1,500
Clobazam	3,000	Nordiazepam	5,000
Clonazepam	800	Temazepam	3,000
Delorazepam	2,000	7-Aminoclonazepam	100
Desalkylflurazepam	2,000		
CARFENTANYL(CFYL 500)			
Carfentanyl	500	Fentanyl	100
Sufentanil	50,000	Ramifentanil	10,000
(±)cis-3-Menthylfentanyl	20,000	Butyl fentanyl	150
CARFENTANYL(CFYL 250)			
Carfentanyl	250	Fentanyl	50
Sufentanil	25,000	Ramifentanil	5,000
(±)cis-3-Menthylfentanyl	10,000	Butyl fentanyl	75
CAFFEINE (CAF 1,000)			
Caffeine	1,000		
CATHINE (CAT 150)			

(+)-Norpseudoephedrine HCl	150	(+)-3,4-Methylenedioxyamphet amine (MDA)	100
d/l-Amphetamine	100	p-Hydroxyamphetamine	100
Tryptamine	12,500	Methoxyphenamine	12,500
TROPICAMIDE (TRO 350)			
Tropicamide	350		
3, 4-METHYLENEDIOXYPYROVALERONE (MDPV 1,000)			
3, 4-methylenedioxy pyrovalerone	1,000		
3, 4-METHYLENEDIOXYPYROVALERONE (MDPV 500)			
3, 4-methylenedioxy pyrovalerone	500		
3, 4-METHYLENEDIOXYPYROVALERONE (MDPV 300)			
3, 4-methylenedioxy pyrovalerone	300		
MEPHEDRONE (MEP 100)			
Mephedrone HCl	100	R(+)-Methcathinone HCl	1500
S(-)-Methcathinone HCl	500	3-Fluoromethcathinone HCl	1500
4-Fluoromethcathinone HCl	300	Methoxyphenamine	100,000
MEPHEDRONE (MEP 500)			
Mephedrone HCl	500	R(+)-Methcathinone HCl	7,500
S(-)-Methcathinone HCl	2,500	3-Fluoromethcathinone HCl	7,500
4-Fluoromethcathinone HCl	1,500	Methoxyphenamine	500,000
ALPRAZOLAM(ALP 100)			
Benzodiazepines	300	Flunitrazepam	200
a-hydroxyalprazolam	1,500	(±) Lorazepam	3,000
Bromazepam	900	RS-Lorazepamglucuronide	200
Chlordiazepoxide	900	Midazolam	6,000
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norchlordiazepoxide	100
Clorazepatedipotassium	500	Nordiazepam	900
Delorazepam	900	Oxazepam	300
Desalkylflurazepam	200	Temazepam	100
Diazepam	300	Triazolam	3,000
Estazolam	6,000		
AB-PINACA (ABP 10)			
AB-PINACA	10	UR-144 4-hydroxypentyl	10,000
AB-PINACA 5-Pentanoic	10	APINACA 5-hydroxypentyl	10,000
AB-PINACA 5-hydroxypentyl	10	ADB-PINACA N-(5-hydroxypentyl)	30
AB-FUBINACA	10	ADB-PINACA Pentanoic Acid	10
AB-PINACA 4-hydroxypentyl	10,000	5-fluoro AB-PINACA N-(4-hydroxypentyl)	30
UR-144 5-Pentanoic	5,000	5-fluoro AB-PINACA	25
UR-144 5-hydroxypentyl	10,000		
ALPHA-PYRROLIDINOVALEROPHENONE (A-PVP 2,000)			
alpha-Pyrrolidinovalerophenone	2,000		
ALPHA-PYRROLIDINOVALEROPHENONE (A-PVP 1,000)			
alpha-Pyrrolidinovalerophenone	1,000		
ALPHA-PYRROLIDINOVALEROPHENONE (A-PVP 500)			
alpha-Pyrrolidinovalerophenone	500		
ALPHA-PYRROLIDINOVALEROPHENONE (A-PVP 300)			
alpha-Pyrrolidinovalerophenone	300		
CANNABINOL (CNB 500)			
cannabinol	500	11-nor-Δ ⁹ -THC-9 COOH	300
Δ ⁹ -THC	10,000		
MEPERIDINE (MPRD 100)			
Normeperidine	100	Meperidine	100
PREGABALIN(PGB 50,000)			
Pregabalin	50,000		
PREGABALIN(PGB 500)			
Pregabalin	500		
TRAZODONE(TZD 200)			
Trazodone	200		
UR-144 25			
UR-144 5-Pentanoic acid	25	5-fluoro AB-Pinaca N-(4-hydroxypentyl)	10,000
UR-144 4-hydroxypentyl	10,000	ADB-PINAC N-(4-hydroxypentyl)	>10,000
UR-144 5-hydroxypentyl	5,000	AB-PINACA 4-hydroxypentyl	>10,000
XLR-11 4-hydroxypentyl	2,000		
ZALEPLON(ZAL 100)			
Zaleplon	100		
MESCALINE(MES 100)			
Mescaline	100		
MESCALINE(MES 300)			

Mescaline	300		
GABAPENTIN(GAB 2,000)			
Gabapentin	2,000		
TILIDINE(TLD 50)			
Nortilidine	50	Tilidine	100
QUETIAPINE(QTP 1,000)			
Quetiapine	1,000	Norquetiapine	10,000
PAPAVERINE(PAP 500)			
Papaverine	500	Diflunisal	1,000,000
Methortrexate	650,000	Methedrone	500,000
Pragablin	500,000	Phenelzine	8,000
Quinine	4,000		
KRATOM(KRA 300)			
Mitragynine	300	7-hydroxymitragynine	>50,000
CARISOPRODOL(CAR 2,000)			
Carisoprodol	2,000		
CARISOPRODOL(CAR 1,000)			
Carisoprodol	1,000		
FLUOXETINE(FLX 500)			
Fluoxetine	500		
OLANZAPINE(OZP 1,000)			
Olanzapine	1,000		
CITALOPRAM(CIT 500)			
Citalopram	500		
FLUOKETAMINE (FKET 1,000)			
2-(2-fluorophenyl)-2-methylamin o-cyclohexanone	1,000		
RISPERIDONE (RPD 150)			
Risperidone	150		
TAPENTADOL (TAP 1,000)			
3-((1R,2R)-3-(dimethylamino)-1- ethyl-2-methylpropyl)phenol	1,000		
N,N-DIMETHYLTRYPTAMINE(NND 1,000)			
N,N-Dimethyltryptamine	1,000		
SCOPOLAMINE(SCOP 500)			
Scopolamine	500	Atropine	3,000
MIRTAZAPINE(MTZ 500)			
Desmethylmirtazapine	500	Mirtazapine	500

Effect of Urinary Specific Gravity

Fifteen (15) urine samples of normal, high, and low specific gravity ranges (1.005-1.045) were spiked with drugs at 50% below and 50% above cut-off levels respectively. The Multi-Drug Rapid Test was tested in duplicate using fifteen drug-free urine and spiked urine samples. The results demonstrate that varying ranges of urinary specific gravity do not affect the test results.

Effect of Urinary pH

The pH of an aliquoted negative urine pool was adjusted to a pH range of 5 to 9 in 1 pH unit increments and spiked with drugs at 50% below and 50% above cut-off levels. The spiked, pH-adjusted urine was tested with the Multi-Drug Rapid Test The results demonstrate that varying ranges of pH do not interfere with the performance of the test.

Cross-Reactivity

A study was conducted to determine the cross-reactivity of the test with compounds in either drug-free urine or drug positive urine containing above calibrator substances. The following compounds show no cross-reactivity when tested with the Multi-Drug Rapid Test at a concentration of 100 µg/mL.

Non Cross-Reacting Compounds

Acetophenetidin	Cortisone	Zomepirac	d-Pseudoephedrine
N-Acetylprocainamide	Creatinine	Ketoprofen	Quinidine
Acetylsalicylic acid	Deoxycorticosterone	Labetalol	Quinine
Aminopyrine	Dextromethorphan	Loperamide	Salicylic acid
Amoxicillin	Diclofenac	Meprobamate	Serotonin
Ampicillin	Diflunisal	Isoxsuprine	Sulfamethazine
I-Ascorbic acid	Digoxin	d,l-Propanolol	Sulindac
Apomorphine	Diphenhydramine	Nalidixic acid	Tetracycline
Aspartame	Ethyl-p-aminobenzoate	Naproxen	Tetrahydrocortisone,
Atropine	β-Estradiol	Niacinamide	3-acetate
Benzilic acid	Estrone-3-sulfate	Nifedipine	Tetrahydrocortisone
Benzoic acid	Erythromycin	Norethindrone	Tetrahydrozoline
Bilirubin	Fenoprofen	Noscapine	Thiamine
d,l-Brompheniramine	Furosemide	d,l-Octopamine	Thioridazine
Caffeine	Gentisic acid	Oxalic acid	d,l-Tyrosine
Cannabidiol	Hemoglobin	Oxolinic acid	Tolbutamide
Chloral hydrate	Hydralazine	Oxymetazoline	Triamterene
Chloramphenicol	Hydrochlorothiazide	Papaverine	Trifluoperazine
Chlorothiazide	Hydrocortisone	Penicillin-G	Trimethoprim
d,l-Chlorpheniramine	o-Hydroxyhippuric acid	Perphenazine	d,l-Tryptophan
Chlorpromazine	3-Hydroxytyramine	Phenelzine	Uric acid
Cholesterol	d,l-Isoproterenol	Prednisone	Verapamil
Clonidine			

ALCOHOL PERFORMANCE CHARACTERISTICS

The detection limit on the Urine Alcohol Rapid Test is from 0.02% to 0.30% for approximate

relative blood alcohol level. The cutoff level of the Urine Alcohol Rapid Test can vary based on local regulations and laws.

ALCOHOL ASSAY SPECIFICITY

The Urine Alcohol Rapid Test will react with methyl, ethyl and allyl alcohols.

ALCOHOL INTERFERING SUBSTANCES

The following substances may interfere with the Urine Alcohol Rapid Test when using samples other than urine. The named substances do not normally appear in sufficient quantity in urine to interfere with the test.

- A. Agents which enhance color development
 - Peroxidases
 - Strong oxidizers
- B. Agents which inhibit color development
 - Reducing agents: Ascorbic acid, Tannic acid, Pyrogallol, Mercaptans and tosylates, Oxalic acid, Uric Acid
 - Bilirubin
 - L-dopa
 - L-methyl dopa
 - Methampyrone

BIBLIOGRAPHY

- B. Cody, J.T., "Specimen Adulteration in drug urinalysis. Forensic Sci. Rev., 1990, 2:63.
- C. Tsai, S.C. et. al., J. Anal. Toxicol. 1998; 22 (6): 474
- Tietz NW. Textbook of Clinical Chemistry. W.B. Saunders Company. 1986; 1735.
- Hawks RL, CN Chiang. *Urine Testing for Drugs of Abuse*. National Institute for Drug Abuse (NIDA), Research Monograph 73, 1986.
- Baselt RC. *Disposition of Toxic Drugs and Chemicals in Man*. 6th Ed. Biomedical Publ., Foster City, CA 2002.

Index of Symbols				
	Consult Instructions For Use		Contains sufficient for <n> tests	EC REP Authorized Representative in the EU
	<i>In vitro</i> diagnostic medical device		Use-by date	Do not re-use
	Temperature limit 2-30 °C	LOT	Batch code	REF Catalogue number
	Do not use if package is damaged		Manufacturer	CE CE CE mark

Hangzhou AllTest Biotech Co.,Ltd.
#550,Yinhai Street,
Hangzhou Economic & Technological Development Area,
Hangzhou, 310018 P.R. China
Web: www.alltests.com.cn Email: info@alltests.com.cn

EC

REP

MedNet EC-REP GmbH
Borkstrasse 10,
48163 Muenster,
Germany

Number: 14601688503
Revision date: 2025-01-24



Test rapido per tazza multi-farmaco a 1 fase con/senza adulterazione (Urina)

Foglio illustrativo

REF			
DOA-R127-A1	DOA-R137-A1	DOA-R147-A1	DOA-R157-A1
DOA-R167-A1	DOA-R177-A1	DOA-R187-A1	DOA-R197-A1
DOA-R1107-A1	DOA-R1117-A1	DOA-R1127-A1	DOA-R1137-A1
DOA-R1147-A1	DOA-R1157-A1	DOA-R1167-A1	DOA-R1177-A1
DOA-R1187-A1			

DUA-R127-A1	DUA-R137-A1	DUA-R147-A1	DUA-R157-A1
DUA-R167-A1	DUA-R177-A1	DUA-R187-A1	DUA-R197-A1
DUA-R1107-A1	DUA-R1117-A1	DUA-R1127-A1	DUA-R1137-A1
DUA-R1147-A1	DUA-R1157-A1	DUA-R1167-A1	DUA-R1177-A1

Foglio di istruzioni per l'analisi di qualsiasi combinazione delle seguenti sostanze:

ACE/AMP/IBAR/BZO/BUP/COC/THC/MTD/MET/MDMA/MOP/MQL/OPI/PCP/PPX/TCA/TML/KET/OXY/COT/EDDP/FYL/K2/6-MAM/MDA/ETG/CLO/LS/MPD/ZOL/DIA/ZOP/MCAT/7-ACL/ICFY/ICAF/CAT/TRO/MDPV/MEP/ALP/ABP/K3/α-PVP/CNB/MPRD/PGB/TZD/UR-144(K4)/ZAL/MES/GAB/TL/DQTP/PAP/KRA/CAR/FLX/CIT/FKET/OZP/RPD/TAP/INN/SCOP/MTZ/ALC

Include test di validità dei campioni (Specimen Validity Test, SVT) per:

Ossidanti/PCC, peso specifico, pH, nitriti, glutaraldeide, creatinina e candeggina

Test rapido per la rilevazione simultanea e qualitativa di diverse sostanze e loro metaboliti presenti nelle urine umane. Per gli operatori sanitari, inclusi i professionisti che eseguono test POC. Immunodosaggio solo per uso diagnostico in vitro.

【USO PREVISTO E RIEPILOGO】

Il test rapido per tazza multi-farmaco è un dosaggio immunologico cromatografico rapido per la rilevazione qualitativa di più farmaci e metaboliti di farmaci nelle urine alle seguenti concentrazioni di cut-off che può essere eseguita con l'uso del Cup Reader.

Test	Calibratore	Cut-off (ng/mL)
Acetaminofene (ACE)	Acetaminofene	5.000
Anfetamina (AMP)	d-Anfetamina	1.000/500/300
Barbiturici (BAR)	Secobarbital	300/200
Benzodiazepine (BZO)	Oxazepam	500/300/200/100
Buprenorfina (BUP)	Buprenorfina	10/5
Cocaina (COC)	Benzolicgonina	300/200/150/100
Marijuana (THC)	11-nor-Δ ⁹ -THC-9 COOH	300/200/150/50/30/25/20
Metadone (MTD)	Metadone	300/200
Metanfetamina (MET)	d-Metanfetamina	1.000/500/300
Metilendiossimetanfetamina (MDMA)	d,l-Metilendiossi-metanfetamina	1.000/500/300
Morfina (MOP/OPI)	Morfina	300/200/100
Metaqualone (MQL)	Metaqualone	300
Opiacei (OPI)	Morfina	2.000/1000
Fenciclidina (PCP)	Fenciclidina	50/25
Propossifene (PPX)	Propossifene	300
Antidepressivi triciclici (TCA)	Nortriptilina	1.000/500/300
Tramadolo (TML)	Cis-Tramadolo	500/300/200/100
Chetamina (KET)	Chetamina	1.000/500/300/100
Ossicodone (OXY)	Ossicodone	300/100
Cotina (COT)	Cotina	500/300/200/100/50/10
2-etilidene-1,5-dimetil-3,3-difenilpirrolidina (EDDP)	2-etilidene-1,5-dimetil-3,3-difenilpirrolidina	300/100
Fentanil (FYL)	Fentanil	20/10/100/200/300
Marijuana sintetica (K2)	JWH-018, JWH-073	50/30/25
6-mono-aceto-morfina (6-MAM)	6-mono-aceto-morfina	10
(±) 3,4-Metilendiossi-anfetamina (MDA)	(±) 3,4-Metilendiossi-Anfetamina	500
Etil-β-D-Glucuronide (ETG)	Etil-β-D-glucuronide	1.000/500/300
Clonazepam (CLO)	Clonazepam	400/150
Dietilammide dell'acido lisergico (LSD)	Dietilammide dell'acido lisergico	50/20/10
Metilfenidato (MPD)	Metilfenidato	300/150
Metilfenidato (MPD)	Acido Ritalinico	1.000
Zolpidem (ZOL)	Zolpidem	50
Diazepam (DIA)	Diazepam	300/200
Zopiclone (ZOP)	Zopiclone	50
Metcatinone (MCAT)	S(-)-Metcatinone	500
7-Amminoclonazepam (7-ACL)	7-Amminoclonazepam	300/200/100

Carfentanil (CFYL)	Carfentanil	500/250
Caffeina (CAF)	Caffeina	1.000
Catina (CAT)	(+)-Norpseudoefedrina	150
Tropicamide (TRO)	Tropicamide	350
3, 4-metilendiossirovalerone (MDPV)	3, 4-metilenediossi pirovalerone	1.000/500/300
Mefedrone (MEP)	Mefedrone	100/500
Alprazolam (ALP)	Alprazolam	100
AB-PINACA (ABP/K3)	AB-PINACA	10
α-Pirolidinovalerofenone (α-PVP)	α-Pirolidinovalerofenone	2.000/1.000/500/300
Cannabinolo (CNB)	cannabinolo	500
Meperidina (MPRD)	Meperidina	100
Pregabalin (PGB)	Pregabalin	50.000/500
Trazodone (TZD)	Trazodone	200
UR-144(K4)	Acido 5-pentanoico UR-144	25
Zaleplon (ZAL)	Zaleplon	100
Mescalina (MES)	Mescalina	100/300
Gabapentin (GAB)	Gabapentin	2.000
Tilidina (TLD)	Nortilidina	50
Quetiapina (QTP)	Quetiapina	1.000
Papaverina (PAP)	Papaverina	500
Kratom (KRA)	Mitraginina	300
Carisoprodol (CAR)	Carisoprodol	2.000/1.000
Fluoxetina (FLX)	Fluoxetina	500
Citalopram (CIT)	Citalopram	500
Fluochetamina (FKET)	Fluochetamina	1.000
Olanzapina (OZP)	Olanzapina	1.000
Risperidone (RPD)	Risperidone	150
Tapentadol (TAP)	Tapentadol	1.000
N,N-dimetiltriptamina (NND)	N,N-dimetiltriptamina	1.000
Scopolamina (SCOP)	Scopolamina	500
Mirtazapina (MTZ)	Desmetilmirtazapina	500
Test	Calibratore	Livello limite
Alcol (ALC)	Alcol	0,02%

Le configurazioni del test rapido per tazza multi-farmaco sono disponibili con qualsiasi combinazione di analiti delle sostanze sopra elencati con o senza SVT. Questo saggio fornisce solo un risultato analitico preliminare. Per ottenere un risultato analitico confermato, è necessario ricorrere a un metodo chimico alternativo più specifico. Nel dettaglio, la gascromatografia-spettrometria di massa (GC-MS) è il metodo di conferma preferito. È opportuno applicare la propria considerazione clinica e il giudizio professionale a qualsiasi risultato del test sulle sostanze d'abuso (drugs of abuse, DOA), in particolare quando sono indicati risultati preliminari positivi.

【SINTESI DELL'ADULTERAZIONE】

L'adulterazione è indicabile come la manomissione di un campione di urina con l'intenzione di alterare i risultati del test. L'uso di adulteranti può causare risultati falsi negativi nei test sulle sostanze, interferendo con il test di screening e/o distruggendo le sostanze presenti nelle urine. Può inoltre essere impiegata diluizione nel tentativo di produrre risultati falsi negativi del test.

Uno dei modi migliori per verificare la presenza di adulterazione o diluizione è stabilire alcune caratteristiche urinarie quali pH, peso specifico e creatinina, nonché rilevare la presenza di ossidanti/PCC, nitriti o glutaraldeide nelle urine.

【PRINCIPIO (PER I TEST SULLE DOA ESCLUSO L'ALCOL)】

Durante l'analisi, il campione di urina migra verso l'alto per azione capillare. La sostanza, se presente nel campione di urina al di sotto della sua concentrazione di cut-off, non saturerà i siti di legame dell'anticorpo specifico corrispondente. Quest'ultimo reagirà quindi con il coniugato sostanza-proteina e una linea colorata risulterà visibile nella regione di test della striscia reattiva della specifica sostanza. La presenza di sostanza al di sopra della concentrazione di cut-off saturerà tutti i siti di legame dell'anticorpo. Pertanto, nella regione di test non si formerà la linea colorata.

Un campione di urina positivo alla sostanza non genererà una linea colorata nella regione di test specifica della striscia reattiva in ragione del legame competitivo della sostanza; di contro, un campione di urina negativo alla sostanza genererà una linea nella regione di test in virtù dell'assenza del legame competitivo della sostanza.

Ai fini del controllo procedurale, una linea colorata apparirà sempre nella regione di controllo a indicare che è stato aggiunto un volume adeguato di campione e che si è verificata la penetrazione per capillarità nella membrana.

【PRINCIPIO DI ADULTERAZIONE】

Test per **ossidanti/PCC (piridinio clorocromato)** per l'individuazione della presenza di agenti ossidanti quali candeggina e perossido di idrogeno. Il piridinio clorocromato (venduto con il marchio Urine Luck) è un adulterante di comune impiego. ² La normale urina umana non deve contenere ossidanti di PCC.

Test del **peso specifico** per individuare la diluizione del campione. L'intervallo normale è compreso tra 1,003 e 1,030. I valori non compresi in questo intervallo possono essere una conseguenza della diluizione o dell'adulterazione del campione.

Test del **pH** per rilevare la presenza di adulteranti acidi o alcalini nelle urine. I livelli di pH normali devono essere compresi tra 4,0 e 9,0. I valori al di fuori di questo intervallo potrebbero indicare che il campione è stato alterato.

Test dei **nitriti** per rilevare la presenza di adulteranti commerciali di uso comune, quali Klear e Whizzies. Tali test agiscono ossidando il principale metabolita dei cannabinoidi, ossia THC-COOH.³ L'urina normale non deve contenere tracce di nitriti. I risultati positivi indicano generalmente la presenza di un adulterante.

Test della **glutaraldeide** per la rilevazione della presenza di un'aldeide. Gli adulteranti quali Urin Aid e Clear Choice contengono glutaraldeide che può causare risultati falsi negativi, rompendo l'enzima utilizzato in alcuni immunodosaggi.³ La glutaraldeide non è normalmente presente nelle urine; pertanto, la sua rilevazione in un campione di urina è generalmente un indicatore di adulterazione.

La **creatinina** è un prodotto di scarto della creatina, ossia un amminoacido contenuto nel tessuto muscolare e presente nelle urine. ¹ Una persona può tentare di eludere un test bevendo quantità eccessive di acqua o diuretici, come le tisane, per "pulire" l'organismo. Creatinina e peso specifico costituiscono due modi per verificare la presenza di diluizione e tentativi di aumentare lo smaltimento mediante escrezione urinaria, ossia i meccanismi più comuni impiegati nel tentativo di aggirare i test volti a individuare la presenza di sostanze. Bassi livelli di creatinina e di peso specifico possono indicare urina diluita. L'assenza di creatinina (<5 mg/dL) è indicativa di un campione non coerente con l'urina umana.

I test di rilevamento della candeggina si riferiscono a una serie di sostanze chimiche che rimuovono il colore, sbiancano o disinfettano, di sovente mediante ossidazione. Le candeggine vengono utilizzate come prodotti chimici per uso domestico per sbiancare gli indumenti e rimuovere le macchie e come disinfettanti. La normale urina umana non deve contenere candeggina.

【PRINCIPIO (PER L'ALCOL)】

Il test rapido per l'individuazione di alcol nelle urine consta di una striscia di plastica con un tampone di reazione applicato alla punta. A contatto con l'alcol, il tampone di reazione cambia colore a seconda della concentrazione di alcol presente. Tale fenomeno si basa sull'elevata specificità dell'alcol ossidasi per l'alcol etilico in presenza di perossidasi e substrato enzimatico come la TMB.

【REAGENTI (PER I TEST SULLE DOA ESCLUSO L'ALCOL)】

Ogni linea di test contiene un anticorpo monoclonale murino anti-sostanza e i corrispondenti coniugati sostanza-proteina. La linea di controllo contiene anticorpi policlonali IgG anti-coniglio di capra e IgG di coniglio.

【REAGENTI (PER L'ALCOL)】

Tetrametilbenzidina/Alcol ossidasi/Perossidasi

【REAGENTI SVT】

Tampone di adulterazione	Indicatore reattivo	Tamponi e ingredienti non reattivi
Creatinina	0,04%	99,96%
Nitriti	0,07%	99,93%
Candeggina	0,39%	99,61%
Glutaraldeide	0,02%	99,98%
pH	0,06%	99,94%
Peso specifico	0,25%	99,75%
Ossidanti/PCC	0,36%	99,64%

【PRECAUZIONI】

- Per gli operatori sanitari, inclusi i professionisti che eseguono test POC.
- Immunodosaggio solo per uso diagnostico *in vitro*. Il test deve rimanere nella confezione sigillata fino all'uso.
- Tutti i campioni devono essere considerati potenzialmente pericolosi e devono essere maneggiati allo stesso modo di un agente infettivo.
- Il test usato deve essere smaltito in conformità alle normative locali.
- Da utilizzare esclusivamente con il lettore di tazze. Non interpretare i risultati del test visivamente.

【CONSERVAZIONE E STABILITÀ】

Conservare nella confezione sigillata a una temperatura compresa tra 2 e 30 °C. Il test è stabile fino alla data di scadenza stampata sulla confezione sigillata. I contenitori per test devono rimanere nella confezione sigillata fino all'uso. **NON CONGELARE.** Non utilizzare oltre la data di scadenza.

【RACCOLTA E PREPARAZIONE DEI CAMPIONI】

Analisi delle urine

Il campione di urina deve essere raccolto in un contenitore pulito e asciutto. È possibile impiegare le urine raccolte in qualsiasi momento della giornata. I campioni di urina che presentano precipitati visibili devono essere centrifugati, filtrati o lasciati sedimentare per ottenere un campione limpido per l'analisi.

Conservazione dei campioni

I campioni di urina possono essere conservati a 2-8 °C per un massimo di 48 ore prima di procedere all'analisi. In caso di conservazione prolungata, i campioni possono essere congelati e conservati a temperature inferiori a -20 °C. I campioni congelati devono essere scongelati e miscelati bene prima dell'analisi. Quando si utilizzano schede di analisi con SVT o alcol, la conservazione dei campioni di urina non deve superare le 2 ore a temperatura ambiente o le 4 ore se i campioni vengono refrigerati prima dell'analisi.

【MATERIALI】

Materiali forniti

- Tazze di test
- Foglio illustrativo
- Scheda di procedura
- Timer
- Materiali necessari ma non forniti
- Cup reader

- 【MODO D'USO】**
Lasciare che il test, il campione di urina e/o i controlli raggiungano la temperatura ambiente (15-30 °C) prima di eseguire il test.
1. Portare il sacchetto a temperatura ambiente prima di aprirlo. Estrarre il tazza dalla busta sigillata e utilizzarlo entro un'ora.
 2. Il donatore fornisce il campione.
 3. Il tecnico sostituisce e fissa il tappo ponendo il tazza su una superficie piana.
 4. Controllare l'etichetta della temperatura (etichetta Temp) fino a 4 minuti dopo il prelievo del campione. Viene visualizzato un colore verde per indicare la temperatura del campione di urina. L'intervallo corretto per un campione non adulterato è 32-38 °C (90-100 °F).
 5. Il tecnico appone la data e la sigla sul sigillo di sicurezza e lo applica sul tappo del tazza.
 6. Il tecnico rimuove l'etichetta sulla tazza del test multi-farmaco per leggere i risultati.
 7. **Inserire la tazza nella camera di rilevamento del Cup reader** e chiudere il coperchio della camera per leggere i risultati dopo 5 minuti. **Non leggere visivamente i risultati del test.**
Per le linee guida sui campioni adulterati, fare riferimento alla propria Politica contro l'uso di sostanze stupefacenti. Si consiglia di non interpretare i risultati dei test antidroga e di ripetere il test delle urine o di raccogliere un altro campione in caso di risultati positivi per un test di adulterazione.


Nota: Per l'installazione, l'avvio, la calibrazione del sistema e le operazioni di test complete del Cup reader, consultare attentamente il Manuale d'uso del Cup reader. L'operatore deve consultare il Manuale d'uso del Cup reader prima dell'uso e acquisire familiarità con le operazioni e le procedure di controllo qualità.

PROCEDURA DEL TEST Tazza



1

- Aprire il coperchio e aggiungere il campione

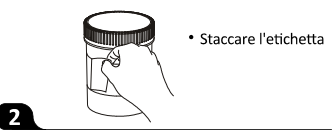

2

- Staccare l'etichetta

3

- Leggere i risultati con il lettore dopo 5 min.

【INTERPRETAZIONE DEI RISULTATI DOA】

● Risultati letti da Cup Reader

Il risultato positivo o negativo per ciascun analita è determinato dal Cup Reader.

Esempio di risultati:

Analita	Monitor	Risultato
MOP	POS	Positivo
OXY	NEG	Negativo
AMP	INV	Non valido

NEGATIVO: significa che le concentrazioni nel campione di urina sono inferiori ai livelli di cut-off designati per un particolare farmaco testato.

POSITIVO: significa che la concentrazione di farmaco nel campione di urina è maggiore del cut-off designato per un farmaco specifico.

NON VALIDO: volume campione insufficiente o tecniche procedurali errate sono le cause più probabili della mancata visualizzazione della linea di controllo. Leggere nuovamente le istruzioni e ripetere il test con un nuovo test. Se il risultato non è ancora valido, contattare il produttore.

【INTERPRETAZIONE DEI RISULTATI (S.V.T./ADULTERAZIONE E ALCOL)】

Esempio di risultati:

Analita	Monitor	Risultato
ALC	NOR	Normale
OXI/SG	ABN/NOR	Anomalo/Normale
NIT/GLU	NOR/NOR	Normale/Normale
CRE/pH	ABN/ABN	Anomalo/Anomalo

【CONTROLLO DI QUALITÀ】

Nel test è incluso un controllo procedurale. La linea che appare nella regione di controllo (C) è considerata un controllo procedurale interno che conferma un volume sufficiente del campione, un'adeguata penetrazione per capillarità nella membrana e una corretta tecnica procedurale. Gli standard di controllo non sono forniti in dotazione con questo kit. Tuttavia, si raccomanda di testare i controlli positivi e quelli negativi come buona pratica di laboratorio per comprovare la procedura di test e verificare le corrette prestazioni del test stesso.

【LIMITAZIONI】

1. Il Test rapido per tazza multi-farmaco fornisce solo un risultato analitico preliminare qualitativo. Per ottenere un risultato confermato, si deve utilizzare un metodo analitico secondario. Nel dettaglio, la gascromatografia-spettrometria di massa (GC-MS) è il metodo di conferma preferito.^{4,5}
2. Sussiste l'eventualità che errori tecnici o procedurali nonché che sostanze interferenti nel campione di urina possano causare risultati errati.
3. Gli adulteranti, quali candeggina e/o allume, nei campioni di urina possono produrre risultati

errati indipendentemente dal metodo analitico utilizzato. Se si sospetta adulterazione, il test deve essere ripetuto con un altro campione di urina.

4. Un risultato positivo non indica il livello o l'intossicazione, la via di somministrazione o la concentrazione nelle urine.

5. Un risultato negativo potrebbe non indicare necessariamente urina priva di sostanze. Risultati negativi possono essere ottenuti quando la sostanza è presente, ma al di sotto del livello di cut-off del test.

6. Il test non distingue tra sostanze d'abuso e alcuni farmaci.

7. Un risultato positivo del test potrebbe essere ottenuto da alcuni alimenti o integratori alimentari.

【LIMITI DI SVT/ADULTERAZIONE】

1. I test di rilevazione dell'adulterazione inclusi con il prodotto fungono da ausilio nella determinazione dei campioni anomali. Sebbene esaustivi, questi test non sono intesi come una rappresentazione onnicomprensiva dei possibili adulteranti.
2. Ossidanti/PCC: la normale urina umana non deve contenere ossidanti/PCC. La presenza di alti livelli di antiossidanti (ad es., l'acido ascorbico) nel campione può dare luogo a risultati falsi negativi per il tampone per ossidanti/PCC.
3. Peso specifico: livelli elevati di proteine nelle urine possono causare valori di peso specifico anormalmente elevati.
4. Nitriti: il nitrito non è un componente normale dell'urina umana. Tuttavia, la sua presenza nelle urine può indicare infezioni del tratto urinario o infezioni batteriche. Livelli di nitriti >20 mg/dL possono produrre risultati falsi positivi della glutaraldeide.
5. Glutaraldeide: non è normalmente presente nelle urine. Tuttavia, alcune anomalie metaboliche, come la chetoacidosi (digiuno, diabete non controllato o diete ad alto contenuto proteico), possono interferire con i risultati del test.
6. Creatinina: i livelli normali di creatinina sono compresi tra 20 e 350 mg/dL. In condizioni rare, alcune malattie renali possono mostrare urina diluita.
7. Candeggina: La normale urina umana non deve contenere candeggina. La presenza di livelli elevati di candeggina nel campione può causare risultati falsi negativi per il tampone di candeggina.

【CARATTERISTICHE PRESTAZIONALI】

Accuratezza % di accordo con GC-MS										
	ACE 5.000	AMP 1.000	AMP 500	AMP 300	BAR 300	BAR 200	BZO 500	BZO 300	BZO 200	BUP 10
Accordo positivo	95,6%	97,1%	97,0%	96,7%	96,7%	95,0%	95,5%	96,0%	96,1%	97,1%
Accordo negativo	98,7%	98,7%	98,6%	98,9%	99,0%	99,0%	99,0%	98,4%	98,4%	99,0%
Risultati totali	97,6%	98,2%	98,1%	98,4%	98,5%	97,9%	98,0%	97,7%	97,8%	98,5%

	BUP 5	COC 300	COC 200	COC 150	COC 100	THC 300	THC 150	THC 50	THC 25	THC 20	THC 200
Accordo positivo	96,4%	95,1%	96,3	95,7%	97,2%	95,8%	96,0%	96,3%	96,5%	96,1%	95,5%
Accordo negativo	98,9%	98,9%	>98,9%	98,9%	98,9%	98,6%	99,0%	98,6%	99,0%	98,5%	99,2%
Risultati totali	98,3%	97,7%	98,3%	97,8%	98,5%	97,9%	97,9%	97,6%	98,1%	97,8%	97,9%

	THC 30	MET 1.000	MET 500	MET 300	MDMA 1.000	MDMA 500	MDMA 300	MOP/ OPI 300	MOP/ OPI 100	MOP/ OPI 200	PPX 300
Accordo positivo	96,2%	95,3%	96,0%	95,8%	95,2%	95,7%	96,2%	95,0%	95,2%	95,1%	99,2%
Accordo negativo	98,6%	99,2%	99,2%	98,6%	99,1%	99,3%	98,6%	99,0%	99,1%	98,9%	98,5%
Risultati totali	98,0%	97,8%	97,9%	97,6%	97,7%	98,1%	98,0%	97,6%	98,0%	97,7%	98,8%

	TCA 1.000	TCA 500	TCA 300	TML 100	TML 200	TML 300	TML 500	KET 1.000	KET 500	KET 300	KET 100
Accordo positivo	98,9%	97,6%	96,4%	96,9%	96,7%	96,9%	96,8%	99,1%	96,8%	98,9%	99,1%
Accordo negativo	98,7%	98,8%	99,4%	99,3%	99,4%	99,4%	99,4%	>99,9%	98,7%	98,1%	99,3%
Risultati totali	98,8%	98,4%	98,4%	98,4%	98,4%	98,0%	98,4%	99,6%	98,0%	98,4%	99,2%

	OXY 100	OXY 300	COT 500	COT 200	COT 100	COT 50	COT 10	COT 300	ETG 500	ETG 1.000	ETG 300
Accordo positivo	97,5%	96,5%	96,5%	99,1%	98,2%	96,0%	95,0%	95,0%	95,3%	98,3%	97,4%
Accordo negativo	99,4%	99,4%	99,4%	98,5%	98,5%	99,3%	99,3%	98,8%	99,3%	98,5%	99,4%
Risultati totali	98,8%	98,4%	98,4%	99,0%	98,4%	98,0%	98,1%	97,6%	97,6%	98,4%	98,8%

	K2 50	K2 30	6-MAM 10	MDA 500	EDDP 300	EDDP 100	CLO 400	CLO 150	LSD 10	LSD 20	LSD 50
--	----------	----------	-------------	------------	-------------	-------------	------------	------------	-----------	-----------	-----------

Accordo positivo	98,4%	95,5%	98,2%	96,2%	99,2%	98,1%	96,1%	99,2%	95,2%	97,9%	98,9%
Accordo negativo	99,2%	99,2%	99,3%	99,4%	99,2%	99,3%	98,6%	98,4%	98,8%	98,1%	99,4%
Risultati totali	98,8%	97,6%	98,8%	98,4%	99,2%	98,8%	97,6%	98,8%	97,6%	98,0%	99,2%

	MEP 100	MEP 500	ZOL 50	DIA 300	DIA 200	ZOP 50	MCAT 500	7-ACL 300	7-ACL 200	7-ACL 100	CFYL 500
Accordo positivo	97,7%	97,5%	98,1%	96,7%	97,0%	98,6%	95,5%	98,0%	95,2%	96,3%	97,8%
Accordo negativo	98,8%	98,8%	99,3%	99,9%	99,3%	98,9%	99,5%	99,3%	98,8%	98,8%	99,4%
Risultati totali	98,4%	98,4%	98,8%	98,8%	98,4%	98,8%	98,0%	98,8%	97,6%	98,0%	98,8%

	CAF 1.000	CAT 150	TRO 350	MDPV 1000	MDPV 500	α-PVP 300	α-PVP 2.000	α-PVP 500	α-PVP 1.000	CNB 500	MPRD 100
Accordo positivo	95,2%	98,8%	96,8%	97,7%	96,9%	98,5%	95,7%	98,6%	96,0%	96,9%	95,9%
Accordo negativo	99,5%	99,4%	99,4%	99,4%	99,5%	98,9%	99,4%	98,9%	98,7%	99,3%	99,3%
Risultati totali	98,4%	99,2%	98,4%	98,8%	98,8%	98,8%	98,0%	98,8%	97,6%	98,4%	98,0%

	PGB 50.000	TZD 200	JR-144 /K4 25	ZAL 100	MES 100	GAB 2.000	MQL 300	ALP 100	ABP /K3 10	TLD 50	QTP 1.000
Accordo positivo	95,7%	97,2%	96,9%	97,0%	97,8%	97,8%	95,7%	97,5%	96,1%	97,3%	96,2%
Accordo negativo	98,7%	98,9%	99,5%	99,5%	99,5%	99,8%	99,1%	98,8%	99,4%	98,9%	98,8%
Risultati totali	97,6%	98,4%	98,8%	98,8%	98,8%	98,4%	98,1%	98,4%	98,4%	98,4%	98,0%

	PAP 500	KRA 300	TAP 1.000	FLX 500	K2 25	CIT 500	FKET 1.000	RPD 150	OPI 2.000	OPI 1.000	CFYL 250
Accordo positivo	96,4%	98,6%	95,2%	96,5%	99,1%	96,8%	97,2%	96,4%	95,1%	97,6%	96,3%
Accordo negativo	99,2%	99,3%	99,1%	99,3%	98,6%	99,3%	99,2%	99,1%	99,0%	99,4%	99,4%
Risultati totali	98,1%	99,0%	97,5%	98,5%	98,8%	98,5%	98,5%	98,0%	97,9%	98,8%	98,4%

	PGB 500	MES 300	OZP 1.000	MDPV 300	CAR 2.000	CAR 1.000	NND 1.000	SCOP 500	MTZ 500	MTD 300	MTD 200
Accordo positivo	97,4%	97,8%	96,2%	97,7%	96,8%	97,8%	97,0%	96,4%	96,8%	95,7%	96,4%
Accordo negativo	98,8%	99,4%	99,2%	99,4%	99,3%	98,8%	99,3%	98,8%	99,3%	98,9%	99,2%
Risultati totali	97,6%	98,8%	98,0%	98,8%	98,5%	98,4%	98,5%	98,0%	98,5%	98,3%	98,8%

	PCP 25	PCP 50	FYL 20	FYL 10	FYL 100	FYL 200	FYL 300	MPD 300	MPD 150	MPD 1.000
Accordo positivo	97,9%	97,4%	98,0%	98,1%	97,4%	97,9%	96,9%	98,0%	98,6%	96,8%
Accordo negativo	98,1%	99,4%	98,8%	98,6%	99,4%	98,1%	98,7%	98,0%	98,9%	99,3%
Risultati totali	98,0%	98,8%	98,6%	98,4%	98,8%	98,0%	98,0%	98,0%	98,8%	98,5%

Precisione

È stato condotto uno studio in tre ospedali utilizzando tre diversi lotti di prodotto per dimostrare la precisione all'interno della corsa, tra le corse e tra gli operatori. Una scheda identica di campioni codificati, contenente farmaci a concentrazioni di livello di cut-off negativo, -50% e 300%, è stata etichettata, testata in cieco in ciascun sito. I risultati hanno evidenziato un'accuratezza del 100% per i campioni con livello di cut-off negativo, -50% e 300%.

Sensibilità analitica

Un pool di urine in cui non sono presenti sostanze è stato addizionato con sostanze alle concentrazioni elencate. I risultati sono riassunti di seguito.

Intervallo di cut-off della concentrazione di sostanze	ACE 5.000	AMP 1.000	AMP 500	AMP 300	BAR 300	BAR 200	BZO 500	BZO 300
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	BZO 200	BZO 100	BUP 10	BUP 5	COC 300	COC 200	COC 150	COC 100
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	THC 300	THC 200	THC 150	THC 50	THC 30	THC 25	THC 20	LSD 10
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	FYL 100	FYL 200	FYL 300	FYL 20	FYL 10	OPI 2.000	OPI 1000	PPX 300
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	MOP/ OPI 300	MOP/ OPI 100	RPD 150	KET 1.000	KET 500	KET 300	KET 100	MQL 300
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	OXY 100	MDMA 1.000	MDMA 500	EDDP 300	EDDP 100	MPD 300	MPD 150	K2 50
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	K2 30		6-MAM 10		MDA 500		ETG 500		ETG 1,000		CLO 400		CLO 150	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	LSD 20		LSD 50		MDPV 300		ZOL 50		MDMA 300		OXY 300		DIA 300	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	29	1	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	DIA 200		ZOP 50		MCAT 500		7-ACL 300		7-ACL 200		7-ACL 100		CFYL 500	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	29	1	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	CAF 1.000		CAT 150		TRO 350		MDPV 1.000		MEP 100		MEP 500		ALP 100	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	COT 500		COT 300		COT 200		COT 100		COT 50		COT 10		MPD 1.000	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	MDPV 500		ABP/K3 10		TAP 1.000		CNB 500		MPRD 100		PGB 50.000		TZD 200	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0

Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	UR-144/ K4 25		ZAL 100		MES 100		GAB 2.000		MOP/OPI 200		ETG 300		K2 25	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	TLD 50		QTP 1.000		PAP 500		KRA 300		FLX 500		CAR 2000		CAR 1.000	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	PGB 500		MES 300		OZP 1.000		CIT 500		FKET 1.000		MTD 300		MTD 200	
	-	+	-	+	-	+	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	CFYL 250	α-PVP 2.000	α-PVP 500	α-PVP 300	α-PVP 1.000	NND 1.000	SCOP 500	MTZ 500
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	MET 1.000	MET 500	MET 300	PCP 25	PCP 50	TCA 1.000	TCA 500	TCA 300
	-	+	-	+	-	+	-	+
Cut-off 0%	30	0	30	0	30	0	30	0
Cut-off -50%	30	0	30	0	30	0	30	0
Cut-off +300%	0	30	0	30	0	30	0	30

Intervallo di cut-off della concentrazione di sostanze	TML 100	TML 200	TML 300	TML 500
	-	+	-	+
Cut-off 0%	30	0	30	0
Cut-off -50%	30	0	30	0
Cut-off +300%	0	30	0	30

La tabella seguente elenca le concentrazioni di composti (ng/mL) che, a 5 minuti, sono rilevate come positive nelle urine dal del Test rapido per tazza multi-farmaco.

Analiti	Concentrazione (ng/mL)	Analiti	Concentrazione (ng/mL)
ACETAMINOFENE (ACE)			
Acetaminofene	5.000		
ANFETAMINA (AMP 1.000)			
D,L-anfetamina solfato	300	Fentermina	1.000
L-anfetamina	25.000	Maprotilina	50.000
(±) 3,4-metilendiossi-anfetamina	500	Metossifenamina	6.000
		D-anfetamina	1.000
ANFETAMINA (AMP 500)			
D,L-anfetamina solfato	150	Fentermina	500
L-anfetamina	12.500	Maprotilina	25.000
(±) 3,4-metilendiossi-anfetamina	250	Metossifenamina	3.000
		D-anfetamina	500
ANFETAMINA (AMP 300)			
D,L-anfetamina solfato	75	Fentermina	300
L-anfetamina	10.000	Maprotilina	15.000
(±) 3,4-metilendiossi-anfetamina	150	Metossifenamina	2.000
		D-anfetamina	300
BARBITURICI (BAR 300)			
Amobarbital	5.000	Alphenol	600
5,5-difenilidantoina	8.000	Aprobarbital	500
Allobarbital	600	Butabarbital	200
Barbital	8.000	Butalbital	8.000
Talbutal	200	Butethal	500
Ciclopentobarbital	30.000	Fenobarbital	300
Pentobarbital	8.000	Secobarbital	300

BARBITURICI (BAR 200)			
Amobarbital	3.000	Alphenol	400
5,5-difenilidantoina	5.000	Aprobarbital	300
Allobarbital	400	Butabarbital	150
Barbital	5.000	Butalbital	5.000
Talbutal	150	Butethal	300
Ciclopentobarbital	20.000	Fenobarbital	200
Pentobarbital	5.000	Secobarbital	200

BENZODIAZEPINE (BZO 500)			
Alprazolam	200	Bromazepam	1.500
a-idrossialprazolam	2.500	Clordiazepossido	1.500
Clobazam	300	Nitrazepam	300
Clonazepam	800	Norclordiazepossido	200
Clorazepato di potassio	800	Nordiazepam	1.500
Delorazepam	1.500	Oxazepam	500
Desalchilflurazepam	300	Temazepam	300
Flunitrazepam	300	Diazepam	500
(±) Lorazepam	5.000	Estazolam	10.000
RS-Lorazepamglucuronide	300	Triazolam	5.000
Midazolam	10.000		

BENZODIAZEPINE (BZO 300)			
Alprazolam	100	Bromazepam	900
a-idrossialprazolam	1.500	Clordiazepossido	900
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norclordiazepossido	100
Clorazepato di potassio	500	Nordiazepam	900
Delorazepam	900	Oxazepam	300
Desalchilflurazepam	200	Temazepam	100
Flunitrazepam	200	Diazepam	300
(±) Lorazepam	3.000	Estazolam	6.000
RS-Lorazepamglucuronide	200	Triazolam	3.000
Midazolam	6.000		

BENZODIAZEPINE (BZO 200)			
Alprazolam	70	Bromazepam	600
a-idrossialprazolam	1.000	Clordiazepossido	600
Clobazam	120	Nitrazepam	120
Clonazepam	300	Norclordiazepossido	70
Clorazepato di potassio	300	Nordiazepam	600
Delorazepam	600	Oxazepam	200
Desalchilflurazepam	120	Temazepam	70
Flunitrazepam	120	Diazepam	200
(±) Lorazepam	2.000	Estazolam	4.000
RS-Lorazepamglucuronide	120	Triazolam	2.000
Midazolam	4.000		

BENZODIAZEPINE (BZO 100)			
Alprazolam	40	Bromazepam	300
a-idrossialprazolam	500	Clordiazepossido	300
Clobazam	60	Nitrazepam	60
Clonazepam	150	Norclordiazepossido	40
Clorazepato di potassio	150	Nordiazepam	300
Delorazepam	300	Oxazepam	100
Desalchilflurazepam	60	Temazepam	40
Flunitrazepam	60	Diazepam	100
(±) Lorazepam	1.000	Estazolam	2.000
RS-Lorazepamglucuronide	60	Triazolam	1.000
Midazolam	2.000		

BUPRENORFINA (BUP 10)			
Buprenorfinà	10	Norbuprenorfinà	50
Buprenorfinà 3-D-glucuronide	50	Norbuprenorfinà 3-D-glucuronide	100

BUPRENORFINA (BUP 5)			
Buprenorfinà	5	Norbuprenorfinà	25
Buprenorfinà 3-D-glucuronide	25	Norbuprenorfinà 3-D-glucuronide	50

COCAINA (COC 300)			
Benzoilecgonina	300	Cocaetilene	20.000
Cocaina HCl	200	Ecgonina	30.000

COCAINA (COC 200)			
Benzoilecgonina	200	Cocaetilene	13.500
Cocaina HCl	135	Ecgonina	20.000

COCAINA (COC 150)			
Benzoilecgonina	150	Cocaetilene	1.0000
Cocaina HCl	120	Ecgonina	15.000

COCAINA (COC 100)			
Benzoilecgonina	100	Cocaetilene	7.000
Cocaina HCl	80	Ecgonina	10.000

MARIJUANA (THC 300)			
Cannabinolo	200.000	Δ ⁹ -THC	100.000
11-nor-Δ ⁸ -THC-9 COOH	200	Δ ⁹ -THC	100.000
11-nor-Δ ⁹ -THC-9 COOH	300		
MARIJUANA (THC 200)			
Cannabinolo	140.000	Δ ⁹ -THC	68.000
11-nor-Δ ⁸ -THC-9 COOH	120	Δ ⁹ -THC	68.000
11-nor-Δ ⁹ -THC-9 COOH	200		
MARIJUANA (THC 150)			
Cannabinolo	100.000	Δ ⁸ -THC	50.000
11-nor-Δ ⁸ -THC-9 COOH	100	Δ ⁹ -THC	50.000
11-nor-Δ ⁹ -THC-9 COOH	150		
MARIJUANA (THC 50)			
Cannabinolo	35.000	Δ8-THC	17.000
11-nor-Δ ⁸ -THC-9 COOH	30	Δ9-THC	17.000
11-nor-Δ ⁹ -THC-9 COOH	50		
MARIJUANA (THC 30)			
Cannabinolo	20.000	Δ ⁸ -THC	10.000
11-nor-Δ ⁸ -THC-9 COOH	20	Δ ⁹ -THC	10.000
11-nor-Δ ⁹ -THC-9 COOH	30		
MARIJUANA (THC 25)			
Cannabinolo	17.500	Δ ⁸ -THC	8.500
11-nor-Δ ⁸ -THC-9 COOH	15	Δ ⁹ -THC	8.500
11-nor-Δ ⁹ -THC-9 COOH	25		
MARIJUANA (THC 20)			
Cannabinolo	14.000	Δ ⁸ -THC	6.800
11-nor-Δ ⁸ -THC-9 COOH	12	Δ ⁹ -THC	6.800
11-nor-Δ ⁹ -THC-9 COOH	20		
METADONE (MTD 300)			
Metadone	300	Doxilamina	100.000
METADONE (MTD 200)			
Metadone	200	Doxilamina	65.000
METAMFETAMINA (MET 1.000)			
ρ-idrossimetamfetamina	25.000	(±)-3,4-metilendiossi-metamfetamina	1.600
D-metamfetamina	1.000		
L-metamfetamina	20.000	Mefentermina	50.000
METAMFETAMINA (MET 500)			
ρ-idrossimetamfetamina	12.500	(±)-3,4-metilendiossi-metamfetamina	800
D-metamfetamina	500		
L-metamfetamina	10.000	Mefentermina	25.000
METAMFETAMINA (MET 300)			
ρ-idrossimetamfetamina	7.500	(±)-3,4-metilendiossi-metamfetamina	500
D-metamfetamina	300		
L-metamfetamina	6.000	Mefentermina	15.000
METILENDIOSSIMETAMFETAMINA (MDMA 1.000) Ecstasy			
(±) 3,4-metilendiossi-Metamfetamina HCl	1.000	3,4-metilendiossi-etilanfetamina	600
(±) 3,4-metilendiossi-anfetamina HCl	6.000		
METILENDIOSSIMETAMFETAMINA (MDMA 500) Ecstasy			
(±) 3,4-metilendiossi-Metamfetamina HCl	500	3,4-metilendiossi-etilanfetamina	300
(±) 3,4-metilendiossi-anfetamina HCl	3.000		
METILENDIOSSIMETAMFETAMINA (MDMA 300) Ecstasy			
(±) 3,4-metilendiossi-Metamfetamina HCl	300	3,4-metilendiossi-etilanfetamina	180
(±) 3,4-metilendiossi-anfetamina HCl	1.800		
MORFINA (MOP/OPI 300)			
Codeina	200	Norcodeina	6.000
Levorfanolo	1.500	Normorfone	50.000
Morfina-3-β-D-glucuronide	800	Ossicodone	30.000
Etilmorfina	6.000	Ossimorfone	50.000
Idrocodone	50.000	Procaina	15.000
Idromorfone	3.000	Tebaina	6.000
6-monoacetilmorfina	300	Morfina	300
MORFINA/OPPIACEI (MOP/OPI 200)			
Codeina	160	Norcodeina	4.000
Levorfanolo	1.000	Normorfone	40.000
Morfina-3-β-D-glucuronide	600	Ossicodone	20.000

Etilmorfina	4.000	Ossimorfone	40.000
Idrocodone	40.000	Procaina	10.000
Idromorfone	2.000	Tebaina	4.000
6-monoacetilmorfina	200	Morfina	200
MORFINA (MOP/OPI 100)			
Codeina	80	Norcodeina	2.000
Levorfanolo	500	Normorfone	20.000
Morfina-3-β-D-glucuronide	300	Ossicodone	10.000
Etilmorfina	2.000	Ossimorfone	20.000
Idrocodone	20.000	Procaina	5.000
Idromorfone	1.000	Tebaina	2.000
6-monoacetilmorfina	200	Morfina	100
METAQUALONE (MQL 300)			
Metaqualone	300		
MORFINA/OPPIACEI (OPI 2.000)			
Codeina	2.000	Morfina	2.000
Etilmorfina	3.000	Norcodeina	25.000
Idrocodone	50.000	Normorfone	50.000
Idromorfone	15.000	Ossicodone	25.000
Levorfanolo	25.000	Ossimorfone	25.000
6-monoacetilmorfina	3.000	Procaina	50.000
Morfina 3-β-D-glucuronide	2.000	Tebaina	25.000
MORFINA/OPPIACEI (OPI 1.000)			
Codeina	1.000	Morfina	1.000
Etilmorfina	1.500	Norcodeina	12.500
Idrocodone	25.000	Normorfone	25.000
Idromorfone	7.500	Ossicodone	12.500
Levorfanolo	12.500	Ossimorfone	12.500
6-monoacetilmorfina	1.500	Procaina	25.000
Morfina 3-β-D-glucuronide	1.000	Tebaina	12.500
FENCICLIDINA (PCP 25)			
Fenciclidina	25	4-idrossifenciclidina	12.500
FENCICLIDINA (PCP 50)			
Fenciclidina	50	4-idrossifenciclidina	25.000
PROPOSSIFENE (PPX 300)			
D-propossifene	300	D-norpropossifene	300
ANTIDEPRESSIVI TRICICLICI (TCA 1.000)			
Nortriptilina	1.000	Imipramina	400
Nordoxepina	500	Clomipramina	50.000
Trimipramina	3.000	Doxepina	2.000
Amitriptilina	1.500	Maprotilina	2.000
Promazina	3.000	Prometazina	50.000
Desipramina	200	Perfenazina	50.000
Ciclobenzaprina	2.000	Dithiaden	10.000
ANTIDEPRESSIVI TRICICLICI (TCA 500)			
Nortriptilina	500	Imipramina	200
Nordoxepina	250	Clomipramina	25.000
Trimipramina	1.500	Doxepina	1.000
Amitriptilina	750	Maprotilina	1.000
Promazina	1.500	Prometazina	25.000
Desipramina	100	Perfenazina	25.000
Ciclobenzaprina	1.000	Dithiaden	5.000
ANTIDEPRESSIVI TRICICLICI (TCA 300)			
Nortriptilina	300	Imipramina	120
Nordoxepina	150	Clomipramina	15.000
Trimipramina	900	Doxepina	600
Amitriptilina	450	Maprotilina	600
Promazina	900	Prometazina	15.000
Desipramina	60	Perfenazina	15.000
Ciclobenzaprina	600	Dithiaden	3.000
TRAMADOLO (TML 100)			
n-desmetil-cis-tramadolo	200	o-desmetil-cis-tramadolo	10.000
Cis-tramadolo	100	Fenciclidina	100.000
Prociclidina	100.000	d,l-o-desmetil venlafaxina	50.000
TRAMADOLO (TML 200)			
n-desmetil-cis-tramadolo	400	o-desmetil-cis-tramadolo	20.000
Cis-tramadolo	200	Fenciclidina	200.000
Prociclidina	200.000	d,l-o-desmetil venlafaxina	100.000
TRAMADOLO (TML 300)			
n-desmetil-cis-tramadolo	600	o-desmetil-cis-tramadolo	30.000
Cis-tramadolo	300	Fenciclidina	300.000
Prociclidina	300.000	d,l-o-desmetil venlafaxina	150.000
TRAMADOLO (TML 500)			
n-desmetil-cis-tramadolo	1.000	o-desmetil-cis-tramadolo	50.000
Cis-tramadolo	500	Fenciclidina	500.000
Prociclidina	500.000	d,l-o-desmetil venlafaxina	250.000

CHETAMINA (KET 1.000)			
Chetamina	1.000	Benzfetamina	25.000
Destrometorfano	2.000	(+) clorfeniramina	25.000
Metossifenamina	25.000	Clonidina	100.000
D-norpropossifene	25.000	EDDP	50.000
Promazina	25.000	4-idrossifenciclidina	50.000
Prometazina	25.000	Levorfanolo	50.000
Pentazocina	25.000	MDE	50.000
Fenciclidina	25.000	Meperidina	25.000
Tetraidrozolina	500	d-metamfetamina	50.000
Mefentermina	25.000	l-metamfetamina	50.000
(1R, 2S) - (-)-efedrina	100.000	3,4-metilendiossimetamfetamina (MDMA)	100.000
Disopiramide	25.000	Tioridazina	50.000
CHETAMINA (KET 500)			
Chetamina	500	Benzfetamina	12.500
Destrometorfano	1.000	(+) clorfeniramina	12.500
Metossifenamina	12.500	Clonidina	50.000
D-norpropossifene	12.500	EDDP	25.000
Promazina	12.500	4-idrossifenciclidina	25.000
Prometazina	12.500	Levorfanolo	25.000
Pentazocina	12.500	MDE	25.000
Fenciclidina	12.500	Meperidina	12.500
Tetraidrozolina	250	d-metamfetamina	25.000
Mefentermina	12.500	l-metamfetamina	25.000
(1R, 2S) - (-)-efedrina	50.000	3,4-metilendiossimetamfetamina (MDMA)	50.000
Disopiramide	12.500	Tioridazina	25.000
CHETAMINA (KET 300)			
Chetamina	300	Benzfetamina	6.250
Destrometorfano	600	(+) clorfeniramina	6.250
Metossifenamina	6.250	Clonidina	30.000
D-norpropossifene	6.250	EDDP	15.000
Promazina	6.250	4-idrossifenciclidina	15.000
Prometazina	6.250	Levorfanolo	15.000
Pentazocina	6.250	MDE	15.000
Fenciclidina	6.250	Meperidina	6.250
Tetraidrozolina	150	d-metamfetamina	15.000
Mefentermina	6.250	l-metamfetamina	15.000
(1R, 2S) - (-)-efedrina	30.000	3,4-metilendiossimetamfetamina (MDMA)	30.000
Disopiramide	6.250	Tioridazina	15.000
CHETAMINA (KET 100)			
Chetamina	100	Benzfetamina	2.000
Destrometorfano	200	(+) clorfeniramina	2.000
Metossifenamina	2.000	Clonidina	10.000
D-norpropossifene	2.000	EDDP	5.000
Promazina	2.000	4-idrossifenciclidina	5.000
Prometazina	2.000	Levorfanolo	5.000
Pentazocina	2.000	MDE	5.000
Fenciclidina	2.000	Meperidina	2.000
Tetraidrozolina	50	d-metamfetamina	5.000
Mefentermina	2.000	l-metamfetamina	5.000
(1R, 2S) - (-)-efedrina	10.000	Tioridazina	5.000
Disopiramide	2.000	3,4-metilendiossimetamfetamina (MDMA)	10.000
Ossicodone (OXY 100)			
Ossicodone	100	Idromorfone	50.000
Ossimorfone	300	Naloxone	25.000
Levorfanolo	50.000	Naltrexone	25.000
Idrocodone	25.000		
Ossicodone (OXY 300)			
Ossicodone	300	Idromorfone	150.000
Ossimorfone	900	Naloxone	75.000
Levorfanolo	150.000	Naltrexone	75.000
Idrocodone	75.000		
Cotina (COT 500)			
(-)-cotinina	500	(-)nicotina	12.500
Cotina (COT 300)			
(-)-cotinina	300	(-)nicotina	7.500
Cotina (COT 200)			
(-)-cotinina	200	(-)nicotina	5.000
Cotina (COT 100)			
(-)-cotinina	100	(-)nicotina	2.500
Cotina (COT 50)			
(-)-cotinina	50	(-)nicotina	1.250
Cotina (COT 10)			

(-) -cotinina	10	(-) -nicotina	250
2-etilidene-1,5-dimetil-3,3-difenilpirrolidina (EDDP 300)			
2-etilidene-1,5-dimetil-3,3-difenilpirrolidina (EDDP)			300
2-etilidene-1,5-dimetil-3,3-difenilpirrolidina (EDDP 100)			
2-etilidene-1,5-dimetil-3,3-difenilpirrolidina (EDDP)			100
FENTANYL (FYL 20)			
Fentanyl	20	Cyclopro Fentanyl	500
Norfentanyl	>100.000	(±)cis-3-Methylfentanyl	500
Butyl fentanyl	300	Valeryl Fentanyl	200
Methoxyacetyl-Fentanyl	40	Acetyl Fentanyl	40
Ocfentanil	200	para-Fluorobutiryl fentanil	200
4-Fluoro-isobutiryl Fentanyl	200	para-Fluorofentanil	100
FENTANYL (FYL 10)			
Fentanyl	10	Cyclopro Fentanyl	250
Norfentanyl	>100.000	(±)cis-3-Methylfentanyl	250
Butyl fentanyl	150	Valeryl Fentanyl	100
Methoxyacetyl-Fentanyl	20	Acetyl Fentanyl	20
Ocfentanil	100	para-Fluorobutiryl fentanil	100
4-Fluoro-isobutiryl Fentanyl	100	para-Fluorofentanil	50
FENTANYL (FYL 100)			
Fentanyl	100	Cyclopro Fentanyl	2.500
Norfentanyl	>100.000	(±)cis-3-Methylfentanyl	2.500
Butyl fentanyl	1.500	Valeryl Fentanyl	1.000
Methoxyacetyl-Fentanyl	200	Acetyl Fentanyl	200
Ocfentanil	1.000	para-Fluorobutiryl fentanil	1.000
4-Fluoro-isobutiryl Fentanyl	1.000	para-Fluorofentanil	500
FENTANYL (FYL 200)			
Fentanyl	200	Cyclopro Fentanyl	5.000
Norfentanyl	>100.000	(±)cis-3-Methylfentanyl	5.000
Butyl fentanyl	3.000	Valeryl Fentanyl	2.000
Methoxyacetyl-Fentanyl	400	Acetyl Fentanyl	400
Ocfentanil	2.000	para-Fluorobutiryl fentanil	2.000
4-Fluoro-isobutiryl Fentanyl	2.000	para-Fluorofentanil	1.000
FENTANYL (FYL 300)			
Fentanyl	300	Cyclopro Fentanyl	7.500
Norfentanyl	>100.000	(±)cis-3-Methylfentanyl	7.500
Butyl fentanyl	4.500	Valeryl Fentanyl	3.000
Methoxyacetyl-Fentanyl	600	Acetyl Fentanyl	600
Ocfentanil	3.000	para-Fluorobutiryl fentanil	3.000
4-Fluoro-isobutiryl Fentanyl	3.000	para-Fluorofentanil	1.500
Marijuana sintetica (K2-50)			
JWH-018 acido 5-pentanoico	50	JWH-073 acido 4-butanoico	50
JWH-018 4-idrossipentil	400	JWH-018 5-idrossipentil	500
JWH-073 4-idrossibutil	500		
Marijuana sintetica (K2-30)			
JWH-018 acido 5-pentanoico	30	JWH-073 acido 4-butanoico	30
JWH-018 4-idrossipentil	250	JWH-018 5-idrossipentil	300
JWH-073 4-idrossibutil	300		
Marijuana sintetica (K2-25)			
JWH-018 acido 5-pentanoico	25	JWH-073 acido 4-butanoico	25
JWH-018 4-idrossipentil	200	JWH-018 5-idrossipentil	250
JWH-073 4-idrossibutil	250		
6-mono-aceto-morfina (6-MAM 10)			
6-monoacetilmorfina	10	Morfina	100.000
(±) 3,4-metilenediossiamfetamina (MDA 500)			
(±) 3,4-metilendiossi-anfetamina	500	Metossifenamina	5.000
D,L-anfetamina solfato	400	D-anfetamina	2.000
L-anfetamina	30.000	Fentermina	2.000
		Maprotilina	100.000
Etil β-D-glucuronide (ETG 500)			
Etil β-D-glucuronide	500	Propil β-D-glucuronide	50.000
Acido glucuronico	100.000	Etanolo	>100.000
Metanolo	>100.000		
Etil β-D-glucuronide (ETG 1.000)			
Etil β-D-glucuronide	1.000	Propil β-D-glucuronide	100.000
Acido glucuronico	>100.000	Etanolo	>100.000
Metanolo	>100.000		
Etil β-D-glucuronide (ETG 300)			
Etil β-D-glucuronide	300	Propil β-D-glucuronide	30.000
Acido glucuronico	60.000	Etanolo	>100.000
Metanolo	>100.000		
CLONAZEPAM (CLO 400)			
Clonazepam	400	Flunitrazepam	300

Alprazolam	200	(±) Lorazepam	1.250
a-idrossialprazolam	2.000	RS-Lorazepamglucuronide	250
Bromazepam	1.000	Midazolam	5.000
Clordiazepossido	1.000	Nitrazepam	200
Clobazam	250	Norclordiazepossido	200
Clorazepato di potassio	600	Nordiazepam	1.000
Delorazepam	1.000	Oxazepam	350
Desalchilfurazepam	250	Temazepam	150
Diazepam	300	Triazolam	5.000
Estazolam	1.250		
CLONAZEPAM (CLO 150)			
Clonazepam	150	Flunitrazepam	120
Alprazolam	75	(±) Lorazepam	500
a-idrossialprazolam	750	RS-Lorazepamglucuronide	100
Bromazepam	400	Midazolam	2.000
Clordiazepossido	400	Nitrazepam	75
Clobazam	100	Norclordiazepossido	75
Clorazepato di potassio	250	Nordiazepam	400
Delorazepam	400	Oxazepam	130
Desalchilfurazepam	100	Temazepam	60
Diazepam	120	Triazolam	2.000
Estazolam	500		
DIETILAMMIDE DELL'ACIDO LISERGICO (LSD 10)			
Dietilamide dell'acido lisergico	10		
DIETILAMMIDE DELL'ACIDO LISERGICO (LSD 20)			
Dietilamide dell'acido lisergico	20		
DIETILAMMIDE DELL'ACIDO LISERGICO (LSD 50)			
Dietilamide dell'acido lisergico	50		
METILFENIDATO (MPD 1.000)			
Methylphenidate (Ritalin)	350	Ritalinic Acid	1.000
METILFENIDATO (MPD 300)			
Methylphenidate (Ritalin)	300	Ritalinic Acid	1.000
METILFENIDATO (MPD 150)			
Methylphenidate (Ritalin)	150	Ritalinic Acid	500
ZOLPIDEM (ZOL 50)			
Zolpidem	50		
Diazepam (DIA 300)			
Diazepam	300	Midazolam	6.000
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norclordiazepossido	100
Clorazepato dipotassio	500	Nordiazepam	900
Alprazolam	100	Flunitrazepam	200
a-idrossialprazolam	1.500	(±) Lorazepam	3.000
Bromazepam	900	RS-Lorazepam glucuronide	200
Clordiazepossido	900	Triazolam	3.000
Estazolam	6.000	Temazepam	100
Delorazepam	900	Oxazepam	300
Desalchilfurazepam	200		
Diazepam (DIA 200)			
Diazepam	200	Midazolam	4000
Clobazam	120	Nitrazepam	120
Clonazepam	300	Norclordiazepossido	70
Clorazepato dipotassio	300	Nordiazepam	600
Alprazolam	70	Flunitrazepam	120
a-idrossialprazolam	1000	(±) Lorazepam	2000
Bromazepam	600	RS-Lorazepam glucuronide	120
Clordiazepossido	600	Triazolam	2000
Estazolam	4000	Temazepam	70
Delorazepam	600	Oxazepam	200
Desalchilfurazepam	120		
Zopiclone (ZOP 50)			
Zopiclone-N-ossido	50	Zopiclone	50
Metcatinone (MCAT 500)			
S(-)-metcatinone HCl	500	R(+)-metcatinone HCl	1500
Metossifenamina	100000	3-fluorometcatinone HCl	1500
7-AMINOCLONAZEPAM (7-ACL 300)			
a-idrossialprazolam	6.000	Flunitrazepam	3.000
Bromazepam	6.000	RS-Lorazepam glucuronide	2.700
Clordiazepossido	6.000	Norclordiazepossido	4.500
Clobazam	9.000	Nordiazepam	15.000
Clonazepam	2.400	Temazepam	9.000
Delorazepam	6.000	7-Aminoclonazepam	300
Desalchilfurazepam	6.000		

7-AMINOCLONAZEPAM (7-ACL 200)			
a-idrossialprazolam	4.000	Flunitrazepam	2.000
Bromazepam	4.000	RS-Lorazepam glucuronide	1.800
Clordiazepossido	4.000	Norclordiazepossido	3.000
Clobazam	6.000	Nordiazepam	10.000
Clonazepam	1.600	Temazepam	6.000
Delorazepam	4.000	7-Aminoclonazepam	200
Desalchilfurazepam	4.000		
7-AMINOCLONAZEPAM (7-ACL 100)			
a-idrossialprazolam	2.000	Flunitrazepam	1.000
Bromazepam	2.000	RS-Lorazepam glucuronide	900
Clordiazepossido	2.000	Norclordiazepossido	1.500
Clobazam	3.000	Nordiazepam	5.000
Clonazepam	800	Temazepam	3.000
Delorazepam	2.000	7-Aminoclonazepam	100
Desalchilfurazepam	2.000		
CARFENTANIL (CFYL 500)			
Carfentanyl	500	Fentanyl	100
Sufentanil	50.000	Ramifentanil	10.000
(±)cis-3-Menthylfentanyl	20.000	Butyl fentanyl	150
CARFENTANIL (CFYL 250)			
Carfentanyl	250	Fentanyl	50
Sufentanil	25.000	Ramifentanil	5.000
(±)cis-3-Menthylfentanyl	10.000	Butyl fentanyl	75
CAFFEINA (CAF 1000)			
Caffeina	1000		
CATINA (CAT 150)			
(+)-norpseudoefedrina HCl (catina)	150	(+)-3,4-metilenediossiamfetamina (MDA)	100
d/l-anfetamina	100	p-idrossiamfetamina	100
Triptamina	12.500	Metossifenamina	12.500
TROPICAMIDE (TRO 350)			
Tropicamide	350		
3,4-METILENEDIOSSIPROVALERONE (MDPV 1.000)			
3,4-metilendiossi pirovalerone	1.000		
3,4-METILENEDIOSSIPROVALERONE (MDPV 500)			
3,4-metilendiossi pirovalerone	500		
3,4-METILENEDIOSSIPROVALERONE (MDPV 300)			
3,4-metilendiossi pirovalerone	300		
MEFEDRONE (MEP 100)			
Mefedrone HCl	100	R(+)-metcatinone HCl	1500
S(-)-metcatinone HCl	500	3-fluorometcatinone HCl	1500
4-fluorometcatinone HCl	300	Metossifenamina	100.000
MEFEDRONE (MEP 500)			
Mefedrone HCl	500	R(+)-metcatinone HCl	7.500
S(-)-metcatinone HCl	2.500	3-fluorometcatinone HCl	7.500
4-fluorometcatinone HCl	1.500	Metossifenamina	500.000
ALPRAZOLAM (ALP 100)			
Benzodiazepine	300	Flunitrazepam	200
a-idrossialprazolam	1.500	(±) Lorazepam	3.000
Bromazepam	900	RS-Lorazepamglucuronide	200
Clordiazepossido	900	Midazolam	6.000
Clobazam	200	Nitrazepam	200
Clonazepam	500	Norclordiazepossido	100
Clorazepato di potassio	500	Nordiazepam	900
Delorazepam	900	Oxazepam	300
Desalchilfurazepam	200	Temazepam	100
Diazepam	300	Triazolam	3.000
Estazolam	6000		
AB-PINACA (ABP/K3 10)			
AB-PINACA	10	UR-144 4-idrossipentil	10.000
AB-PINACA 5-pentanoico	10	ALINACA 5-idrossipentil	10.000
AB-PINACA 5-idrossipentil	10	ADB-PINACA N-(5-idrossipentil)	30
AB-FUBINACA	10	ADB-PINACA acido pentanoico	10
AB-PINACA 4-idrossipentil	10.000	5-fluoro AB-PINACA N-(4-idrossipentil)	30
UR-144 5-pentanoico	5.000	5-fluoro AB-PINACA	25
UR-144 5-idrossipentil	10.000		
ALFA-PIRROLIDINOVALEROFENONE (α-PVP 2.000)			
Alfa-pirrolidinovalerofenone	2.000		
ALFA-PIRROLIDINOVALEROFENONE (α-PVP 1.000)			
Alfa-pirrolidinovalerofenone	1.000		
ALFA-PIRROLIDINOVALEROFENONE (α-PVP 500)			

Alfa-pirrolidinovalerofenone	500		
ALFA-PIRROLIDINOVALEROFENONE (α-PVP 300)			
Alfa-pirrolidinovalerofenone	300		
CANNABINOLO (CNB 500)			
Cannabinolo	500	11-nor-A9 -THC-9 COOH	300
Δ ⁹ -THC	10.000		
MEPERIDINA (MPRD 100)			
Normeperidina	100	Meperidina	100
PREGABALIN (PGB 50.000)			
Pregabalin	50.000		
PREGABALIN (PGB 500)			
Pregabalin	500		
TRAZODONE (TZD 200)			
Trazodone	200		
UR-144/K4			
UR-144 acido 5-pentanoico	25	5-fluoro AB-Pinaca N-(4-idrossipentil)	10.000
UR-144 4-idrossipentil	10.000	ADB-PINAC N-(4-idrossipentil)	>10.000
UR-144 5-idrossipentil	5.000	AB-PINACA 4-idrossipentil	>10.000
XLR-11 4-idrossipentil	2.000		
ZALEPLON (ZAL 100)			
Zaleplon	100		
MESCALINA (MES 100)			
Mescalina	100		
MESCALINA (MES 300)			
Mescalina	300		
GABAPENTIN (GAB 2000)			
Gabapentin	2.000		
TILIDINA (TLD 50)			
Nortilidina	50	Tilidina	100
QUETIAPINA (QTP 1000)			
Quetiapina	1.000	Norquetiapina	10.000
PAPAVERINA (PAP 500)			
Papaverina	500	Diflunisal	1.000.000
Metotrexato	650.000	Metedrone	500.000
Pragablin	500.000	Fenelzina	8.000
Chinino	4.000		
KRATOM (KRA 3000)			
Mitraginina	300	7-idrossimitraginina	>50.000
CARISOPRODOL (CAR 2.000)			
Carisoprodol	2.000		
CARISOPRODOL (CAR 1.000)			
Carisoprodol	1.000		
FLUOXETINA (FLX 500)			
Fluoxetina	500		
OLANZAPINE(OZP 1.000)			
Olanzapine	1.000		
CITALOPRAM(CIT 500)			
Citalopram	500		
FLUOKETAMINE (FKET 1.000)			
2-(2-fluorophenyl)-2-methyla mino-cyclohexanone	1.000		
RISPERIDONE (RPD 150)			
Risperidone	150		
TAPENTADOL (TAP 1.000)			
3-((1R,2R)-3-(dimethylamin o)-1-ethyl-2-methylpropyl)ph enol	1.000		
N,N-DIMETHYLTRYPTAMINE(NND 1.000)			
N,N-Dimethyltryptamine	1.000		
SCOPOLAMINE(SCOP 500)			
Scopolamine	500	Atropine	3.000
MIRTAZAPINE(MTZ 500)			
Desmethylmirtazapine	500	Mirtazapine	500

Effetto del peso specifico delle urine
Quindici (15) campioni di urine con intervalli di peso specifico normale, elevato e basso (1,005-1,045) sono stati addizionati con sostanze a livelli di cut-off rispettivamente inferiori al 50% e superiori al 50%. Il Test rapido per tazza multi-farmaco è stato analizzato in duplicato usando quindici campioni di urina in cui non erano presenti sostanze e addizionati con sostanze. I risultati dimostrano che intervalli variabili di peso specifico delle urine non influiscono sui risultati dei test.

Effetto del pH delle urine
Il pH di un pool di urine negativo aliquotato è stato corretto a un intervallo di pH compreso tra 5 e 9 con incrementi di 1 unità di pH e addizionato con sostanze a livelli di cut-off inferiori al 50% e superiori al 50%. L'urina addizionata e con pH corretto è stata analizzata con del Test rapido per tazza multi-farmaco. I risultati dimostrano che intervalli variabili di pH non interferiscono con le prestazioni dei test.

Reattività crociata
È stato condotto uno studio per stabilire la reattività crociata del test con composti in urina priva di sostanze o in urina positiva alle sostanze contenente le sostanze del calibratore di cui sopra. I seguenti composti non mostrano reattività crociata quando vengono analizzati con del Test rapido per tazza multi-farmaco a una concentrazione di 100 µg/mL.

Composti che non presentano reattività crociata			
Acetilfenetidina	Cortisone	Zomepirac	d-pseudoefedrina
N-acetilprocainamide	Creatinina	Ketoprofene	Chinidina
Acido acetilsalicilico	Deossicorticosterone	Labelalolo	Chinino
Aminopirina	Destrometorfano	Loperamide	Acido salicilico
Amoxicillina	Diclofenac	Meprobamato	Serotonina
Ampicillina	Diflunisal	Isoxsuprina	Sulfametazina
Acido L-ascorbico	Digossina	d,l-propanololo	Sulindac
Apomorfina	Difenidramina	Acido nalidixico	Tetraciclina
Aspartame	Etil-p-aminobenzoato	Naproxene	Tetraidrocortisone
Atropina	β-estradiolo	Niacinamide	3-acetato
Acido benzilico	Estrone-3-solfato	Nifedipina	Tetraidrocortisone
Acido benzoico	Eritromicina	Noretindrone	Tetraidrocolina
Bilirubina	Fenoprofene	Noscapina	Tiamina
d,l-bromfeniramina	Furosemide	d,l-octopamina	Tionidazina
Caffeina	Acido gentisico	Acido ossalico	d,l-tirosina
Cannabidiolo	Emoglobina	Acido ossolinico	Tolbutamide
Cloralio idrato	Idralazina	Ossimetazolina	Triamere
Cloramfenicolo	Idroclorotiazide	Papaverina	Trifluoperazina
Clorotiazide	Idrocortisone	Penicillina-G	Trimetoprim
d,l-clorfeniramina	o-acido idrossi-ippurico	Perfenazina	d,l-triptofano
Clorpromazina	3-idrossitiramina	Fenelzina	Acido urico
Colesterolo	d,l-isoproterenolo	Prednisone	Verapamil
Clonidina			

【CARATTERISTICHE PRESTAZIONALI DEL TEST PER L'INDIVIDUAZIONE DI ALCOL】
Il limite di rilevamento del **Test rapido per la rilevazione di alcol nelle urine** è compreso tra lo 0,02% e lo 0,30% per il livello relativo approssimativo di alcol nel sangue. Il livello di cut-off del **Test rapido per la rilevazione di alcol nelle urine** può variare in base alle normative e alle leggi locali.

【SPECIFICITÀ DEL SAGGIO PER L'ALCOL】
Il **Test rapido per la rilevazione di alcol nelle urine** reagirà con gli alcol metilici, etilici e alilici.

【SOSTANZE INTERFERENTI CON L'ALCOL】
Quando si utilizzano campioni diversi dalle urine, le seguenti sostanze possono interferire con il **Test rapido per la rilevazione di alcol nelle urine**. Le sostanze indicate di solito non sono presenti nelle urine in quantità sufficiente da interferire con il test.

- A. Agenti che potenziano lo sviluppo del colore
- Perossidasi
 - Forti ossidanti
- B. Agenti che inibiscono lo sviluppo del colore
- Agenti riducenti: acido ascorbico, acido tannico, pirogallolo, mercaptani e sililati, acido ossalico e acido urico
 - Bilirubina
 - L-dopa
 - L-metildopa
 - Metampirone

- 【BIBLIOGRAFIA】**
1. B. Cody, J.T., "Specimen Adulteration in drug urinalysis. Forensic Sci. Rev., 1990, 2:63.
 2. C. Tsai, S.C. et. al., J. Anal. Toxicol. 1998; 22 (6): 474
 3. Tietz NW. Textbook of Clinical Chemistry. W.B. Saunders Company. 1986; 1735.
 4. Hawks RL, CN Chiang. *Urine Testing for Drugs of Abuse*. National Institute for Drug Abuse (NIDA), Research Monograph 73, 1986.
 5. Baselt RC. Disposition of Toxic Drugs and Chemicals in Man, 6th Ed. Biomedical Publ., Foster City, CA 2002.

Indice dei simboli					
	Consultare le istruzioni per l'uso o consultare le istruzioni per l'uso elettroniche		Contenuto sufficiente per <n> test		Limite di temperatura
	Dispositivo medico-diagnostico <i>in vitro</i>		Codice lotto		Numero di catalogo
	Rappresentante autorizzato nella Comunità europea		Data di scadenza		Non riutilizzare
	Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso		Produttore		Marchio CE



Hangzhou AllTest Biotech Co.,Ltd.
#550,Yinhai Street,
Hangzhou Economic & Technological Development Area,
Hangzhou, 310018 P.R. China
Web: www.alltests.com.cn Email: info@alltests.com.cn



EC REP
MedNet EC-REP GmbH
Borkstrasse 10,
48163 Muenster,
Germany

Numero: 14601688503
Data di revisione: 2025-01-24