SPUGNETTA STERILE MONOUSO PER ELETTROCHIRURGIA
 STERILE DISPOSABLE CLEANING PAD FOR ELECTROSURGERY
 TAMPON STÉRILE À USAGE UNIQUE POUR ÉLECTROCHIRURGIE
 ELEKTRODEN-REINIGUNGS-PAD STERIL, ZUM EINMALGEBRAUCH
 ESPONJA ESTÉRIL MONOUSO PARA ELECTROCIRUGÍA
 ESPONJA ESTERILIZADA DESCARTÁVEL PARA A ELETRO-CIRURGIA
 СТЕРИЛЬНАЯ ГУБКА ДЛЯ ОДНОРАЗОВОГО ИСПОЛЬЗОВАНИЯ В ЭЛЕКТРОХИРУРГИИ
 СТЕРИЛНА ГЪБИЧКА ЕДНОКРАТНИ ЗА ЕЛЕКТРОХИРУРГИЯ
 STERILNÍ HOUBÍČKA NA JEDNO POUŽITÍ PRO ELEKTROCHIRURGII
 STERIL SVAMP ENGANGSBRUG TIL ELEKTROKIRURGI
 ΑΠΟΣΤΕΙΡΩΜΕΝΟ ΣΦΟΥΓΓΑΡΑΚΙ ΜΙΑΣ ΝΟΝΟ ΧΡΗΣΗΣ ΓΙΑ ΗΛΕΚΤΡΟΧΕΙΡΟΥΡΓΙΚΗ
 STERIILNE TAMPOON ÜHEKORDELT KASUTATAVAD ELEKTROKIRURGIA TOOTED
 STERIILI SIENI KERTAKÄYTTÖINEN SÄKHÖKIRURGIAN VÄLINE
 STERILNA SPUŽVICA ZA JEDNOKRATNU UPOTREBU ZA ELEKTROKIRURGIJU
 STERIL SZIVACS EGYSZER FELHASZNÁLHATÓK ELEKTROSEBÉSZETBEN
 STERILA ŠVAMMĪTE VIENREIZ LIETOJAMI ELEKTROKIRURĢIJĀ
 STERILI VIENKARTINĒ ABRAZYVINĒ KEMPINĒLĒ
 STERIELE SPONS EENMALIG GEBRUIK VOOR ELEKTROCHIRURGIE
 GĄBKA STERYLNA DO JEDNORAZOWEGO UŻYTKU W ELEKTROCHIRURGII
 BURETE STERIL PRODUSE DE UNICĂ FOLOSINȚĂ PENTRU ELECTROCHIRURGIE
 STERILNÁ HUBKA JEDNORAZOVÉ KOMPONENTY PRE ELEKTROCHIRURGIU
 STERILNE GOBICE ZA ENKRATNO UPORABO V ELEKTROKIRURGIJI
 STERIL SVAMP ENGÅNGSBRUK FÖR ELEKTROKIRURGI
 STERİL SÜNGER ELEKTROCERRAHİ İÇİN TEK KULLANIMLIK
 STERILNI SUNĐER JEDNOKRATNA OPREMA ZA ELEKTROHIRURGIJU
 STERIL SVAMP ENGANGSUTSTYR TIL ELEKTROKIRURGI

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INSTRUCTIONS FOR USE**WARNINGS**

The product is disposable and sterilised with EO gas. Do not reuse. Reusing the product may lead to: possible presence of biological residues that may cause cross-infections; alteration of materials; loss of initial functional features of product.

Sterility is not guaranteed if the package has been opened and/or damaged.

Do not use after the expiration date. Sterility is not guaranteed.

After checking the integrity of the package, please check its content; in case of visible damages or defects, do not use the product and return it to FIAB.

The devices must be connected and operated by qualified staff.

For the use of the generator, handpieces, neutral plates, cables and other accessories, please refer to the instructions for use supplied by the manufacturer.

Waste from health facilities must be disposed of in accordance with regulations in force.

INDICATIONS

Cleaning the electrode of the handpiece during an electrosurgical procedure.

CONDITIONS OF USE

Remove the protective film and carefully attach the sticker that is on the back of the sponge to the drape sheet to obtain its stable positioning and an easy cleaning of the tip of handpiece.

NB: In order to avoid any material that is removed falling on the surgical field, do not clean the tip of the handpiece in the immediate vicinity of the field.

The radio-opaque tracer included in the product allows for its radiographic detection, in case of accidental loss in the surgical field.

In the event of a reduction in the abrasive and cleaning effect of the sponge, it is recommended to replace the product.

Dispose of the product after each procedure.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force.

GENERAL NOTES

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

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