OCCHIALI D'ISOLAMENTO MEDICALI
MEDICAL ISOLATION GOGGLES
LUNETTES MÉDICALES D'ISOLEMENT
GAFAS DE AISLAMIENTO MÉDICO
ÓCULOS DE ISOLAMENTO MÉDICO
MEDIZINISCHE ISOLATIONSBRILLE
ΙΑΤΡΙΚΑ ΓΥΑΛΙΑ ΑΠΟΜΟΝΩΣΗΣ
MEDISCHE ISOLATIEBRIL
MEDICINSK ISOLATIONSBRILLE
MEDICINSK ISOLERINGSGLÖGONE
LÄÄKETIETEELLISET ERISTYSLASIT
MEDISINSKE ISOLASJONSBRILLE
OKULARY IZOLACYJNE MEDYCZNE
LÉKAŘSKÉ IZOLAČNÍ BRÝLE
MEDICINSKA IZOLACIJSKA OČALA
MEDICINSKE IZOLACIJSKE NAOČALE
ORVOSI SZEMÉVEZET
OCHEAREI DE IZOLAT MEDICAL
МЕДИЦИНСКИ ИЗОЛАЦИОННИ ОЧИЛА
ةي ب ط ل ا ل ز ع ل ا ت ا ر ا ظ ن

REF **GDG02 (Gima 25266)**

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ENGLISH

Instructions for Use

Model: Fully Enclosed with Air Vents

Rev. A/1 Effective:2022-10-17

Description

The Medical isolation goggles is intended to create a physical barrier separating the eyes from the immediate surroundings. This device prevents liquids or droplet splash from contacting the user.

The Medical isolation goggles is intended for single use.

Indication for Use:

Medical isolation goggles are personal protective equipment intended for protection of the associated mucous membranes eyes from splashes, sprays, and spatter of body fluids.

Contraindications:

None.

Product side effects:

This product has no side effects.

Warning:

1. Do not use if the components show any sign of damage.
2. Do not expose components to temperature greater than listed on package labeling.
3. Dispose of the product as biohazardous waste per institutional protocol.

Operation:

Take out the goggles from the packaging bag, confirm that the goggles in good condition, and tear off the protective film; the goggles will fix the elastic band behind the head, and adjust the soft fixation device to make it fit closely with the face.

Storage:

Store product in a dry place at 0 - 30°C.

Keep away from sunlight.

Explanations for symbols

	Expiration date		Disposable device, do not re-use
	Lot number		Don't use if package is damaged
	Date of manufacture		Keep away from sunlight
	Medical Device compliant with Regulation (EU) 2016/425		Keep in a cool, dry place
	Manufacturer		Authorized representative in the European Community
	Product code		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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