REF: 0111 PINZETTA OMBELICALE MONOUSO

ISTRUZIONI D'USO

AVVERTENZE:

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INSTRUCTIONS FOR USE

WARNINGS

- The device is supplied sterile.
- Do not use if the package is not intact.
- Use immediately after opening the enclosure.
- **The device should only be used by qualified medical personnel.**
- To be used only for the umbilical cord.
- The product is single use. Do not use after use.

INDICATIONS OF USE

The device is intended for use in obstetrics the closure of the umbilical cord.

METHOD OF USE

Remove the product from the sterile envelope.

Place two umbilical tweezers on the cord, one about ten centimeters from the baby's body, the other at about fifteen centimeters (the distance between tweezers must be about five centimeters).

Take the two ends of the tweezers with two fingers and after placing it on the umbilical cord, tighten until it clicks into place. Placement is at the discretion of the doctor

MAKE SURE THAT THE TWEEZER IS TIGHTENED AND THE STOPS ARE WELL ENTANGLED BETWEEN THEM.

Cut the umbilical cord still attached to the placenta (the placenta is naturally expelled at most twenty minutes after the baby's exit).

The tweezers can only be removed by pulling the back of the clip.

WARNINGS

- **Periodically check the tightness of the tweezers on the newborn at short intervals of time until the cord has healed.**
- **In the event that continuous control is impossible, it is advisable to apply an additional safety forceps.**

STORAGE / STORAGE

Store in a dry place. Do not expose to temperatures above 50 ° C.

ELIMINATION AND DISPOSAL AFTER USE

The product is single use so it is not reusable, it can be discharged or burned together with other waste, observing the provisions of the Local Authorities. Elimination by incineration does not give rise to the formation of toxic residues.

ANY SERIOUS INCIDENT WITH THE MEDICAL DEVICE SUPPLIED BY US MUST BE REPORTED TO THE MANUFACTURER AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH IT IS LOCATED

LEGEND SYMBOLOGY			
	MEDICAL DEVICE		DO NOT USE IF DAMAGED PACKAGING
	DEVICE STERILE ETO		PROTECT FROM SUNLIGHT
	DOES NOT CONTAIN LATTICE		PROTECT FROM WET
	DO NOT STERILISE TWO		QR IN DATA MATRIX FORMAT
	SINGLE-USE		CE MARKING
	MANUFACTURER		IDENTIFICATION CODE
	DATE OF MANUFACTURE		PRODUCTION BATCH
	EXPIRY DATE		UNIQUE DEVICE IDENTIFICATION
	CORRUGATED PAPER		PLASTIC MATERIAL

AK MEDICAL S.R.L. Via del Chioso N.8/10 - 24030 Mozzo (BG) Italia
TEL. 035 613712 CELL. 334 1807240 Site: <http://www.akmedical.it>
Mail: commerciale@akmedical.it
Rev. 01-06/2025 Made to Romania