Fig. 1

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EN REUSABLE ELECTROCAUTERY TIP FOR REUSABLE ELECTROCAUTERY DIRECTIONS FOR USE

INDICATIONS

Cauterisation of tissues and small blood vessels during surgical operations. No high frequency generator is required. Specially suitable for ambulatory care surgery.

DESCRIPTION

F7297 and F7299 are reusable electrocauteries which allows batteries and tip replacement. Inserting a disposable sterile tip (F7298 series) on the anterior part, reusable electrocautery becomes a self-powered system for cauterisation of small vessels, and it provides an excellent alternative to the radio frequency electrocautery unit. F7297 and F7299 cauteries are intended for different fields of application: F7297 is specific for ophthalmology, while F7299 can be used for gynaecology, orthopedics, Otorhinolaryngology, urology and dermatology. Anyway, they can be both used in specialties where an accurate and quick cauterisation is required. Since electrocautery body is supplied non-sterile (only tips are sterile) these models can be used for ambulatory procedures only. Tips are supplied with a sterile protection sheath in PE, which must be used to protect electrocautery body when in use.

Depending on the need, three different tips can be supplied: fine long (F7298FG), fine short (F7298F) and fine large (F7298L). F7297 cautery must be used with the fine short one (F7298). The Working temperatures reach 600°C/1100°F for F7297 cautery 1.200°C/2200°F for F7299 cautery.

WARNING

- Power is supplied by batteries.
- Use only alkaline batteries SIZE AA 1,5V LR6.
- It is advisable to replace batteries after every application.
- The tips, and the supplied protections, are single-use and sterilized with ethylene oxide gas – do not reuse – Reusing the product may lead to:
 - o Possible presence of biological residues that may cause cross-infections;
 - o Alteration of materials;
 - o Loss of initial functional features of product;

- The tips and protections may not be sterile if the package has been opened and/or damaged.
- After having controlled that packaging is not damaged, verify cautery body; in case of visible damages or defects, do not use the product and send it back to FIAB.
- Avoid direct contact of cautery with liquids.
- To clean the body use only water-based disinfectants (containing no alcohol).
- Do not sterilise with steam.
- The body has a safety cap-closing system guaranteeing protection when the tip is not inserted.
- The equipment is to be prepared and used by specialized personnel.
- **Avoid use in presence of flammable anaesthetic products and mixtures.**
- Replace the protection when replacing the tip.
- After use always reinsert protection cap before disposal.
- If needed, the body may be sterilized (without tip and batteries) with ethylene oxide for up to 10 times (validation of sterilization is under the responsibility of each end user). After sterilization, the end user should operate in order to avoid any contamination in replacing batteries.
- As concerns disposal, please follow what explained at the paragraph "DISPOSAL".

Caution: remove the tip when the equipment is not used or it may cause a fire.

PREPARATION AND USE

- Open the package containing the tip.
- Extract the protection cap from the handle by making it slide in parallel to the long axis.
- Insert the transparent protection (Fig. 1) on the stem, inserting the handle between the white strip and the paper, thus protecting the tip lodgings.
- Remove the protection package.
- Insert the tip on the front of the stem, piercing the transparent protection.
- For a preliminary control of the electrocautery, follow these instructions:
 - o Keep the equipment far from things or people, and press the start button;
 - o Make sure the tip becomes incandescent;
NB As concerns the low temperature model, since its tip does not become incandescent, you can verify the right functioning as follows: press start button and touch a wet gauze (wet with sterile water or saline solution), then wait 2/3 seconds before noticing water evaporation;
 - o If needed, replace the batteries (see corresponding paragraph);
- When using the equipment on the patient, make short (2-3 seconds) and repeated applications. Do not keep the equipment on for too long.
- Do not exert pressures that may misshape the tip.
- Once the application is over, remove the tip from the handle, remove the transparent protection and throw away; if needed, clean the handle and insert the cap.

REPLACING BATTERIES

To replace the batteries, press the cap at the bottom of the handle, turn clockwise and pull it out. Replace the batteries, inserting them with the positive pole towards the inside of the handle. Put the cap back in place by pressing it and turning it anticlockwise until it stops.

CONTRAINDICATIONS

- F7299: Don't use for intra-ocular applications.
- The electro-cauteries are not suitable to cauterize heavy haemorrhages.

- If the stem protection is not used during applications, there is the risk of infections since the stem is not sterile.

STORAGE

The product must be stored in its original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package could damage the product.

GENERAL NOTES

If a serious incident occurs while using this device or as a result of its use, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

WASTE DISPOSAL

Waste coming from health facilities must be disposed of according to the regulation in force.

The batteries must be disposed of in line with current regulations. Do not burn batteries.

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