GIMA

PROFESSIONAL MEDICAL PRODUCTS

DISPOSABLE PVC LARYNGEAL MASK

MASCHERA LARINGEA MONOUSO PVC

MASQUE LARYNGÉ JETABLE PVC

MASCARILLA LARINGEA DE PVC DESECHABLE

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Contents: 1 Laryngeal Mask Airway

- Deflate cuff before insertion
- The index finger is placed at the junction of the cuff and the tube.
- With the hand extended and the neck flexed, carefully flatten the laryngeal mask airway tip against the hard palate.
- To facilitate laryngeal mask airway introduction into the oral cavity, gently press the middle finger down on the jaw.
- Using the index finger, advance the laryngeal mask airway while following the contours of the hard and soft palates.
- Maintain pressure with the finger on the airway tube and advance the laryngeal mask airway until definite resistance is felt at the base of the hypopharynx. Note the flexion of the wrist.
- Inflate the cuff with enough air to obtain an airway at 18-20 cm H2O. Do not exceed the maximum volume recommended for the cuff.
- Final assessment of seal, suggest inserting a bite block and attach the tube to the ventilation equipment.

Caution: Gently maintain cranial pressure with the non-dominant hand while removing the index finger. Inflation without holding the tube allows the mask to dislodge. Do not attempt to reconfirm mask position after any change in the patient's head or neck position.

Warning: Check for clear airway passage prior to use. Do not use in the presence of lasers and electrosurgery equipment, as it may be flammable. The laryngeal mask airway device does not protect the patient from the effects of regurgitation and aspiration.

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