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GIMA 34075 - 34102

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu
Province, CHINA
Made in China

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YDC-6A1, YDC-6A3

Intended Use

Vacuum Mattress Stretcher is used to transferring the wounded in hospitals, gymnasiums, ambulance and station. This product is used for emergency rescue.

Storage

Store this stretcher in the place of damp proof and non-corrosion environment.

Storage, transportation, as shown by the box stacking.

Component

Vacuum Mattress Stretcher is composed of fabric, stuffing, safety belt and special air pump.

Material

Vacuum Mattress Stretcher is made of TPU and Nylon.

Specification

The dimension is as below:

Model	Specification	Load-bearing	Picture
YDC-6A1	197x97x7cm	160kg	
YDC-6A3	205x100x5cm	160kg	

Shelf Life

8 years

How to use the device

- 1 Open the vacuum mattress stretcher flat and keep the particles in uniform distribution inside of stretchers.
- 2 When patient lie on the stretcher, the first-aid person need to connect the pump trachea with the stretcher valve together, and then open the valve as counterclockwise direction, exhaust the air from the stretcher according to the demand can be adjust the stretcher to be soft or hard. Closed the valve by hand as clockwise direction if feel the stretcher is comfortable.
- 3 Using the safety belt restraint the patient, first-aid person can hold the handles transporting patients.

Warning

1. Use safety belt when necessary.
2. Keep clean as routine.
3. Store this stretcher in the place of damp proof and non-corrosion environment.
4. Be transferred by common transporting tools.
5. Packed in cartons.
6. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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