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VERSIONE/ VERSION N.02	REDATTO/DREW UP BY <i>F. Marrone</i>	DATA/DATE 26/05/2021	APPROVATO/ APPROVED BY <i>Fabio Casella</i>	DATA/DATE 26/05/2021
VERSIONE/ VERSION N.01	REDATTO/DREW UP BY <i>G. Andraghetti</i>	DATA/DATE 26/10/2005	APPROVATO/ APPROVED BY <i>Maurilio Longo</i>	DATA/DATE 26/10/2005

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1. INTRODUCTION

The models SPD & SPD/A are stools for professional use, completely built in stainless steel, 360° pivoting self-breaking casters, rubber-tyred.

The height can be comfortably adjusted by means of a gas pump; the back-rest is 180° revolving and depth-adjustable.

The seat is balanced density padded and Peltex-lined.

There are two models: SPD, SPD/A and they all differ in their specific base shape.

2. INSTALLATION

(See following scheme)

Open the package containing the stool and follow step by step the following set-up instructions:

- Insert the pivot to the wheels
- Insert the wheels to the basement
- Set up the gas column on the basement
- Set up the movement gear to the gas column
- Insert the backrest blade in the apposite support and screw the knob
- Fix the upholsteries with the supplied screws

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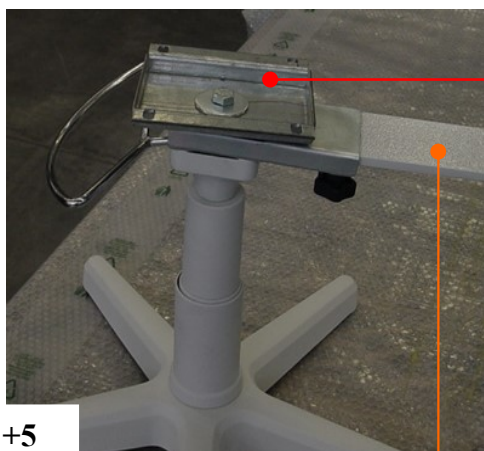
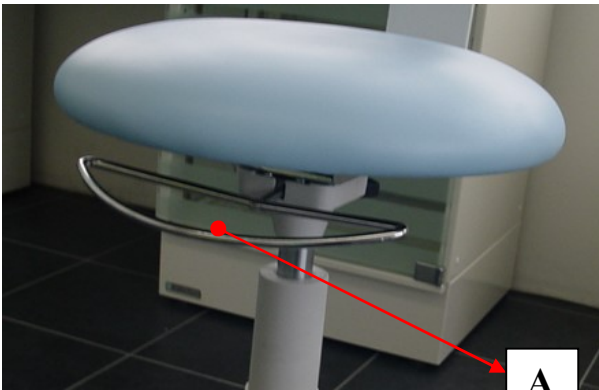
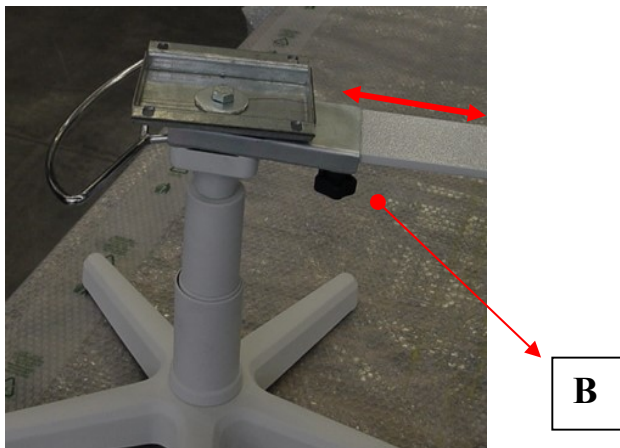
3. SEAT MOVEMENT

By lifting up the lever A both rising and lowering movements are blocked; to block the stool in the desired position it is enough to push down the same lever.

4. BACKREST MOVEMENT

It is possible to move longitudinally the backrest through the knob B.

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5. MAINTENANCE.
CHAIR CLEANING.

All materials used to construct the equipment can support cleaning and disinfections operations due in these instructions.

The great variety of drugs chemical products, which are used on medical study can damage painted surfaces and plastic material parts.

Proofs and inquiries made substantiate that surfaces cannot be outright protected by aggression of all products which you can find on the market.

The effects of these products above all depend on time and stay on surfaces.

It is important to remove that chemical by a damp cloth from equipment surfaces immediately.

To clean, please, kindly use a damp cloth; do not spray the detergents directly on the equipment.

ATTENTION!

Do not use products which contain alcohol, ammonia, abrasive substances, benzene.

The concentrations listed below are the maximum ones that are consented and they cannot be surpassed:

Quaternary Ammonium: maximum 2%

Glutaric Aldehyde: maximum 2%

**MEDI-CARE SOLUTIONS WILL NOT BE HELD
RESPONSIBLE FOR DAMAGES CAUSED BY
USE OF DISINFECTANTS WHICH CONTAIN
ACTIVE SUBSTANCES DIFFERENT FROM
THOSE ABOVE MENTIONED.**

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