**SET STECCOBENDE A DEPRESSIONE  
SPLINT****KIT ATTELLE GONFLABLE****KIT DE FÉRULA DE VACÍO****CONJUNTO DE TALA A VÁCUO****VAKUUM-SCHIENENSATZ****ΣΕΤ ΑΠΟ ΝΑΡΘΗΚΕΣ ΚΕΝΟΥ****ZESTAW SZYN PRÓŻNIOWYCH****VACUUM SPLINT KIT****VACUUM SPLINT KIT****ALIPAINELASTASARJA****KOMPLET VAKUUMSKE OPORNICE****SÚPRAVA VÁKUOVEJ DLAHY****SET ATELĚ VACUUM****VACUÜMSPALK-KIT****KOMPLET VAKUUMSKIH UDLAGA****VÁKUUM SÍN KÉSZLET****VACUUM SPLINT-SÆT****VACUUM SPLINT KIT****VAKUUMINIO ĮTVARO RINKINYS****VAKUUMA ŠINU KOMPLEKTS****VACUUM SPLINT KIT**

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## Intended Use

Splint is used to fix each part of the body.

## Component

Splint is mainly composed of frame, foam and burrs and zipper, safety belt, air pump and valves.

## Material

Splint is made of Aluminum Alloy, PVC, EVA, TPU and Oxford leather.

## Specification:

The dimension is as below:

| Model | Specification    | Picture                                                                            | Material |
|-------|------------------|------------------------------------------------------------------------------------|----------|
| AS-04 | Full leg<br>45,8 |  | TPU      |
|       | Full arm<br>41,3 |                                                                                    |          |
|       | Full elbow<br>52 |                                                                                    |          |

## Storage

- Store in moisture-proof and non corrosive environment.
- Stacking and stacking according to the boxes in storage and transportation.

## Cleaning and maintenance

Keep clean as routine. Wipe with 75% alcohol solution.

Check the connecting and rotating parts of the stretcher.

Check the safety belts regularly, if they are worn or have cracks, replace them immediately.

## Transportation

The Splint can use general means of transport.

No inversion is allowed during storage and transportation, and stacking is carried out as shown in the packing box.

## Shelf Life

8 years

### Use method

| Model | Picture                                                                           | Use method                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|-------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| AS-04 |  | <p>Open the bag, take out the splint, fix it on four limbs of the patient according to wounded place.</p> <p>The splint can be used on full elbow, full arm, hull leg.</p> <ol style="list-style-type: none"> <li>1. Wrap a splints around the injured extremity, then stick burrs.</li> <li>2. Connect the valve and air exhausted with air pump.</li> <li>3. Rotate the valve and deflate.</li> <li>4. Remove the stick burrs and take off splint.</li> </ol> |

### Precaution and Warning

1. Always keep clean (including disinfection).
2. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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