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**DIRECTIONS FOR USE
CONNECTING CABLES BETWEEN PADS
AND ELECTROSURGERY DEVICES****DESCRIPTION AND INDICATIONS**

The packaged product is sold in is **NOT STERILE**.

CONNECTING CABLES WITH TAB CONNECTIONS FOR DISPOSABLE PADS FOR ELECTROSURGERY. On one end the cables are fitted with a clamp connection, and on the other, with various connectors which make the cables compatible with numerous electrosurgery devices.

MODELS F7902 – F7902-S – F7904 - F7902/3MT

Connecting cable with a mono Ø 6,3 mm jack connection

MODELS F7903

Connecting cable with Valleylab-type connection

MODELS F7902/4

Connecting cable with male Ø 4,0 mm plugs

MODELS F7902/24

Connecting cable with 2 male Ø 4,0 mm plugs

MODELS F7903/F

Connecting cable with Valleylab-type connector for Force 1-2-4-4B models

CONNECTING CABLES FOR RE-USABLE PADS FOR ELECTROSURGERY WITH FEMALE 4MM CONNECTORS

On one end the cables are fitted with 4 mm safety terminals for connecting to the pads, and on the other with various connections which make the cables compatible with numerous electrosurgery devices.

MODELS F7922 – F7924

Connecting cable with a mono Ø 6,3 mm jack connection

MODELS F7923

Connecting cable with Valleylab-type connection

MODELS F7922/4

Connecting cable with male Ø 4,0 mm plugs

MODELS F7922/24

Connecting cable with 2 male Ø 4,0 mm plugs

MODELS F7923/F

Connecting cable with Valleylab-type connection for Force 1-2-4-4B models

INSTRUCTIONS FOR USE

Carefully check packaging, cables and connection plugs. Do not use the product in case of visible damages or defects.

Connecting of cables with clamp connection:

- raise the moveable lever positioned on the blue plastic part of the clamp.
- insert the tab of the disposable plate into the receptacle located on the blue plastic part of the clamp.
- replace the lever in the home position.

Connecting of cables with 4 mm diameter male plugs:

- Insert the plugs into the holes located on the pad;
- On being inserted the protective sheath on the tip of the plug retracts in order to allow for connection to the pad.

WARNING

- Connect the clamp to the tongue of the disposable plate, being careful to position the clamp outside the gel-covered area.
- The product is supplied non-sterile.**
- The equipment and plates used with this product must conform to the regulations currently in force.

NB: The products must be used by qualified healthcare personnel.

Do not pull cables at the insulation

CLEANING

Cables can be cleaned with a soft cloth soaked with disinfecting solution.

LIFE OF PRODUCT

The life of the products depends on the number of uses and on maintenance type. The products must be immediately destroyed in case of deterioration of the material.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

WASTE DISPOSAL

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force..

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