ITALIANO

INDICAZIONI

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ENGLISH

INDICATIONS

The Tondaus Surgical Skin Marker is intended to be used to mark on human skin for positioning identifications in Ordinary Surgery, Enterochirurgia, Orthopedics, Neurohormone Surgery, Cardiovascular Surgery and Radiotherapy.

CONTRAINDICATIONS

Do not apply on damaged mucous membranes and skin or when wounds are present.

CAUTION AND WARNING

1. Patients should be considered for allergic reactions to Gentian violet/Allura Red AC/Indigo/Titanium dioxide(corresponding to different colors). During use, if the patient is found to be unwell, stop using it immediately and use it properly.
2. This product can be used only once in a single patient to avoid cross infection.
3. This product should be used on dry skin. If the patient's skin is wet or sweaty, please dry the skin and keep the skin. Dry and fresh, then follow the instructions above.
4. Pay attention to the protection of the pen tip when using it. When not in use, cover the pen cover to prevent dry ink.
5. The package should be used immediately after opening. Do not use if the package is damaged or stained. After the product is used, it should be disposed as medical waste.
6. Do not soak or wipe the product with an organic solvent. Organic solvents may damage the product.
7. Operate by trained medical professionals.
8. Dispose of as medical waste after use.

DIRECTIONS FOR USE

1. Check the integrity of the inner package prior to use. Do not use if the inner package breaks.
2. Disinfect and dry the skin first (note: be sure to use it on dry skin);
- 2). Use a skin marker to mark the skin;
- 3). To remove the mark, use 75% alcohol to erase.

STORAGE AND TRANSPORTATION CONDITIONS

- 1). This product should be stored in a cool, ventilated, dry place, avoiding contact with water, direct sunlight, extreme temperature or humidity, and prevent contact with corrosive gases. Temperature: ≤ 40°C , humidity: ≤90%, pay attention to anti-rat, pest control, to prevent damage to the packaging.
- 2). One-time use of medical skin markers allows the use of general transportation vehicles, but must prevent severe shocks, vibrations and rain and snow during transportation.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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