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1. MANUAL INTRODUCTION

The manual stair stretcher is widely used in the home for the old people, school security, hospitals, hotels etc for emergency evacuation

Note:

Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.

► This manual is applicable to SKB1C02-1 manual stair stretcher. It contains the installation method, operating instructions and maintenance check points.

► The graphics in this manual are for users' reference only. In actual use, please refer to the real object.

► This user manual includes the description, instruction, maintenance and technical parameters of the product.

► Any questions, please contact the after-sale service center or distributor.

► Anyone using or operating this product should read and comply with it.

► Except for some unexpected situations, users are required to be familiar with the usage and precautions.

I. Manual stair climbing wheelchair overview

SKB1C02-1 Climbing chairs made of aluminum alloy material, lightweight, easy folding, easy to operate etc. The chair takes up little storage space after folding, and is suitable for carrying in a private car. Weight capacity:180KG

II. Wheelchair structure

III. Specification

Chair size 109*51*155cm
Folded size 109*51*25cm
Seat 49*42.5cm;Height=50cm
Top gripping frame H=108,125,142,155cm
Weight 15Kg
Load capacity MAX.180Kg

IV. Downstairs operation

NOTICE!:

The manual stair stretcher SKB1C02-1 can't be used for upstairs since the track is one-direction 2 operators must carry the stair stretcher to upstairs and then use for downstairs operation.

1. Push the stair climbing chair through the floor until placing the front wheels on the edge of the stairs.

2. Tread the lower cross pipe with the right foot, pull the tie rod back with both hands to tilt the chair down.

3. Slowly, put down the chair on the first edge, and lean down the stairs.

4. Slightly lean to the angle of the belt and the stairs.

5. During the going down process, place the hands slightly down the rod to not produce bumps.

6. When the track reaches the landing, slowly lift the chair.

V. Operation of each part

a. Handle

Open: Press the button. While pressing it, lift the handle up slightly up, then release the button to fix it.

Lock: Press the button, lift the handle up, and release the button. Then, it will be locked.

b. Front lifting handle

Press the red button to unlock the hole and extend the handle. Release the button to fix the handle.

c. Rubber track frame

Open: Step on the foot pedal; then, the track frame will automatically open through its gas lift system.

Lock: Lift the evac chair with both hands and hold it down, press the crawler belt support bar down until the pedal locks it again.

d. Pull rod

Push the bar inwards to unlock the lock system. Once it is unlocked, pull the adjusting bar upwards or downwards to adjust it to the needed height.

e. Folding wheelchair

Grasp the support rod and pull it downwards. When the lock slides out of the curved slot, rotate the chair surface tube to fold the chair surface.

f. Handrails

g. Pedal

h. Safe-belt, Headrest & Straps

i. Track Adjustment

j. Brakes

2. MAINTENANCE

- In order to use the Manual Stair Stretcher safely, it is necessary to perform regular safety inspections on the Manual Stair Stretcher. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
- When the Manual Stair Stretcher frame ages and it reaches a certain servicing time, which the metal part of the Manual Stair Stretcher and plastic parts of the Manual Stair Stretcher frame

can be recycle.

- Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
- If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the Electric Stair Stretcher, which will cause the stainless steel surface to rust.
- Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
- It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
- The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
- Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axle bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.
- For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a bedside table equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the bedside table also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
- If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

3. PACKAGING, TRANSPORT, STORAGE

- The packaging of trolley is carried out according to the contract or product standard.
- Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the trolley.
- The trolley should be stored:
 - Ambient temperature: -10°C ~ +40°C.
 - Relative humidity: ≤ 85%.
 - Atmospheric pressure: 70 kPa ~ 106 kPa.
- The max load is 180Kg

4. AFTER-SALES SERVICE

- Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
- If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
- From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification, the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
- From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
- Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD

Notice! any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Don't use the device in direct contact with the skin.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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Jiangsu Saikang Medical Equipment Co.,Ltd
SUNGO Europe B.V. Fascinatio Boulevard 522,
Gima S.p.A.