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ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Product Name: Sterile Gynaecological Collectors for single use

Indications: This product is suitable for diseases that require gynecological examination.

Intend purpose: The product is used in gynecology and venereal disease.

Contraindications: None.

Caution:

A. When the product is used, it should strictly implement the requirements of aseptic operation specifications and related regulations, and be used by trained doctors and nurses. In the course of use, the operator or user's body should be treated in a timely manner according to medical regulations.

B. If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.

C. This product is used for single use and is destroyed after using; if the small package is damaged and the product is prohibited to be used; this product is sterilized by ethylene oxide and the product is sterile; the product is valid for 5 years and should not be used after it expires.

D. Appropriate types and specifications should be selected according to the patient's body type.

E. If there are burrs, cracks, or nodes in the application area of the gynaecological collector, they should not be used.

F. This product shall be disposed of according to the requirements of waste disposal in the local medical regulations after using.

Patient group: It is suitable for people who require gynecological examination.

Intended users: It can be used by trained doctors, nursing staff.

Clinical benefits: The product plays an auxiliary role in the treatment of diseases.

Any residual risk and any undesirable side-effects: none

Install information: none

Performance characteristics: The product is provided in a sterile state and sterilized with ethylene oxide. Single-use

Physical performance:

a. The plastic part is complete, without obvious defects such as sharp edges and burrs, and there is no obvious bending phenomenon in the plastic part.

b. The surface is clean and free from obvious foreign objects and impurities.

Structural composition: The product are composed of handle and collector head

Method:

- Use the dilator to open the vagina at an appropriate angle.
- Determine the location to be inspected or sampled.
- Deepen the sampling head of the gynecological sampler to the sampling site and take a rotating or wiping flat method to sample.

Storage: The product should be stored in a well-ventilated room temperature.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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