**BARELLA DI SOCCORSO
EMERGENCY STRETCHER
BRANCARD D'URGENCE
NOTFALLTRAGE
CAMILLA PARA EMERGENCIAS
MACA DE EMERGÊNCIA
ΦΟΡΕΙΟ ΕΚΤΑΚΤΗΣ ΑΝΑΓΚΗΣ
АВАРИЙНА НОСИЛКА
POHOŤOVOSTNÍ NOSÍTKA
NØDBÅRE
KANDERAAM HÄDAOLUKORRAKS
HÄTÄPAARIT
NOSILA ZA HITNE SLUČAJEVE
VÉSZHELYZETI HORDÁGY
NEŠTUVAI SKUBIAI PAGALBAI
AVĀRIJAS NESTUVES
FØRSTEHJELPSBÅRE
NOODBRANCARD
NOSZE RATUNKOWE
TARGÅ DE URGENTÅ
NÚDZOVÉ NOSIDLÁ
NOSILA ZA NUJNE PRIMERE
NØDBÅR**

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Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd
No.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China
Made in China

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Name: Emergency Stretcher
YXH-4A**Accessories:**

two safety belts plastic lock & polypropylene fiber.

Technical data:

1. Product size (LxW xH): 168x42x6cm, the longest can be up to 210cm.
2. Folding size (LxW xH): 120x42x8cm
3. Carton size (1pc): 173x48x9cm
4. N.W.(1pc): 7.4kg G.W.: 10kg
5. Load bearing: ≤159kg

Description:

1. It is a separately-type stretcher to transfer fractured and serious patients.
2. At both ends of the stretcher, a hinged and joined arrangement is set in the middle, which can set the stretcher into two parts and it is easy to lock and unlock.
3. The stretcher can strengthen the patients at original place; it can reduce the further hurt of the patients, the patients can be quickly & easily put in the stretcher and also the stretcher can be taken out from the back of the patient without moving the patient.
4. The length of the stretcher can be adjusted according to patients' body. The foot end is designed as a narrow-frame structure.

Intended use:

It is used to help the transportation of the injured people.

Attention:

When carrying patients with this stretcher, be sure to fasten the safety belts for safety sake.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Maintenance:

1. Keep clean as routine
2. Often check if any parts loose or not.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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