QUICK SPLINT

SET

REF **PNE244**

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GB INSTRUCTIONS

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E MODO DE EMPLEO

P INSTRUÇÕES

2021-7-30(A/1)
EU Regulation 2017/745(MDR)

PVS SPA VIA LEONARDO DA VINCI 18 20051 CASSINA DE PECCHI (MI) ITALY

PORTUGUES

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ENGLISH

INDICATIONS : Inflatable QUICK SPLINT are used to immobilise broken or injured limbs. This system is specially indicated to limit all movements during transportation and therefore enhances patient comfort and exerts a uniform pressure around the injury to control bleeding. QUICK SPLINT material permit the X ray procedures and therefore can remain in position throughout the X ray hence minimising the patient discomfort.

SPECIFICATIONS : QUICK SPLINT are produced in PVC , with zip for closed and open the splint . They should be inflated through the pump
Inflatable QUICK SPLINT are available in four sizes :

1 - LONG LEG

2 - ANCKLE-FOOT

3 - LONG ARM

4 - SHORT ARM - HAND

5 - PUMP

INSTRUCTIONS : How to fit the inflatable QUICK

SPLINT: put and wrap it around the injured limb and close the zipper. Blow the air through the pump

How to liberate the inflatable QUICK SPLINT : open the zipper , take out the air from it pressing the valve and take out the injured limb.

CAUTION : should be applied by skilled staff.

Please stop using if the material damage or air leakage is found.

CLEANING INSTRUCTIONS : after using the inflatable QUICK SPLINT , it should be accurately washed with water and soap. It can also be disinfected by a bactericide solution better if manufactured in according to new MDR 745/2017.

STORAGE : keep in a cool and dust free side

It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities

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