COGEL NS™

EN

Non-sterile lubricating gel

INSTRUCTIONS FOR USE

DESCRIPTION

Cogel NS is a water-based, biocompatible and hydrosoluble gel. It is a non-sterile, drug and active ingredient-free product for medical use in urological and gynecological diagnostic procedures, digestive endoscopy, insertion of rectal thermometers, enemas, etc. The effect of lubrication reduces friction between surfaces in contact, facilitating sliding. The product has no spermicide.

COMPOSITION AND INFORMATION ON INGREDIENTS

The gel contains: purified water, Propylene glycol, Hydroxypropyl methylcellulose, Sodium carbomer, Methylparaben, Propylparaben, Disodium EDTA.

INDICATIONS FOR USE

Open the bottle or tube by pressing the cap. When opening it for the first time, remove the warranty seal on the product. Apply the gel to the parts to be lubricated, making sure that a thin surface layer forms. Do not exceed the recommended dose. The lubricating effect is long-lasting.

CONTRAINDICATIONS

Cogel NS should not be used in people with known hypersensitivity to the ingredients mentioned above.

STORAGE CONDITIONS

Store the gel in a cool place and away from light. Do not freeze the product. The expiration date refers to correctly stored products in intact packaging. Do not use the product after that date.

WARNINGS AND PRECAUTIONS

- Do not apply Cogel NS on injured or damaged skin.
 - After use, remove the gel with water, cloth or paper towel.
 - Store it in a dry place, away from light.
 - Do not use it after the indicated expiration date.
 - Product residues and bottles/tubes containing the gel should be disposed of in accordance with the legislation in force.
 - Keep it out of the reach and sight of children.
 - Do not ingest the gel and avoid direct contact with the eyes.
- The safety data sheet is available upon request.

PREGNANCY AND BREASTFEEDING

Cogel NS does not contain any drugs or active ingredients. There are no indications that require limitations of use. However, do not exceed the recommended dose of the product.

EFFECTS ON ABILITY TO DRIVE AND USE MACHINES

The gel does not contain any substances that may impair the ability to drive and operate machinery.

SIDE EFFECTS

By using the product properly and complying with what is indicated in the "WARNINGS AND PRECAUTIONS" section, side effects are not known to occur. If necessary, consult your doctor or pharmacist and show them this instruction sheet and the Cogel NS safety data sheet.

DRUG INTERACTIONS

Cogel NS does not contain any drugs or active ingredients.

OVERDOSE

There is no risk of overdose, Cogel NS does not contain any drugs or active ingredients. It is advisable not to exceed the recommended dose of the product and remove residue after use with water, cloth or paper towel.

STERILIZATION

The container and the product are NOT sterile.

PACKAGING

The gel is supplied in disposable PE bottles or tubes.

 CNS0082MT048	Cogel NS 82 g (48 tubes per pack)
 CNS0150MT048	Cogel NS 150 g (48 tubes per pack)
 CNS0250MF036	Cogel NS 250 g (36 bottles per pack)

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