**DERMATOSCOPIO GIMA XENON ALOGENO - 10x**  
**GIMA XENON HALOGEN DERMATOSCOPE - 10x**  
**DERMATOSCOPE GIMA XÉNON HALOGÈNE - 10x**  
**DERMATOSCOPIO GIMA XENON HALÓGENO - 10x**  
**DERMATOSCÓPIO GIMA XENON HALOGÊNIO - 10X**  
**DERMATOSCP GIMA XENON HALOGEN - 10x**  
**DERMATOSKOP GIMA XENON HALOGEN - 10x**  
**GIMA XENON HALOGÉN DERMATOSZKÓP - 10x**

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### ENGLISH

#### Attention:

Thank you for purchasing our product. The dermatoscope meets the most stringent requirements concerning material selection, performance, safety and final inspection. This user manual applies to "GIMA 2000" brand dermatoscopes. These instructions must be followed to ensure the performance and durability of the dermatoscope. The operator/user must thoroughly read and understand this manual in order for the product to operate in a long-lasting and reliable manner for a defined period of time.

After opening the package, we recommend you check all the components, referring to the standard configuration. Check that none are missing and that they are in perfect condition.

#### Intended use:

The dermatoscope is used for a preoperative assessment of pigmented skin lesions with the purpose of differentiating primary melanoma, which requires excision, from non-melanoma pigmented lesions that may safely be left untreated.

#### Patient population:

The dermatoscope is designed and intended for the examination of skin across the entire patient population.

#### Features:

The dermatoscope is equipped with optical magnifying lenses that enhance the view of the skin surface during the examination. The dermatoscope head is used together with the battery-powered grips for external skin examination.


The dermatoscope is intended for transient use and examination.

#### Features of all dermatoscopes

|               |                                                       |
|---------------|-------------------------------------------------------|
| Devices       | GIMA 2000                                             |
| Models        | Standard                                              |
| Lock          | Bayonet lock                                          |
| Magnification | 10x                                                   |
| Focal length  | 25 mm                                                 |
| Configuration | Replaceable 2.5 V Xenon / Halogen lamp                |
| Batteries     | 2 x "AA" 1.5 V batteries 2.5 V rechargeable batteries |

Note: All accessories for the various dermatoscope models are listed separately in the section dedicated to accessories.

#### Product overview: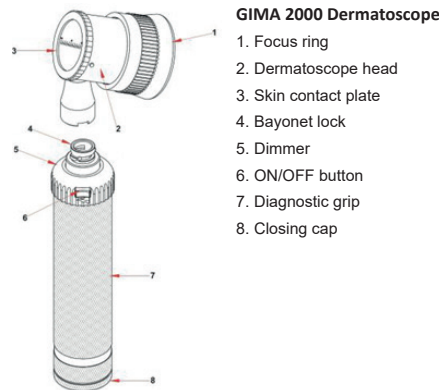

### Instructions for Use

Attach the dermatoscope head and grip relative to the respective lock configuration. Support the dermatoscope head and grip by each handle, align both parts with each other, and assemble them. See Fig. no. 1

1. In the GIMA dermatoscope, align the bayonet grooves on the head of the dermatoscope with those on the grip. Rotate the head or grip in the opposite direction to the other to ensure a secure connection
2. Check that the connection between the head and the grip is perfect.
3. Check the proper working order of the dimmer on the GIMA dermatoscope.
4. On the GIMA dermatoscope, press the button on the grip and rotate the knob clockwise. The light will turn on and the dermatoscope will be in the operating position. The "ON" position of the grip is indicated by the red indicator. The light intensity can be adjusted by turning the knob clockwise or counter-clockwise as required.
5. Gently place the contact plate on the lesion, so that this is in the centre of the contact plate, then examine its surface structure. To reduce reflection from the stratum corneum, apply oil (mineral oil, petroleum jelly etc.).
6. With your index finger on the focus ring, adjust for optimal focus while your eye is on the eye rest.
7. To measure pigmented skin lesions, you must use the graduated contact plate.
8. Switch off the device after use to preserve the batteries. On the GIMA dermatoscope, rotate the knob counter-clockwise. A click will be heard when the button reaches the OFF position.

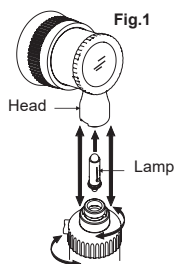
Fig. 1

The dermatoscope is intended to be used for transient examination. The grip may warm up, but the maximum temperature will not exceed 48°C. The contact plate is an applied part.

The dermatoscope may only be used by trained personnel and by duly qualified medical personnel in a healthcare facility. It cannot be used by ordinary people.

Do not use the equipment if it is damaged. Avoid precarious repairs. Repairs must be carried out only with original spare parts, which must be installed according to their intended use. For any other repairs/replacements, contact the dealer.

### Magnifying glasses:

All the dermatoscopes are equipped with 10x magnifying glasses. Magnifying glasses are used to magnify the object under examination. The user rotates the focus ring to adjust the magnification of the skin being examined.

### Accessories:

Skin contact plates:

The skin contact plate is an essential accessory of the der-

matoscope. Skin examination can only be carried out when the skin contact plate is attached to the dermatoscope head. There are two types of skin contact plates supplied and they must be attached to the dermatoscope head as shown in fig. no. 2.

- 1 Flat contact plate
- 2 Graduated contact plate

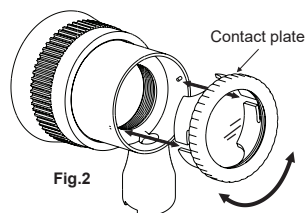
Fig. 2

1. Do not use the dermatoscope without the skin contact plate as it may contaminate the device and cause cross-infection.
2. Inspect the skin contact plate before each use for rough edges or surfaces.
3. Do not reuse the skin contact plate without cleaning/sterilising it, as it may cause cross-infection.

### Replacement procedures:

#### Lamp replacement procedure:

All these dermatoscope models contain LED and halogen/xenon lamps. The replacement procedure for the lamp and LED is given below. Switch off the device and disassemble the dermatoscope head with the grip. Pull the lamp downwards/outwards from the lamp/LED holder as shown in fig. no. 3.

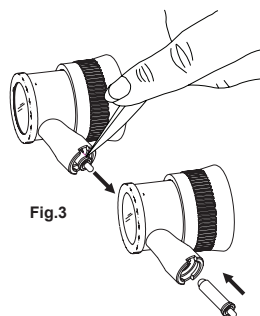
Fig. 3


Always switch off the device before unlocking the dermatoscope head from the grip and allow the lamp to cool before touching it for replacement. Otherwise, there is a risk of serious burns from touching the hot lamp.

### Replacing the batteries:

Switch off the device and unscrew the grip cap, then remove the old batteries as shown in Fig. no. 4. Insert the new batteries as per the procedure detailed in Fig. no. 4. The GIMA dermatoscope is powered by two "C" type batteries; the rechargeable version is powered by two 1.5 V rechargeable alkaline or "C" type batteries. Take care that the poles point in the correct direction as shown, i.e. the positive pole takes precedence when inserting both normal and rechargeable batteries.

Battery replacement must always be carried out by qualified personnel. When inserting the lithium-ion battery, always follow the marking on the battery label.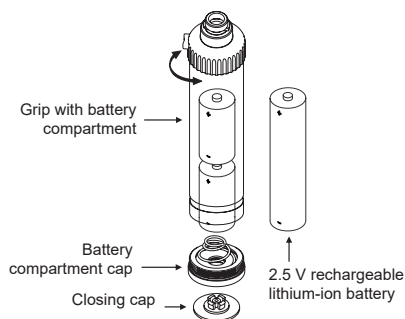

### Rechargeable system:

All these dermatoscopes can be converted into a rechargeable system. Insert the rechargeable batteries into the respective grip, as described in the battery replacement procedure. The charging procedure for each grip is described below: The GIMA 2000 dermatoscope diagnostic grip, together with the rechargeable battery, can be charged using the Zoom charger.

Note: For further details, please read the Zoom and Zi-mini charger IFUs.

### Cleaning and Disinfecting

#### Dermatoscope head:

Clean and disinfect the external surface of the dermatoscope head with a soft cloth and the internal surface with a cotton swab dipped in 70% isopropyl alcohol or water. Do not immerse the head in any disinfectant solutions.

#### Lenses:

Lenses can be cleaned using a cotton-tipped applicator moistened with alcohol or a solution of water and mild detergent.

Do not use an abrasive ultrasonic cleaner, alcohol or other chemicals to clean the lenses as they will degrade the quality and scratch the coated surface of the lenses.

Do not use a rough cloth to clean the lens to avoid scratches.

#### Skin contact plate:

The skin contact plate must be removed after each examination. It can bear cleaning or disinfection by soaking in alcohol solution, Cidex etc. It can also be cleaned with a 70% isopropyl alcohol solution. Skin contact plates can also be autoclaved. Autoclave the skin contact plate at 134°C for at least 5 minutes, or at 121°C for 30 minutes.

Note: Do not exceed a temperature of 135°C and pressure of 28 psi.

#### Grip:

The grip can be cleaned with a cloth soaked in disinfectant. However, it cannot be immersed/soaked in liquids. Remove batteries before cleaning.

#### Operating Environment:

Temperature limits Humidity limits Atmospheric pressure limits

### Storage/Transportation Conditions:

### Maintenance:

The dermatoscope does not require routine maintenance. However, the light source, i.e. the lamp or LED, can be replaced according to the replacement procedure detailed above.

### Support:

The device has no components that can be repaired by the end user.

### General Precautions/Warnings:

1. Read the label before unpacking the product.
2. Do not use the device if it is damaged. Contact your dealer/distributor.
3. Periodically check the condition of the batteries, verifying that there are no signs of corrosion or oxidation. If necessary, replace them with new ones.
4. Handle batteries with care, as the liquids can irritate skin and eyes.
5. Check the product carefully before use. The same checks must be carried out after cleaning.
6. Check that the connection between the head and the grip is perfect and that the on/off button works correctly.
7. If the light works intermittently or if it does not turn on at all, check the lamp/LED, batteries and electrical contacts.
8. This product must not be introduced or used in areas with strong magnetic fields, e.g. near MRI scanners.
9. Any unauthorised modification/repair by the user is strictly prohibited, therefore we advise always contacting the person in charge or returning the unit to the authorised agent.
10. Autoclaving or flash sterilisation of the dermatoscope head and grip with battery compartment will cause serious electrical and mechanical damage; it is therefore strictly prohibited.
11. Ultrasonic cleaning is not recommended.
12. Store the device in a clean environment and as recommended.

### Shelf life:

The useful shelf life of the dermatoscope is 5 years.

**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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