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INSTRUCTION FOR USE

Group Name: Besmed Disposable Cushion Mask Models:

- CM-65810 - ULTRA-SMT Anesthesia Mask, w/o hook, Size: 1, Infant
- CM-65820 - ULTRA-SMT Anesthesia Mask, w/o hook, Size: 2, Pediatric
- CM-65830 - ULTRA-SMT Anesthesia Mask, w/o hook, Size: 3, Small, Adult
- CM-65840 - ULTRA-SMT Anesthesia Mask, w/o hook, Size: 4, Medium, Adult
- CM-65850 - ULTRA-SMT Anesthesia Mask, w/o hook, Size: 5, Large, Adult
- CM-65504 - Air Cushion Mask, Green Hook, Size: 0, Neonatal
- CM-65514 - Air Cushion Mask, Pink Hook, Size: 1, Infant
- CM-65524 - Air Cushion Mask, Red Hook, Size: 2, Child
- CM-65534 - Air Cushion Mask, Yellow Hook, Size: 3, Adult
- CM-65544 - Air Cushion Mask, White Hook, Size: 4, Adult
- CM-65554 - Air Cushion Mask, Blue Hook, Size: 5, Adult
- CM-65564 - Air Cushion Mask, Transparent Hook, Size: 6, Adult
- CM-65364 - Disposable Air Cushion Mask, w/ hook, Size: 4, Extra Large, Adult

Intended Use

Besmed Disposable Cushion mask is intended to channel anesthesia gases or mixture of oxygen to the patients for inhalation via patient's mouth and nose. Anesthesia masks are to be used in conjunction with anesthesia and ventilator breathing systems. It provides a seal such that positive pressure from the positive pressure source is directed into the patient's nose and mouth.

Product Description

Besmed Disposable Cushion mask is designed for connection to breathing circuit for use as an adjunct to artificial respiration and cardiopulmonary resuscitation, automatic resuscitators, ventilators. Its a non-invasive method to either induce or maintain general anesthesia. It also used in resuscitation procedures as an effective mode of ventilation for unconscious patients. According to patient category, there are six different sizes are available from neonates to adult. Anti-slip design for easy holding. Transparent dome provides clear observation of patient's condition and anatomical shape with soft touch provides better seal and patient comfort.

Technical Specifications of Besmed Disposable Cushion mask

Group name	Disposable Cushion Mask	
Manufacturer	Besmed Health Business Corp.	
Device name	Ultra-SMT Anesthesia Mask	Air Cushion Mask
Reference No.	CM-65810, CM- 65820, CM-65830, CM-65840, CM-65850	CM-65504, CM-65514, CM- 65524, CM- 65534, CM-65544, CM-65554, CM-65564

Patient category	Patient Group	Anesthesia Mask	Air Cushion Mask
	Neonate	-	CM-65504
	Infant	CM-65810	CM-65514
	Pediatric / Child	CM-65820	CM-65524
	Adult	CM-65840, CM- 65850, CM-65860	CM-65534, CM- 65544, CM-65554, CM-65564

Material	TPR, PP, PVC		
	Mask dome: Transparent		
Color and specifications	Patient group (size)	Mask cuff (Anesthesia Mask)	Hook color (Air cushion mask)
	Taglia 0	Transparent	Green
	Taglia 1	Pink	Pink
	Taglia 2	Yellow	Red
	Taglia 3	Green	Yellow
	Taglia 4	Red	White
	Taglia 5	Blue	Blue
	Taglia 6	-	Transparent
Component type	Accessory		
Device type	Disposable		
Patient usage type	Single patient use		
Patient weight	Taglia 0	< 2 kg	
	Taglia 1	2 ~ 10 kg	
	Taglia 2	10 ~ 25 kg	
	Taglia 3	25 ~ 60 kg	
	Taglia 4	60 ~ 100 kg	
	Taglia 5	>100 kg	
Specifications	Engagement of 22 mm latching sockets to cones	Did not disconnected at (50 ± 5) N for 10 s	
	Leakage from 22 mm latching sockets	No signs of leakage at (8 ± 0.5) kPa	
	Drop procedure for 22 mm latching sockets	No signs of damage	

Latex/DEHP content	None		
Package	Non-sterile package		
Connection diameter	Diameter	Patient group (size)	
	15 mm	Size 0, Size 1	
	22 mm	Size 2, Size 3, Size 4, Size 5	
Operation temperature	25 ± 5°C, Room Temperature		
Storage temperature	-20°C ~ 60°C		
Relative humidity	Da 30 - 60% UR		
Shelf life	5 years		
Attestation	Test Item	Guideline	Result
	Conical connectors	EN ISO 5356-1:2015	PASS
	Cytotoxicity	EN ISO 10993-5:2009	PASS
	Sensitization	EN ISO 10993-10:2023	PASS
	Skin Irritation	EN ISO 10993-23:2021	PASS
	Usability	IEC 62366-1	PASS

Intended users

Medical personnel received the training of respiratory care.

Indication

Besmed Disposable Cushion mask is designed to ensure a tight seal around the patient nose and mouth to deliver oxygen gases to the patient. Besmed Disposable Cushion masks are transparent to facilitate visual inspection of breathing condition of the patient. Signs of inadequate ventilation can be identified early such as reduced chest movement, absent or quiet breathing sounds, patient cyanosis, audible gas leaks. Also, late signs of inadequate ventilation include haemodynamic consequences of hypoxaemia or hypercarbia.

Instruction for use

Check for clean airflow passage prior to use
 Connect to breathing circuit or resuscitation with appropriate fitting.
 Set the gas to the proper flow and check for gas flow through the device
 Position the elastic strap behind the neck.
 Make sure all connections are compatible and fit firmly.
 Gently pull the ends of the strap until the mask is secure.

Contraindications

Besmed Disposable Cushion mask is possibly not suitable for the use to the patients with recent facial, oral, skull surgery, or trauma, and patients who have claustrophobia, vomiting, or active GI bleeding. Inability to maintain an adequate seal between the mask and the patient's face, which may result in inadequate oxygenation.

Risks/Side Effects

Inappropriate size of mask and strap tension may cause discomfort.

Mask related complication includes claustrophobia, ulceration, facial rash, or itching.

Cramped hands and tired muscles and limits the user's ability to perform other tasks especially in obese and patients with a larger head.

Warning/ Caution

Read this instruction before use.

Do not use if the package is open or damage or contaminated.

Any skin redness, itching or any abnormal situation happens, stop using this mask.

Single use devise only. Do not re-use.

Do not soak, wash, rinse or sterilize the product.

Any serious incident that has occurred in relation to the device should be reported to Besmed or its Authorized Representative/competent authorities and the Regulatory Authorities of the country.

Disposal

Dispose of in accordance with local regulations.

Legend of symbols

	Medical Device compliant with Directive 93/42/EEC		Product code
	Imported by		Manufacturer
	Please read instructions carefully		Date of manufacture
	Lot number		Disposable device, do not re-use
	Expiration date (see box / package)		Keep in a cool, dry place
	Keep away from sunlight		Authorized representative in the European community

	Not made with natural rubber latex		No-DEHP formulation
	Temperature limit		Non-sterile
	Medical Device		Disposable device, do not re-use

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.