GIMA

PROFESSIONAL MEDICAL PRODUCTS

CINGHIE MONITOR FETALE RIUTILIZZABILI
REUSABLE FOETAL TRANSDUCER BELTS
SANGLES RÉUTILISABLES POUR TRANSDUCTEUR FOETAL
CORREAS PACIENTE INDIVIDUAL PARA Sonda FETAL
CORREIA FETAL TRANSDUTORA REUTILIZÁVEL
EINWEG-GÜRTEL FÜR FETALEN SCHALLKOPF
ΕΠΑΝΑΧΡΗΣΙΜΟΠΟΙΟΥΜΕΝΗ ΖΩΝΗ ΜΟΡΦΟΤΡΟΠΕΑ
ΕΜΒΡΥΪΚΟΥ ΕΛΕΓΧΟΥ
EENMALIGE FOETALE TRANSDUCER RIEM
PASEK PRZETWORNIAK PŁODU DO WIELOKROTNEGO
UŻYTKU
CUREA ABDOMINALĂ PENTRU MONITORIZARE FETALĂ,
REFOLOSIBILĂ
ÅTERANVÄNDBART FOSTERGIVAREBÄLTE

REF B08 (GIMA 29545)

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante / Hersteller /
Κατασκευαστής / Fabrikant / Producent / Producător / Tillverkare:
Dongguan Belcare Medical Technology Co., Ltd.
402 of 4th. Block, #3 of Keyuan Road, Tangxia Town,
Dongguan, Guangdong, China 523710 - joe@belcare-med.com
Made in China

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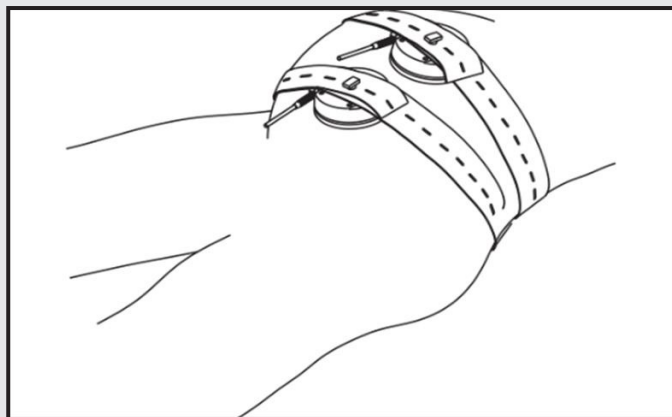
Intended use:

- This product works according to adjust buckle's place on the belt to fix the fetal transducer.

Product introduction:

- Product name: Reusable fetal transducer belt
- Product structure: It's made up of elastic belt and buckle.

Product range:

- It used with fetal transducer when pregnant in fetal monitoring, to fix the fetal transducer.

Use condition:

- Temperature: 0°C~40°C
- Relative humidity: 15%~95%
- Atmospheric pressure: 70kpa~106kpa

Package and storage:

- This fetal belt should be single packaged, packaged product should be stored in a non-corrosive gas, good ventilation room.

Store environment as below:

- Temperature: -10°C~+40°C
- Relative humidity: ≤80%
- Atmospheric pressure: 70kpa~106kpa

Installation/use:

- Attach one end of the belt to the transducer by engaging one of the holes in the belt over the button on the top of the transducer.
- Keeping the transducer in position, tension the other end of the belt and engage the belt over the button, ensuring sufficient tension to keep the transducer in firm contact with the abdomen.
- Avoid over-tightening as this will cause unnecessary discomfort to the patient.

Product clean:

- Wash belt in warm water, do not rub the edge fetal belt to avoid removal, do not exposure.

Validity period:

- Validity period: 2 years.

Replace and abandon:

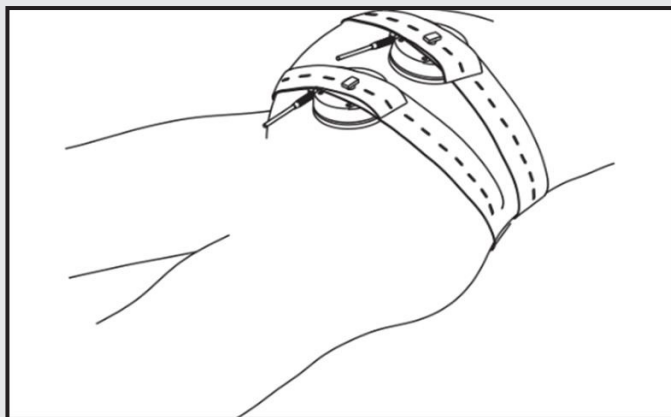
- Replace: Under these situations such as fracture, elastic damage, expiry date, please change new fetal belt to use.
- Abandon: Abandoned belt can be processed as normal waste.

Contraindications, precautions, warnings and cautionary statements:

- Contraindications: The belts must not be fixed at the injury site.
- Spandex allergies to stop using.
- Cautionary statements: Please read the instructions before use.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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