PROFESSIONAL MEDICAL PRODUCTS

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Jiangsu Suyun Medical Materials Co., Ltd. No.18
Jin Qiao Road Dapu Industrial Park 222002 Lianyungang
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA
Made in China

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ENGLISH

Intended purpose: The main clinical usage of Sterile gynaecological collectors for single use is to sample for gynecology and venereal disease.

Indications: Women who need gynecological examination, such as vaginal examination and cervical examination.

Method:

- Use the dilator to open the vagina at an appropriate angle.
- Determine the location to be inspected or sampled.
- Deepen the sampling head of the gynecological sampler to the sampling site and take a rotating or wiping flat method to sample.

Contraindications: No special contraindication.

Caution:

- When the product is used, it should strictly implement the requirements of aseptic operation specifications and related regulations, and be used by trained doctors and nurses. In the course of use, the operator or user's body should be treated in a timely manner according to medical regulations.

- If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.
- This product is used for single use and is destroyed after using; if the small package is damaged and the product is prohibited to be used; this product is sterilized by ethylene oxide and the product is sterile; the product is valid for 5 years and should not be used after it expires.
- Appropriate types and specifications should be selected according to the patient's body type.
- If there are burrs, cracks, or nodes in the application area of the gynaecological collector, they should not be used.
- This product shall be disposed of according to the requirements of waste disposal in the local medical regulations after using.

Storage: The product should be stored in a room with no corrosive gas and well ventilated, with adequate protection for the product.

Any serious incident occurring in relation to the medical device provided by us must be reported to the manufacturer and to the competent authority of the Member State in which you are established.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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ČEŠTINA

Intended purpose: The main clinical usage of Sterile gynecological collectors for single use is to sample for gynecology and venereal disease.

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Method:

a. Use the dilator to open the vagina at an appropriate angle.

b. Determine the location to be inspected or sampled.

c. Deepen the sampling head of the gynecological sampler to the

sampling site and take a rotating or wiping flat method to sample.
Contraindications: No special contraindication.

Caution:

- When the product is used, it should strictly implement the requirements of aseptic operation specifications and related regulations, and be used by trained doctors and nurses. In the course of use, the operator or user's body should be treated in a timely manner according to medical regulations.
- If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.
- This product is used for single use and is destroyed after using; if the small package is damaged and the product is prohibited to be used; this product is sterilized by ethylene oxide and the product is sterile; the product is valid for 5 years and should not be used after it expires.
- Appropriate types and specifications should be selected according to the patient's body type.
- If there are burrs, cracks, or nodes in the application area of the gynaecological collector, they should not be used.
- This product shall be disposed of according to the requirements of waste disposal in the local medical regulations after using.

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